# AESCULAP® DENTAL

HAUPTKATALOG  
MAIN CATALOG

# AESCULAP® DENTAL

COMPETENCE IN DENTISTRY

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Dentistry is currently facing drastic changes. The current state of medical and technological knowledge means that operations can be performed with deepening precision and using less and less invasive techniques.

Along with the availability of increasingly sophisticated diagnostic tools, the decisive factor underlying this progress is the parallel ongoing development of the entire range of dental instruments.

AESCULAP®'s products are developed strictly by the close contact to our customers. This applies as much to new and innovative areas such as microsurgical instruments for implantology, periodontics and endo-

dontics, as to standard dental instruments in general. Throughout the world, AESCULAP® products have proved themselves in day-to-day practice, not least from the point of view of quality and cost. With limited investment budgets available, optimized hospital procedures and resulting efficiency gains offer a means of making better use of existing resources. AESCULAP®'s range of services can help to achieve such improvements.

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Good functionality

- Precise manufacturing and high class materials lead to high quality and reliability
- The working tips are matched to the requirements of dentistry

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Comfortable ergonomics

- Shape and thickness ensure low-fatigue work
- Lightweight handles

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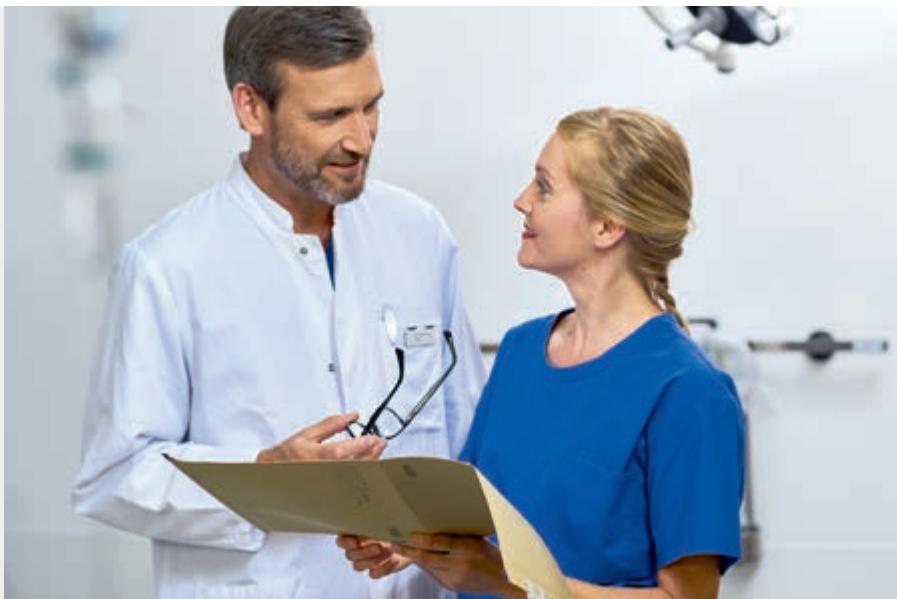
Aesthetics

- The winner of the **iF DESIGN AWARD 2015** Category Medicine/Healthcare

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**AESCULAP ACADEMY –**  
Forum for contemporary medicine.

The Aesculap Academy is one of the leading medical forums for everyone who is professionally, passionately and ambitiously committed to people's health. To those medical professionals we offer top quality knowledge transfer based on globally recognized quality criteria using innovative methods and technologies.

The Aesculap Academy has its roots in the company B. Braun, which has been protecting and improving people's health for more than 175 years. With our courses, hands-on trainings and symposia we help to honor our parent company's promise of SHARING EXPERTISE.

The courses at the Aesculap Academy offer participants who want to continue to learn in an inspiring environment knowledge transfer and teaching that are adapted to real life: lifelong learning, true-to-life training situations and realistic content for a better life for patients and medical staff.

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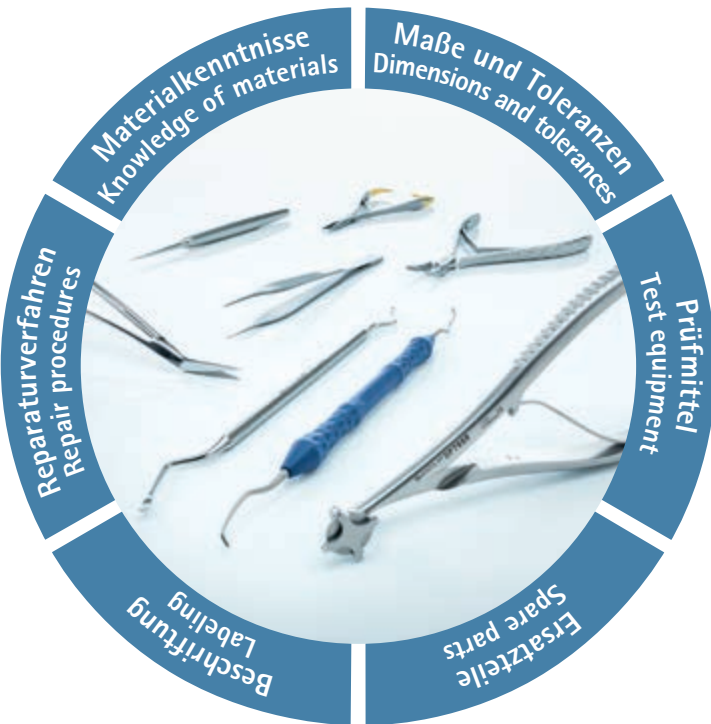
### Characteristics of an AESFULAP® quality repair

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#### Repair procedures

- High-quality repair procedures
- Work process conforming to the manufacture of the new product
- Access to current original drawings and modification information

#### Dimensions and tolerances

- Clearly-defined repair limits in accordance with AESFULAP® quality standards
- Minimum size specifications for the reliable use of an instrument set
- Preservation of model consistency even after multiple repairs

#### Spare parts

- Exclusive use of AESFULAP® original spare parts
- Safeguarding of functionality and durability
- Prevention of corrosion and changes in the material

#### Knowledge of materials

- Processing methods are adapted to the composition of the material
- Reworking of surfaces complies with valid procedure for new products
- Exclusive use of validated auxiliary materials

#### Test equipment

- Complete control after the repair
- Guarantee of functionality by testing all important features
- Use of the same test steps and parameters used for new products

#### Labeling

- Restoration of original AESFULAP® labeling
- Application of ATS repair labels with month and year of repair

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## Planning and Consulting

In an age of managed care, quality assurance circles and rapid progress in the field of minimally invasive techniques, the future has already begun today. This future vision invariably also means investment in human resources and technology.

Limits on financial resources in the healthcare sector and complex documentation and settlement procedures mean that decisions on such investments are becoming increasingly important, but at the same time more and more difficult.

*Investment and future* – two concepts that have now become more important than ever before. For only those who make the

right investments today will be in the position to reap the benefits of those investments tomorrow.

Having devoted particular attention to the demand for competent and fair advice, AESCULAP® has continually expanded this area of service. Workable planning for hospital surgical departments depends partly on sound analyses of the actual situation in and around the operating theatre.

Plans to expand, strengthen or restructure facilities can only be addressed properly if you are, for example, aware of the current status of your instruments.

Especially, if new long-term objectives are envisaged, there is no alternative to a detailed appraisal of the status quo. This is where our AESCULAP® planning and advice team can help. We analyse the quality, condition and further usability of your present set of instruments quickly, thoroughly, competently and independently. We then work with you to develop facility criteria indicating the most efficient way to put your plans into practice.

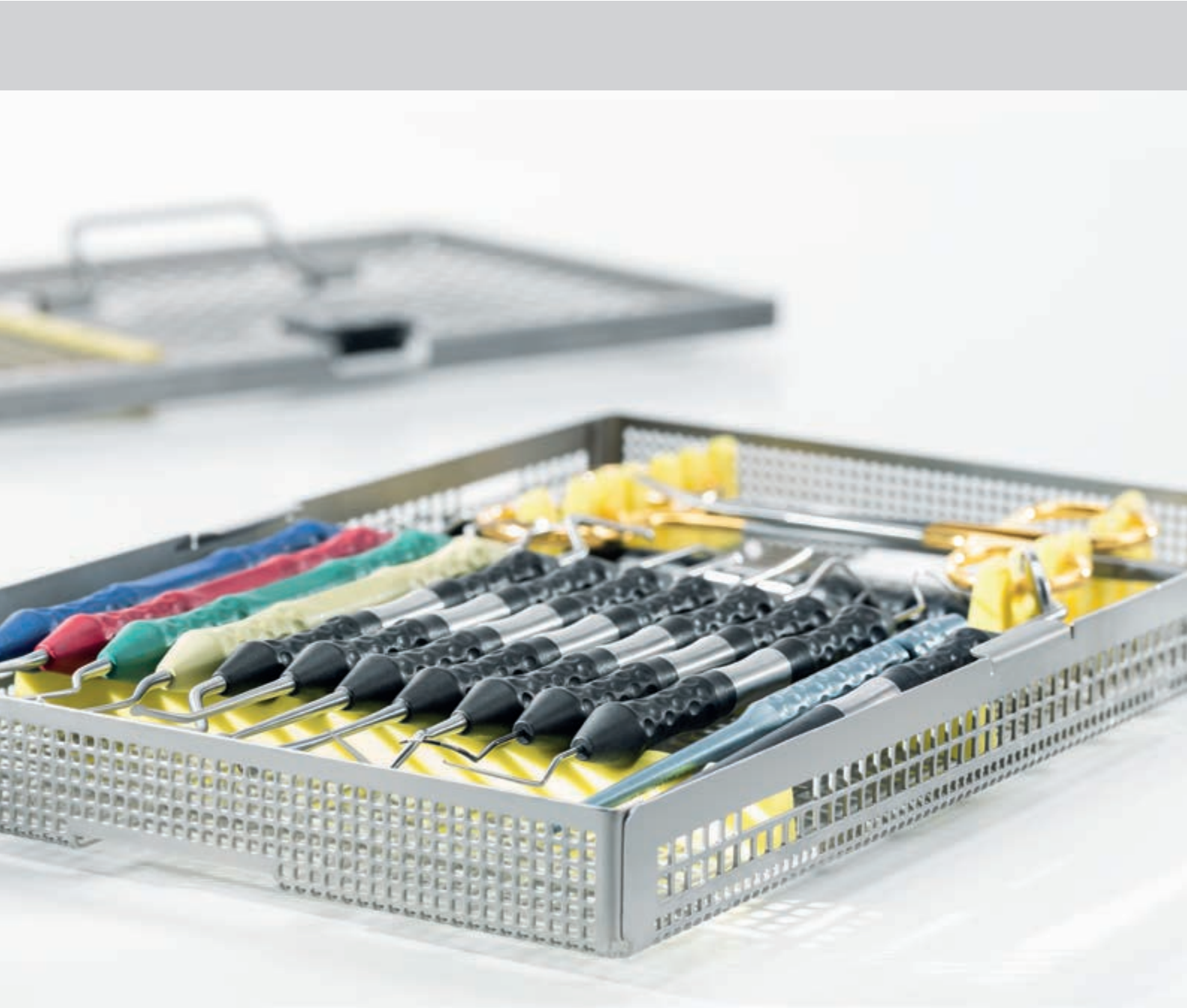
Although every hospital and dental practice has its own individual problem profile, there are standard situations applicable to the planning of operating theatre instrument trays. AESCULAP® can call upon decades of experience for creating a useful tool to provide you with fast, practical solutions to problems – a surgical instrument 'library', that you can fall back on if and when the specific need arises.

We have given the utmost priority to ensure that our planning and advice enables you to take your decisions on a basis of maximum certainty.

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
AESCULAP® „Surface“

For mid luminous conditions like  
diagnostics, scaling, etc.

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AESCULAP® „Rhodium“

For higher luminous conditions  
like surgery.

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Suitable for mechanical  
cleaning in the assembled  
state.

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### Make yourself comfortable.

You'll never want to put them down again – the new instruments in AESCULAP®'s Ergoprobe range make working that much easier, more relaxed and less tiring. The secret lies in the extremely lightweight, ergonomic handles with surfaces specially designed to have a particularly pleasant feel.

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Ergoprobe combines design with comfortable ergonomics and top functionality

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#### Good Functionality

- Dental probe working ends made of Phynox steel  
→ highly gracile | elastic | dimensionally stable
- Periodontal probes are coated with TiAl-nitride  
→ accurate marking | smooth transitions

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#### Comfortable ergonomics

- Shape and thickness ensure low-fatigue work
- Extremely lightweight handles

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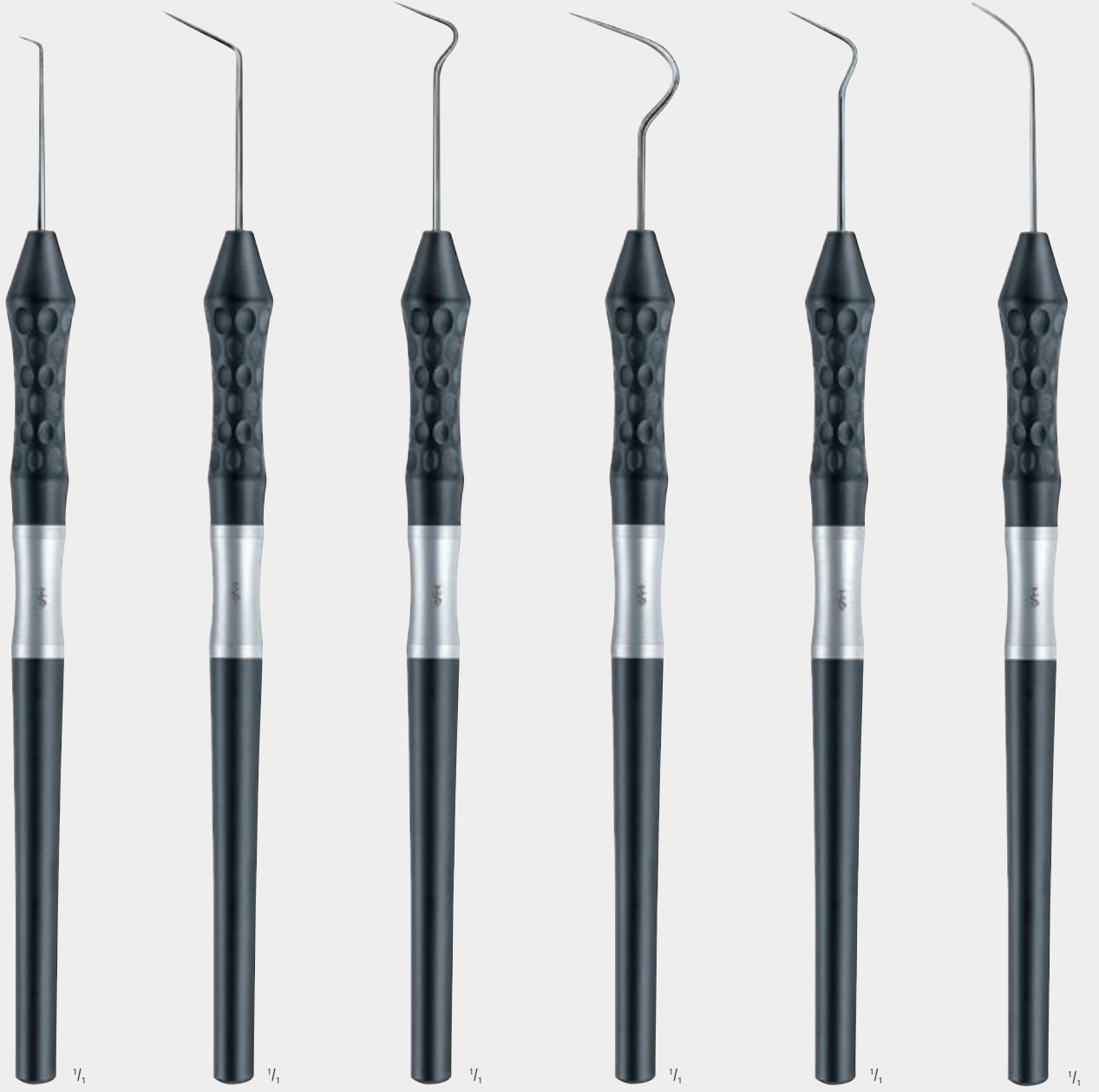
#### Aesthetics

- The winner of the iF DESIGN AWARD 2015 Category Medicine/Healthcare

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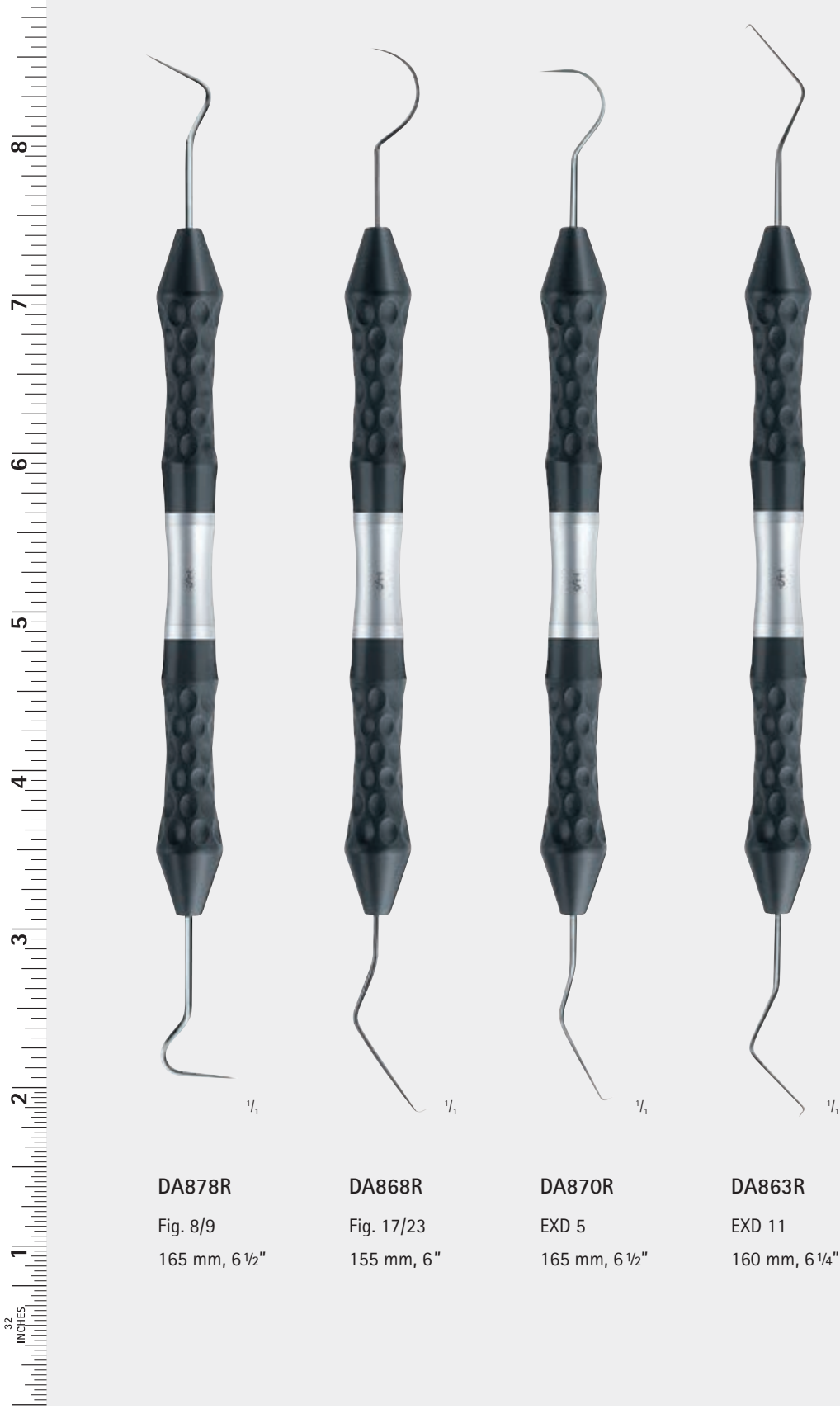
Due to a lack of measurement pressure standardisation, clinical measurements taken using different probing techniques often provide contradictory results. Objective evaluation of periodontal status is only possible when using pressure-sensitive periodontal probes. When assessing periodontal disease, probe measurements are subject to numerous instrumental and subjective influences, which can often result in underestimation of actual probing depths. When too little pressure is used, the main limiting factor on measurement probes is soft-tissue morphology rather than proportions of coronal, supra-alveolar fibre structures. Thanks to their defined spring pressure of 0.2 N, the DB764R and DB765R periodontal probes allow standardised probing in follow-up care of periodontal diseases, and especially in cases of inflammatory gingival changes in long-term studies.

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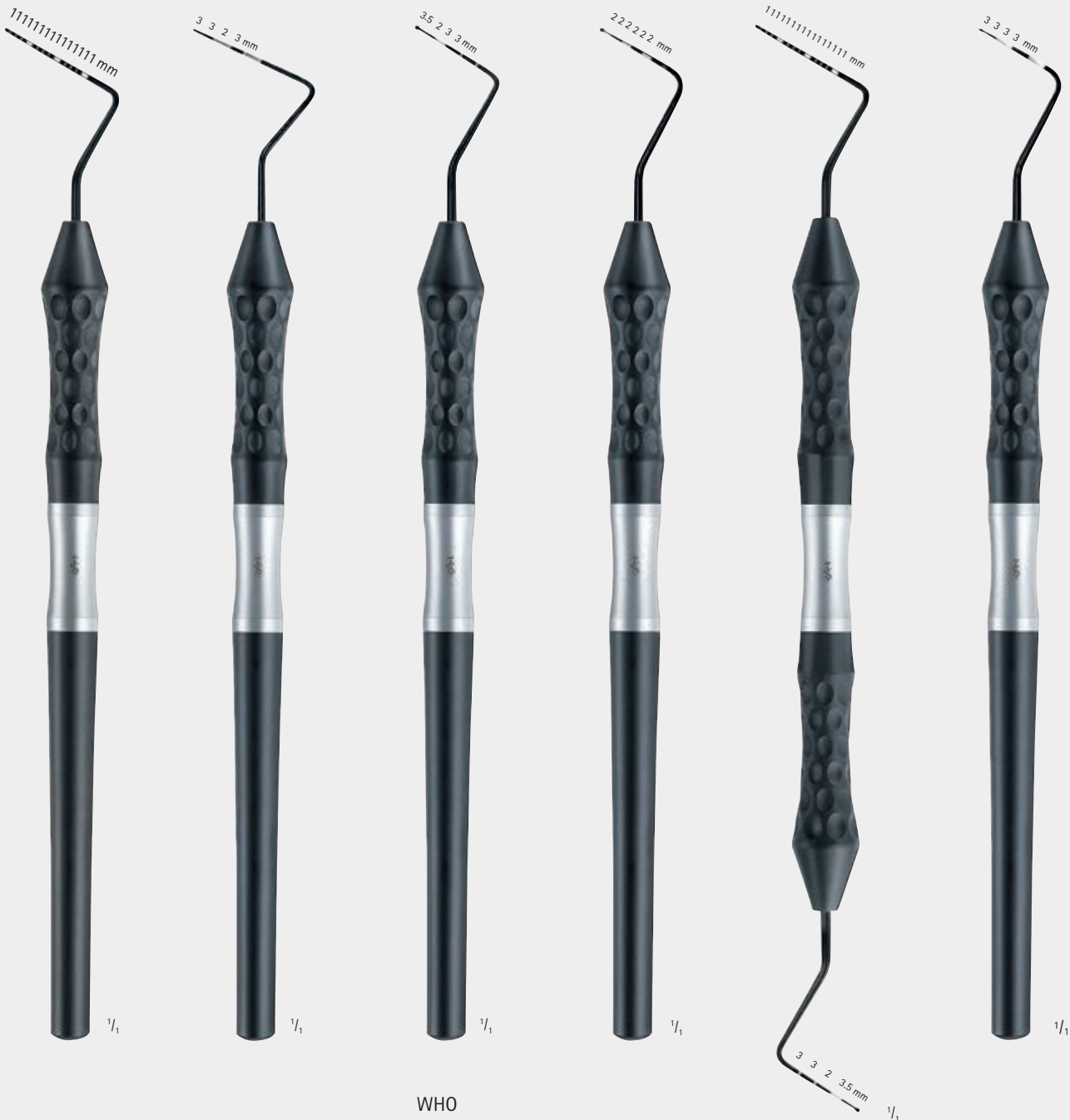
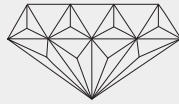
Special tooth tweezers LONDON-COLLEGE, mod.  
Particularly strong, distortion-free design, serrated.

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DUROGRIP®-Tooth forceps with tungsten carbide insert for secure grasping of matrices and interdental wedges.

**DA246R**  
Zahnpinzette  
Dental forceps  
150 mm, 6"

**DA254R**  
150 mm, 6"

The Winner of the  
**iF DESIGN AWARD 2015**  
Category Medicine / Healthcare.

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Colour coding system, coated with TiAl-nitride

■ exact marking and smooth transition

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Colour coding system, coated with TiAl-nitride  
■ exact marking and smooth transition

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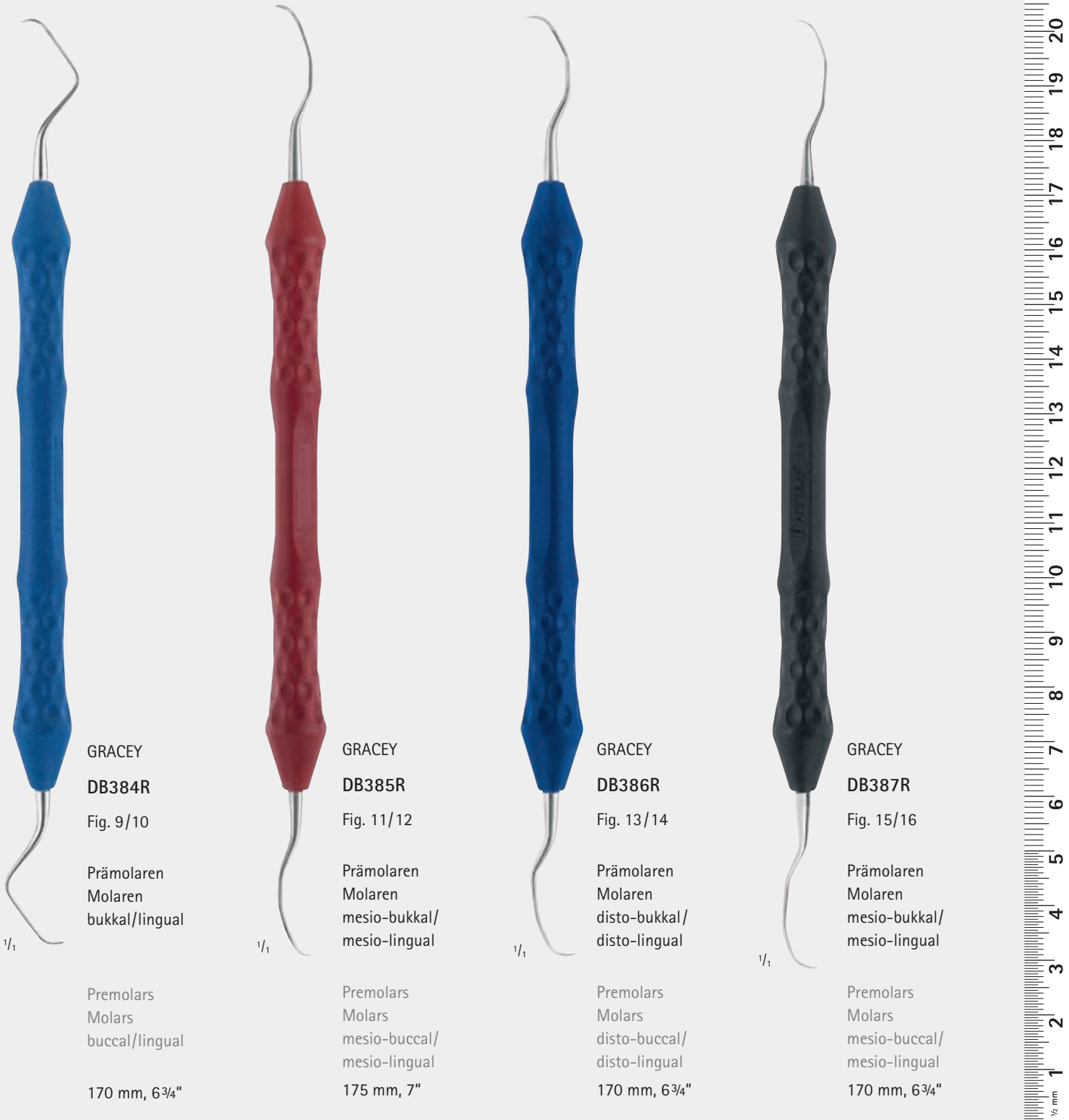
The combination of high rigidity and filigree-fine working ends means that stubborn plaque in particular can be removed thoroughly and efficiently with rigid curettes. It is no longer necessary to carry out time-consuming repeat scaling with flexible curettes. Flexible curettes should always be used to complete root planing.

The Gracey Figures 12/13 and 11/14 prove to be particularly time-saving, since using Figure 12/13 all mesio-distal surfaces in the side tooth area, e.g. in the first quadrant, can be completely treated in vestibular direction in one step without having to change the instrument. It is only with the change to palatinal direction that Figure 11/14 is used. The second quadrant begins of course with Figure 11/14 for mesio-distal in vestibular direction, etc.

SPEZIELLE KOMBINATION VON GRACEY-FIGUREN, RIGID GRACEY-KÜRETTEN  
 SPECIAL COMBINATION OF GRACEY-FIGURES, RIGID GRACEY-CURETTES

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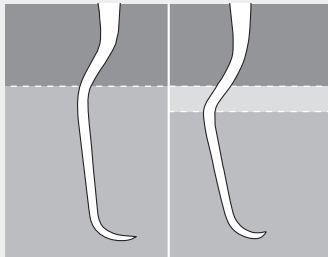
Shaft 3 mm longer than  
Standard-GRACEY narrow  
working tip

■ for easier access to the deep  
root surfaces

After-GRACEY  
DB590R  
Fig. 1/2  
Frontzähne

universal

Anteriors

universal

170 mm, 6<sup>3</sup>/<sub>4</sub>"

After-GRACEY  
DB591R  
Fig. 3/4  
Frontzähne  
Prämolaren

universal

Anteriors  
Premolars

universal

170 mm, 6<sup>3</sup>/<sub>4</sub>"

AFTER-GRACEY-KÜRETTEN  
AFTER-GRACEY-CURETTES

After-GRACEY  
**DB592R**  
Fig. 5/6  
Frontzähne  
Prämolaren  
universal  
  
Anteriors  
Premolars  
universal  
  
170 mm, 6¾"

After-GRACEY  
**DB593R**  
Fig. 7/8  
Prämolaren  
Molaren  
bukkal/lingual  
  
Premolars  
Molars  
buccal/lingual  
  
170 mm, 6¾"

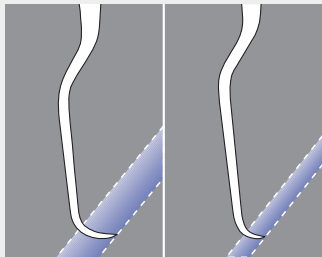
After-GRACEY  
**DB595R**  
Fig. 11/12  
Prämolaren  
Molaren  
mesio-bukkal/  
mesio-lingual  
  
Premolars  
Molars  
mesio-buccal/  
mesio-lingual  
  
170 mm, 6¾"

After-GRACEY  
**DB596R**  
Fig. 13/14  
Prämolaren  
Molaren  
disto-bukkal/  
disto-lingual  
  
Premolars  
Molars  
disto-buccal/  
disto-lingual  
  
170 mm, 6¾"

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Shaft 3 mm longer than  
Standard-GRACEY, narrow  
working tip shortened  
■ for easier access to the deep,  
narrow root surfaces

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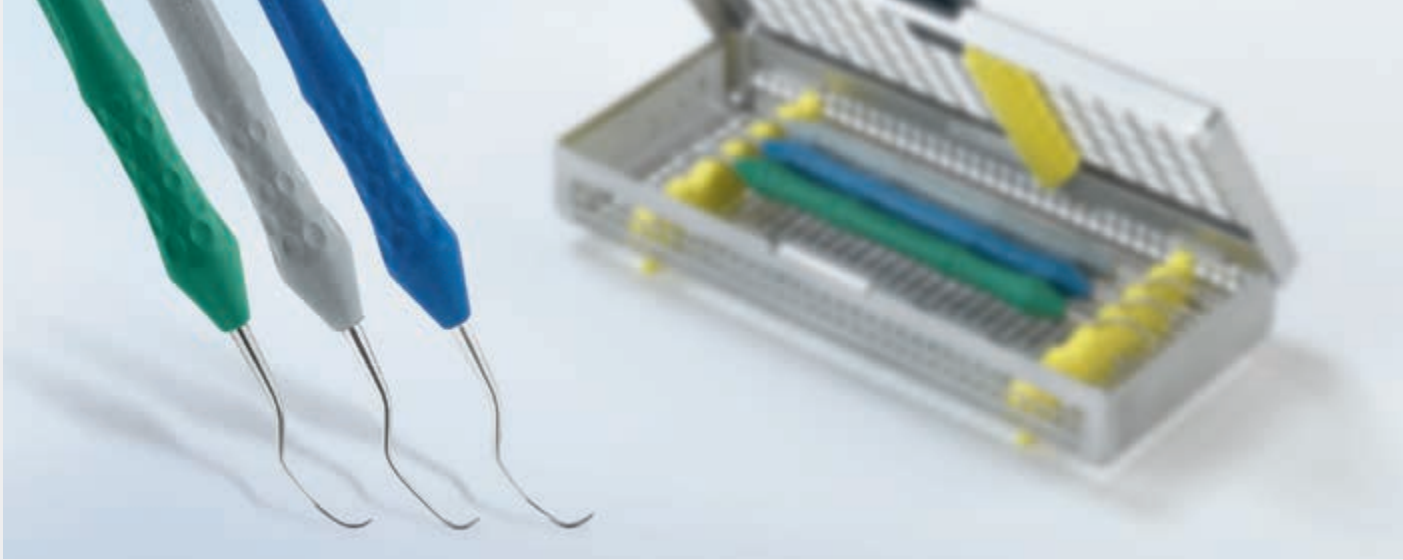
Titanium curettes for implant cleaning

- Manually sharpenable

To ensure the long-term success of an implant, consistent aftercare is very important. To eliminate plaque on the implant-abutment interface, titanium curettes can be preferable.

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Force-controlled periodontal probe

- Defined probing force of 0.2 N (20 g)
  - Low implant damage risk
- TiAl-nitride coating
  - Exact marking and smooth transition
  - No abrasion

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3rd European Consensus Conference (EuCC) Cologne 2008 on Periimplantitis: Diagnosis & procedure: ...careful probing of pocket on four sides (0.2 N probe)...

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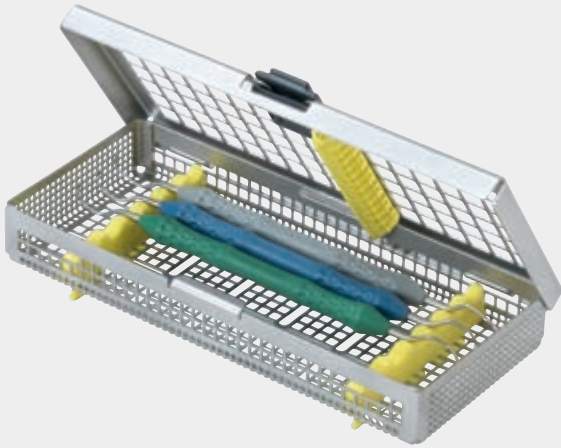
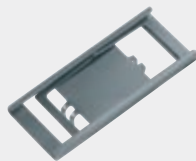
Recall Set consisting of  
 DB180T, DB181T, DB182T  
 and JG387R

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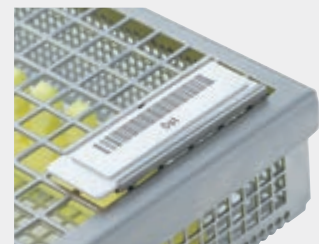
Holding device for  
 identification labels

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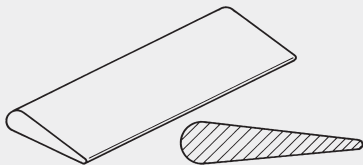
Plastic identification labels, white,  
 with 10-digit barcode for praxiDoc  
 or DIOS software applications,  
 labeling as specified

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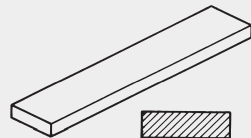
India Corund grinding stone, beige for sharpening and pregrinding of very blunt instruments

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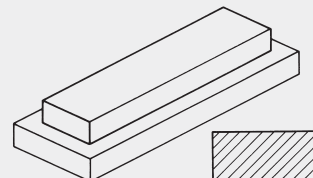
Original Arkansas grinding stone, grey-white, for repeated grinding on scalers and curettes as well as for finishing

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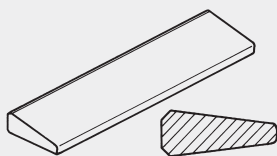
Super Arkansas grinding stone, white, for repeated grinding on scalers and curettes as well as for finishing

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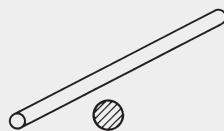
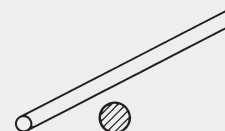
100 mm lang, Ø 6 mm

India Corund grinding stone, round file, brown beige, for deburring the working ends

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Plexiglas test rods, for checking cutting effect of scalers and curettes.

Sales Unit: PAK = Package of 6 pieces

**\* JG598**

Sterilit<sup>®</sup>i AESCULAP<sup>®</sup>-Schleifstein-Öl

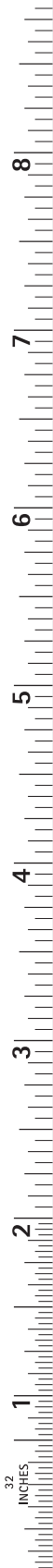
Inhalt: 50 ccm

Sterilit<sup>®</sup>i AESCULAP<sup>®</sup> oil for grinding stones

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Items marked \* are necessary for the basic equipment

STERILISIERBARE SCHLEIFSTEINE, ZUBEHÖR  
STERILIZABLE GRINDING STONES, ACCESSORIES

BB511

BB515-C

BB515

BB542

doppelschneidig  
double cutting

vakuum-vergütet  
heat-treated in vacuum

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Titanium scalpel handle for microsurgery blades, round handle. Light, rotatable and handy for precise cutting

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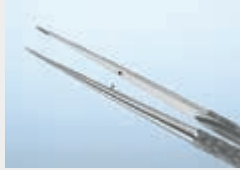
#### New micro instrument handles

As well as its optically appealing design the new surface we have created for the handles of our successful micro instruments offers principally hygienic advantages. The substantially improved grip allows working with minor physical strain more fatigue-free working.

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#### Scissors

Screw connection enables the regrinding of the scissors.

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#### Special hint

Since suture material is rather tough, the cutting capacity of scissors is quickly reduced which is noticeable as soon as tissues are cut afterwards.

#### Our solution

To cut sutures with special ligature scissors. The ligature scissor FD252R has additional rounded edges to prevent jamming of the suture.

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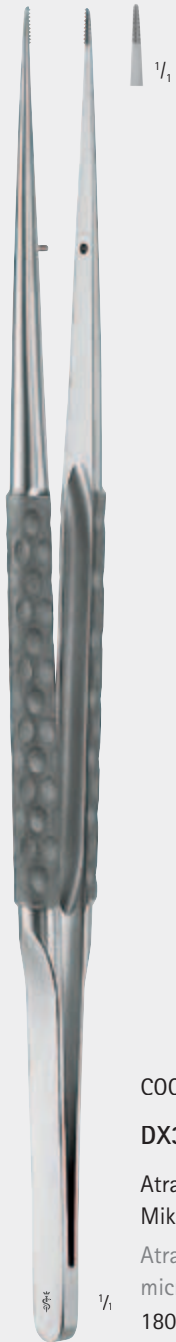
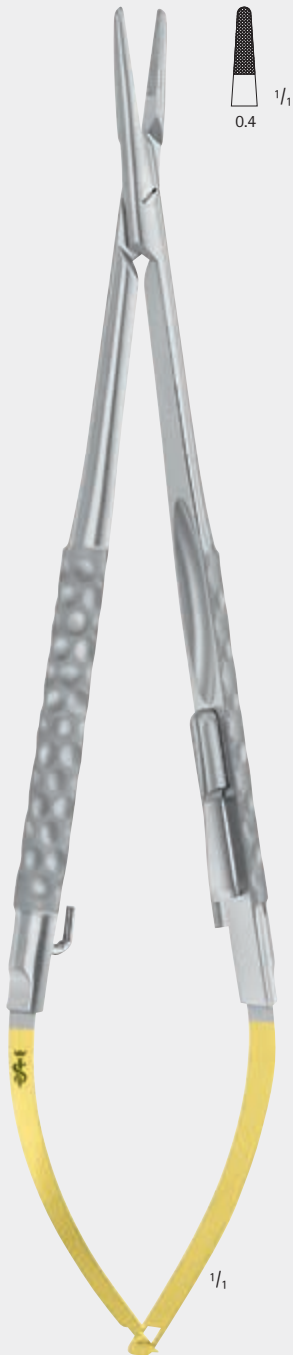
Rounded plate-shaped working tips allow more movement, and significantly increase the radius of mobilisation in frontal and lateral sub-periosteal preparation.

The sharpened edge of the instrument tips also allows sulcular as well as tunnel technique preparation (as with the envelope technique).

The design and angle of the working tips significantly facilitates preparation in convex areas.

**DB095**  
Original Arkansas-Schleifstein, zum Aufschleifen der Tunnelierinstrumente  
Original Arkansas grinding stone, for grinding on tunnelling instruments  
75 x 25 x 7 mm

**JG598**  
Sterilit®i AESCLAP®-Schleifstein-Öl  
Inhalt: 50 ccm  
Sterilit®i AESCLAP® oil for grinding stones  
Contents: 50 ccm

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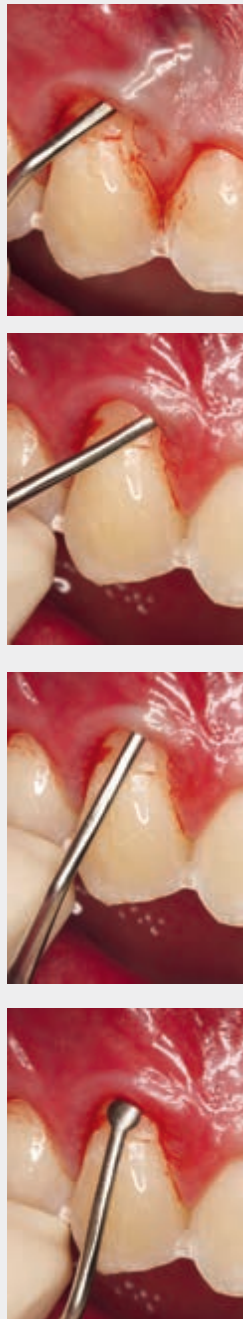
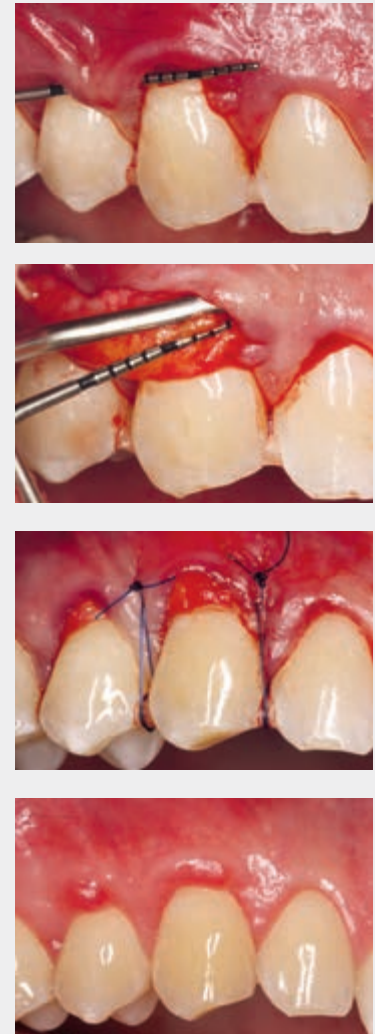
The **straight** tunnelling instrument is mainly used in the upper jaw, in the frontal and lateral region. This allows large areas of the gingiva to be undermined using both tunnel and sulcular techniques.

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The **angled** tunnelling instrument is primarily used for preparing lateral, buccal regions. It can also be used distally, without tearing the mucosal tissue and also for the lower jaw.

PRICHARD  
**DX200R**  
Raspatorium  
Raspatory  
200 mm, 8"

PRICHARD  
**DB020R**  
Raspatorium  
grazil  
Raspatory  
delicate  
195 mm, 7 3/4"

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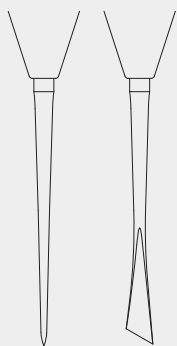
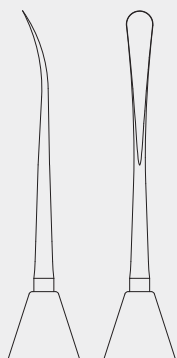
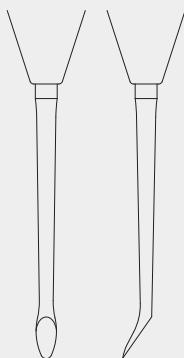
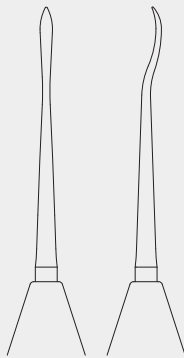
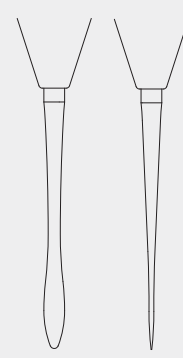
Elevation of the flap starting from the relieving incision with the disk-shaped elevator (DB407R).

**DB407R**

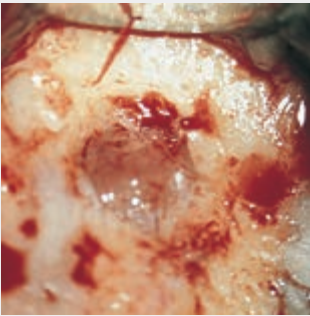
Tellerraspatorium und Dissektor

Disk-shaped elevator and dissector

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Retraction of the flap using the retractor.  
The apical lesion is visible in the center of the picture.

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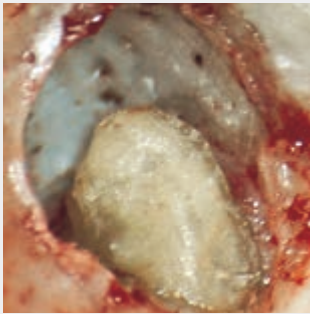
The apical lesion removed using the following:  
disk-shaped curettes (DB406R resp. DB408R, the spoon-shaped curette (DB409R)).

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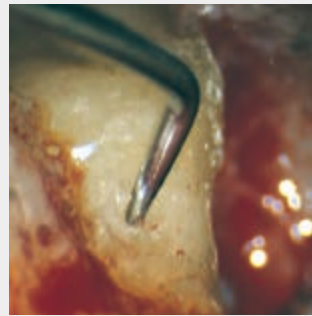
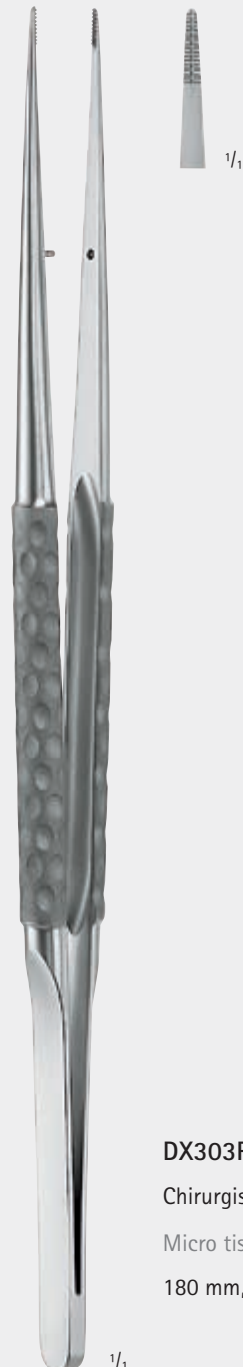
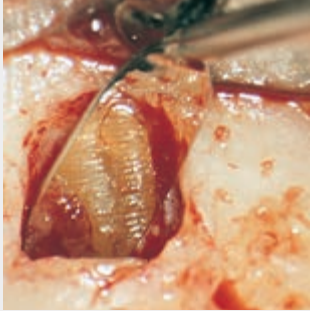
Use of the oval micro mirror (DA045R resp. DA046R). Fracture line barely detectable in low magnification (12x).

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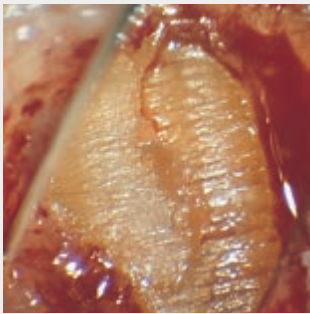
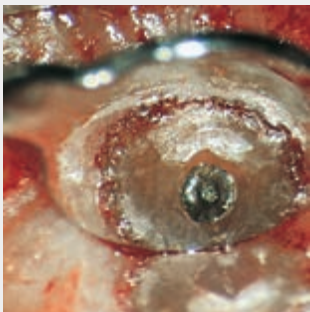
Large magnification (25x) of the above illustration. The fracture line is visible in the center of the illustration.

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Use of the round micro mirror (DA043R resp. DA044R). View of prepared retrograde cavity with the post in the root canal.

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Titanium handle for interchangeable micro mirrors  
DA043R – DA048R and for microscalpel blades

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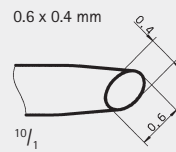
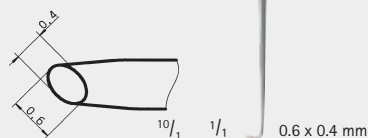
Oval retrograde cavity, seen after preparation of the isthmus between the root canals. The oval shape plugger (DE442R) is used to condense the retrograde filling material.

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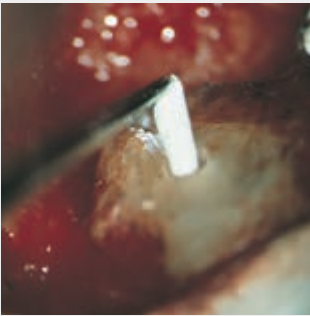
Application of the retrograde filling material into the cavity with the micro spatula (DE440R resp. DE441R).

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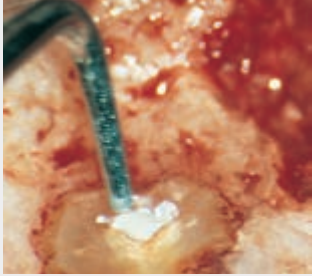
Ball burnisher/spatula,  
Micro spatula, angulated

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Condensation of the filling material with the round plugger (DE444R resp. DE445R or DE443R) (in round canals).

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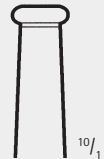
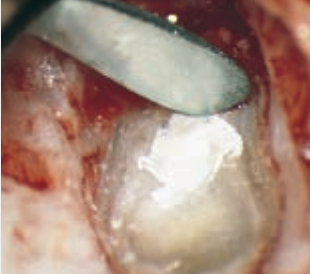
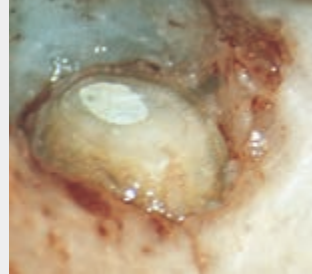
Moulding instrument for exact moulding of the cusp

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Moulding instrument for adapting the cavity edges

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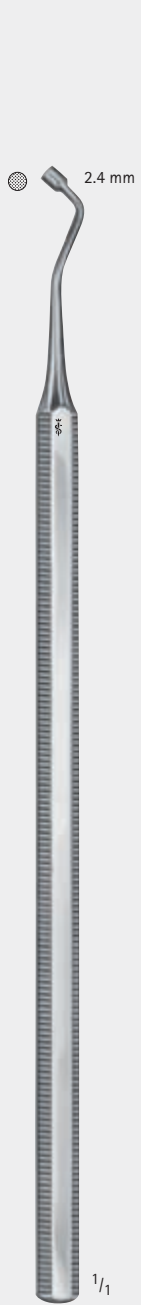
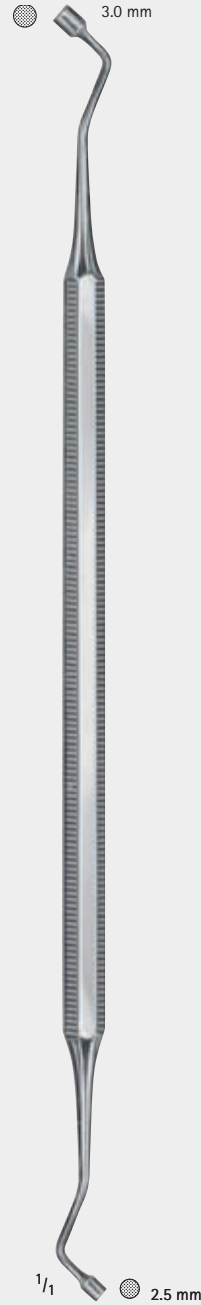
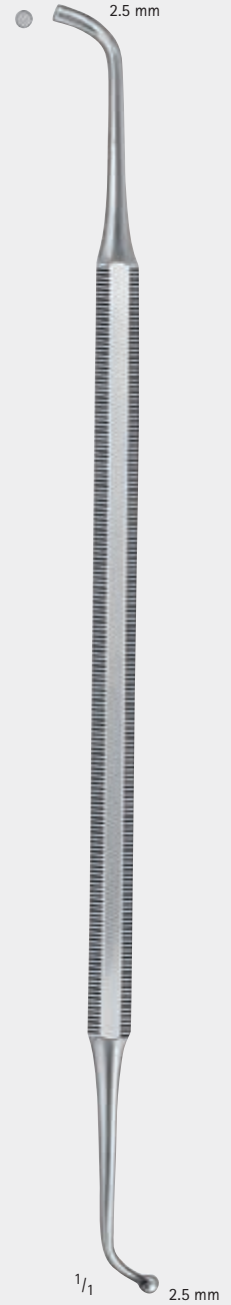
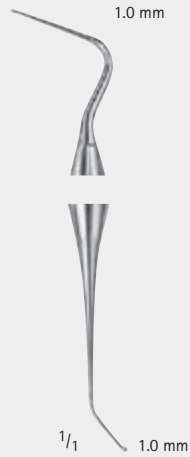
For applying and plugging of lower filling material and plastic filling material

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For applying lower filling material and protective varnish out of calcium hydroxide and glass ionomers.

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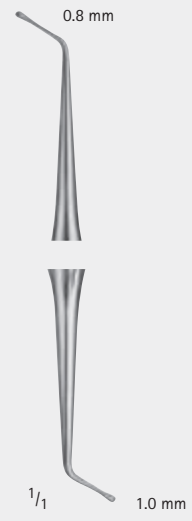
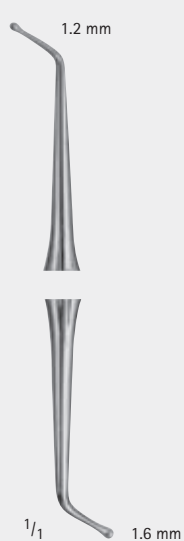
For applying medicaments, lining of cavities and condensing of amalgam

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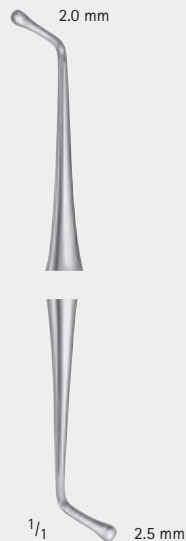
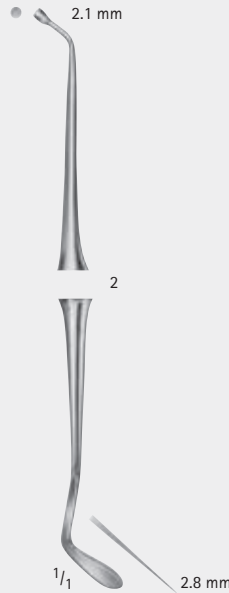
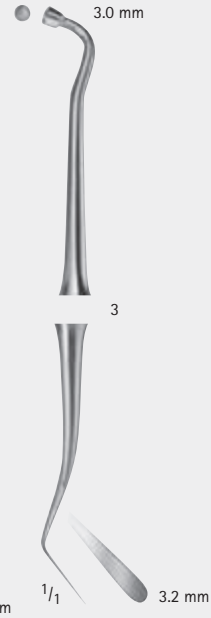
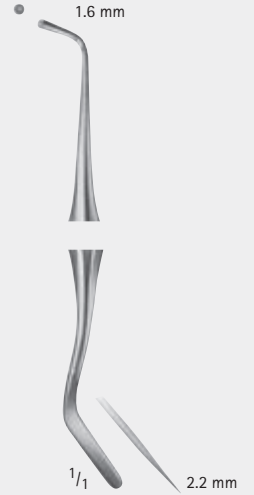
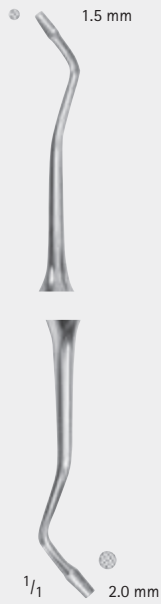
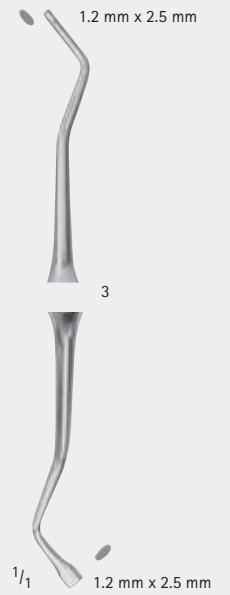
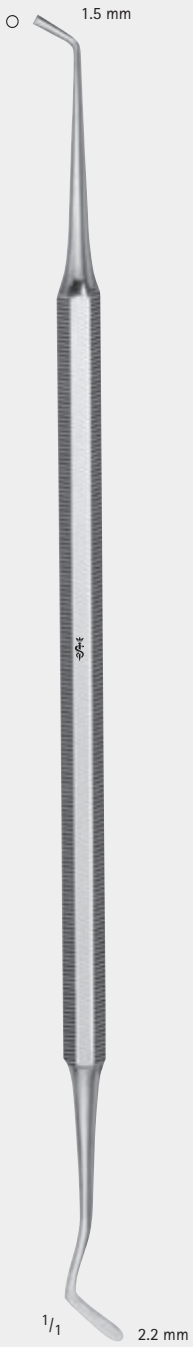
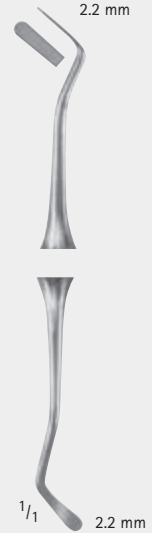
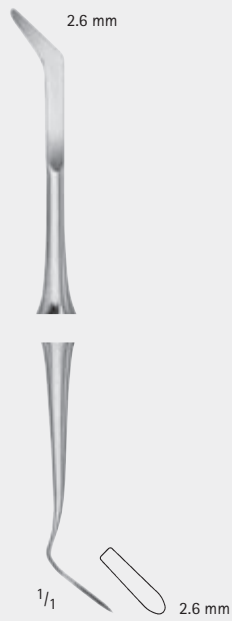
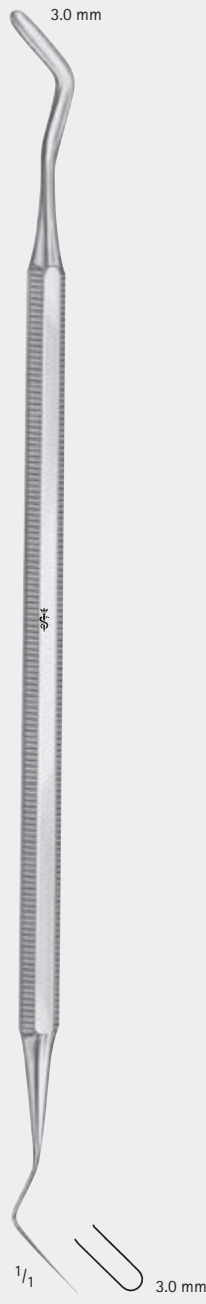
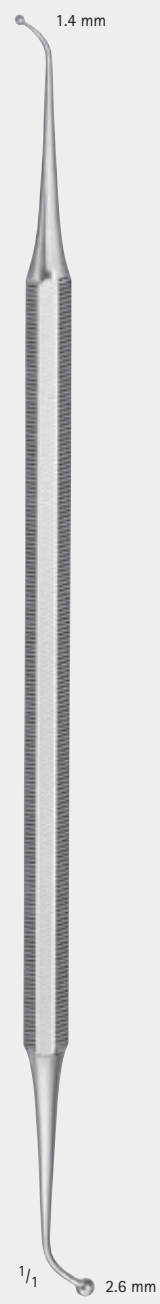
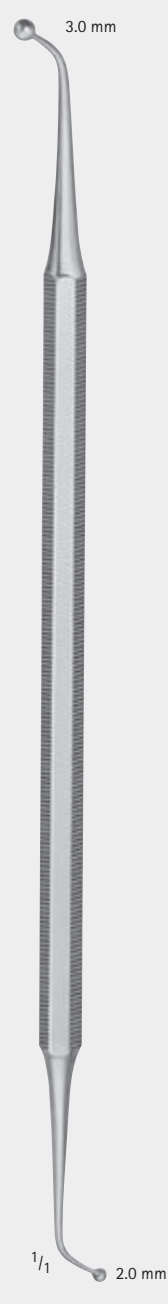
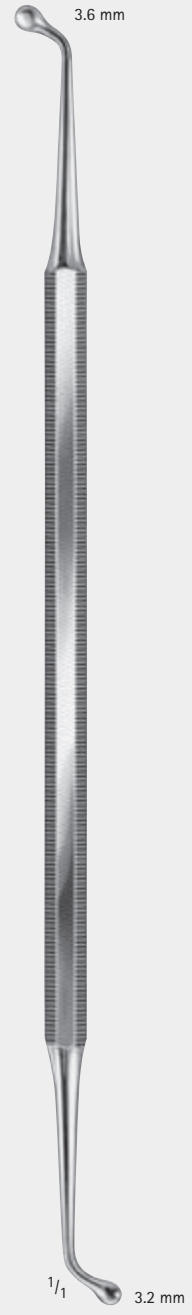
WHITE  
**DF028R**  
Glätter  
Burnisher  
150 mm, 6"

HOLLENBACK  
**DE676R**  
Modellier Instrument  
Finishing instrument  
170 mm, 6 3/4"

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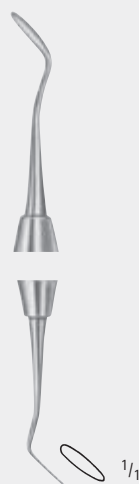
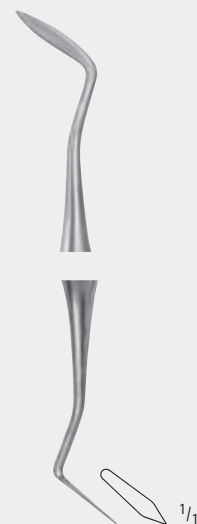
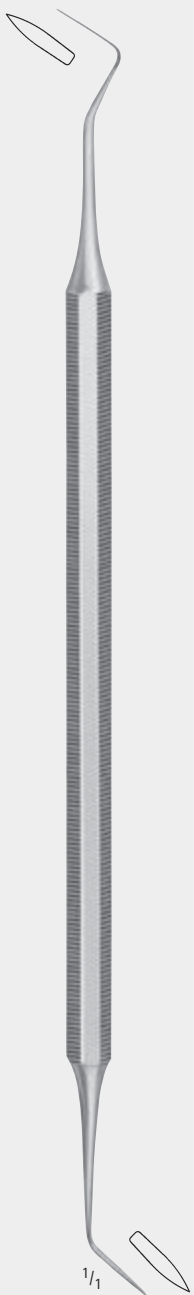
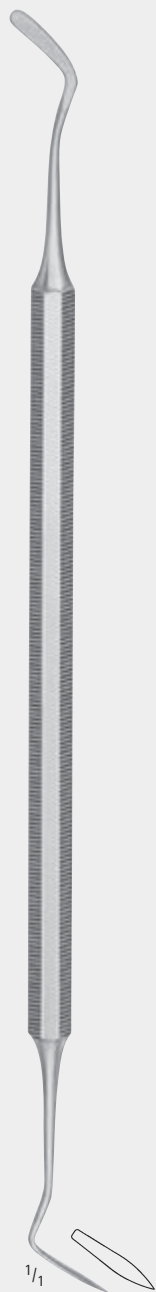
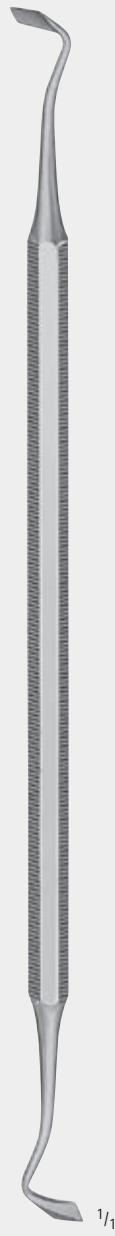
WARD  
**DE425R**  
 Modeller Instrument  
 Finishing instrument  
 170 mm, 6 3/4"

WARD  
**DE426R**  
 Modeller Instrument  
 Finishing instrument  
 170 mm, 6 3/4"

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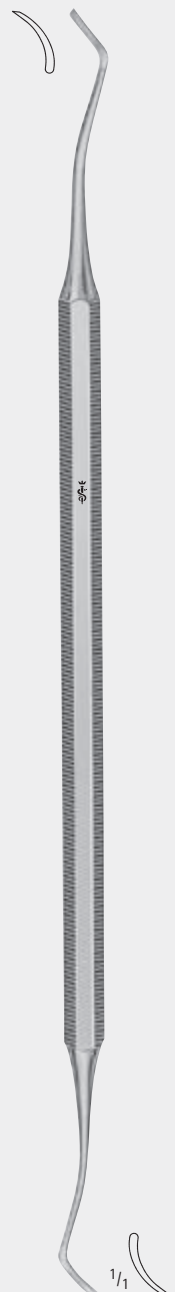
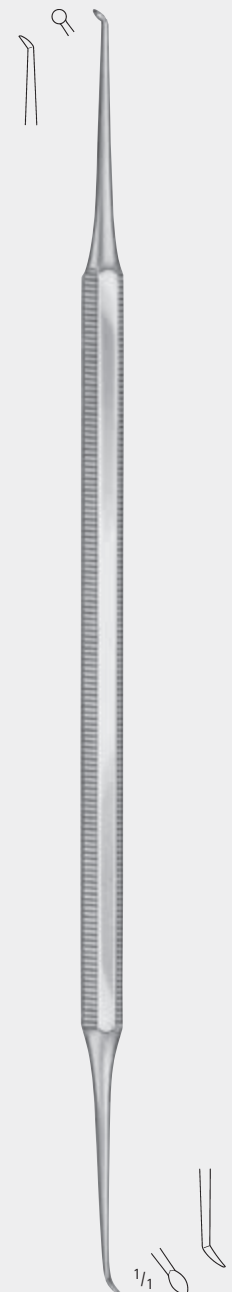
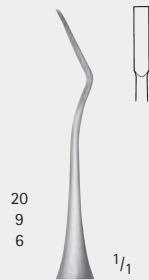
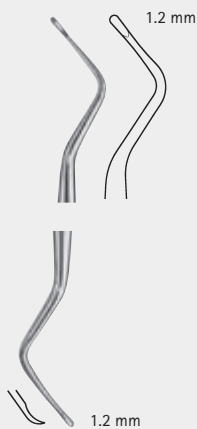
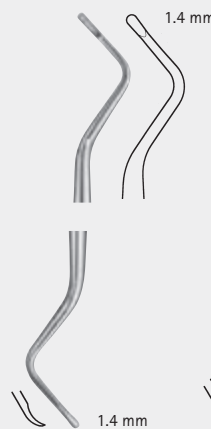
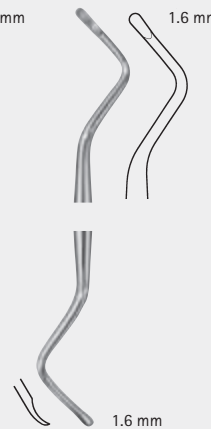
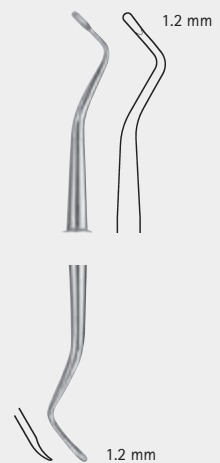
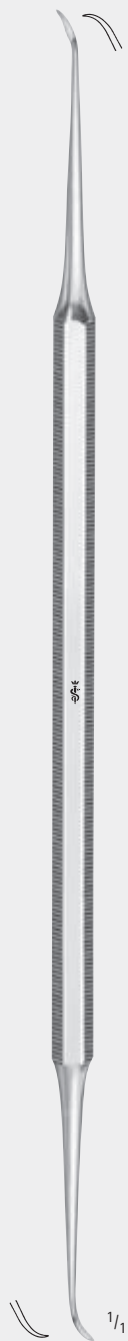
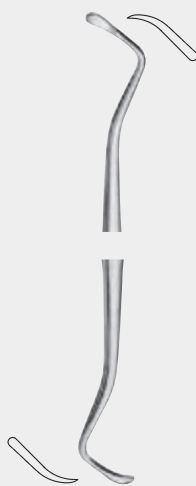
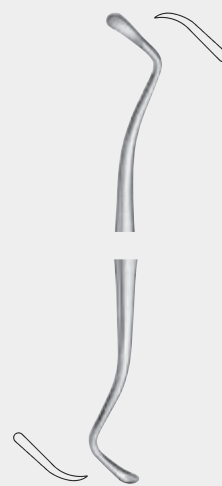
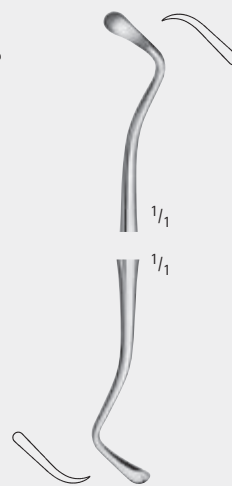
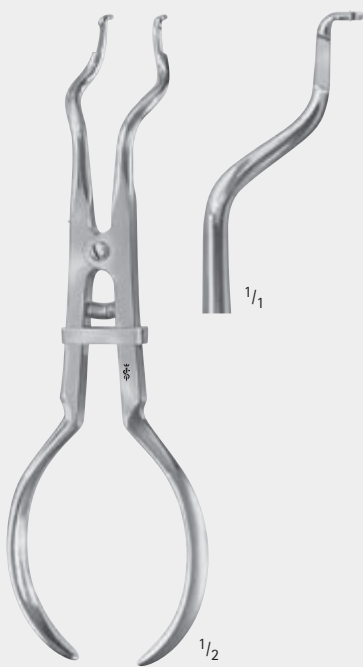
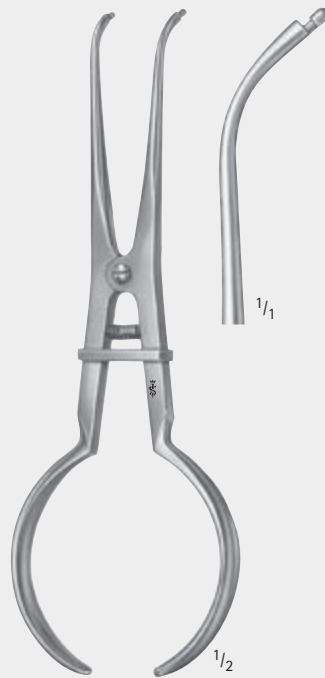
## DENTAL SURGERY

Skalpellgriffe für sterile Skalpellklingen  
 BB510 - BB515, BB540, BB542

Scalpel handles for sterile scalpel blades  
 BB510 - BB515, BB540, BB542

vakuum-vergütet  
 heat-treated in vacuum

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Scalpel blades  
 Sales unit: PAK = Package of 100 pieces

SKALPELLGRIFFE, STERILE SKALPELLKLINGEN  
SCALPEL HANDLES, STERILE SCALPEL BLADES

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Scalpel handles for sterile scalpel blades  
BB518 - BB536

vakuum-vergütet  
heat-treated in  
vacuum

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Sterile scalpels with plastic handle and blade protector  
Sales unit: PAK = Package of 10 pieces

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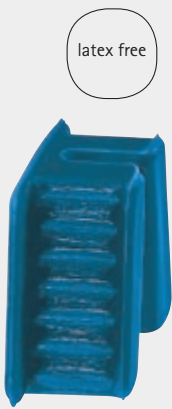
Silicone mouth wedge with metal core  
 Size: M

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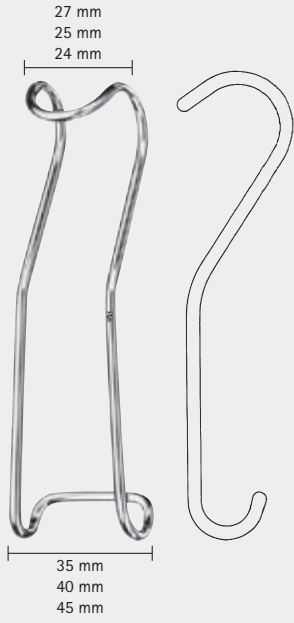
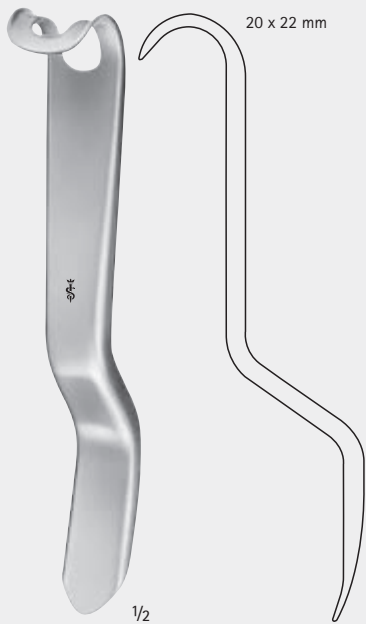
Silicone mouth wedge with metal core  
 Size: L

**STERNBERG**  
**OM251R** – 120 mm, 4 3/4"  
**OM252R** – 130 mm, 5 1/8"  
**OM253R** – 135 mm, 5 1/4"  
 Wangen- und Lippenhalter  
 Cheek and lip retractor

**MINNESOTA, modif.**  
**DO010R**  
 Wangenhalter  
 Cheek retractor  
 160 mm, 6 1/4"

**DO011R**  
 Vestibulumhaken  
 Vestibulum hook  
 140 mm, 5 1/2"

**ERGOPLANT**  
**DX101R**  
 Vestibulumhaken  
 Vestibulum hook  
 220 mm, 8 3/4"

ZUNGEN- UND WANGENHALTER, ZUNGENSPEL, ERGOPLANT ZUNGENRETRAKTOR  
 TONGUE AND CHEEK RETRACTORS, TONGUE SPATULA, ERGOPLANT TONGUE RETRACTOR

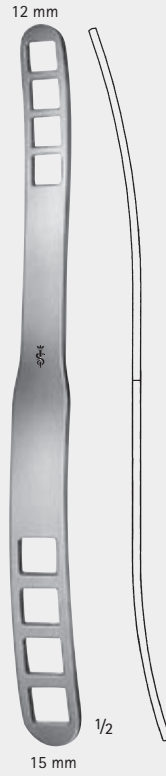
**OM202R**  
 Zungenspatel  
 Tongue spatula  
 145 mm, 5 3/4"

**OM203R**  
 Zungenspatel  
 Tongue spatula  
 140 mm, 5 1/2"

**BUCHWALD**  
**OM205R**  
 Zungenspatel  
 Tongue spatula  
 180 mm, 7"

**BRUENINGS**  
**OM208R**  
 Zungenspatel  
 Tongue spatula  
 190 mm, 7 1/2"

**ERGOPLANT**  
**DX105R**  
 Zungenretraktor  
 Tongue retractor  
 190 mm, 7 1/2"

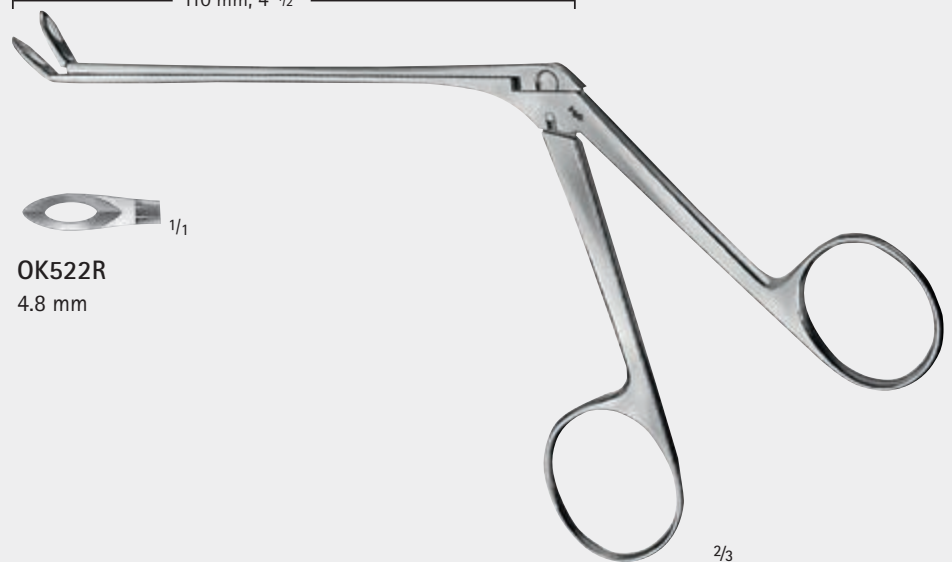
 1/1  
**OK520R**  
3.6 mm

 1/1  
**OK521R**  
4.2 mm

 1/1  
**OK522R**  
4.8 mm

110 mm, 4 1/2"

WEIL BLAKESLEY  
**OK520R – OK522R**  
Siebbeinzange, gebogen  
Ethmoidal forceps, curved

SIEBBEINZANGE  
ETHMOID FORCEPS

1/1

**OK506R**

3.6 mm

1/1

**OK507R**

4.2 mm

1/1

**OK508R**

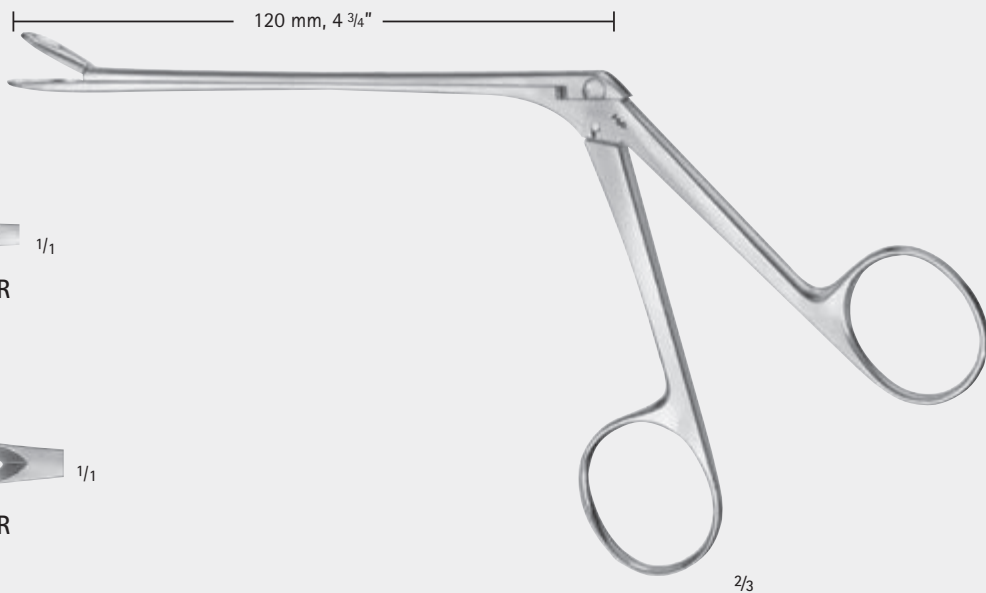
4.8 mm

1/1

**OK509R**

5.6 mm

BLAKESLEY-WILDE

**OK506R – OK509R**

Siebbeinzange, gerade

Ethmoidal forceps, straight

1/2

**ERGOPLANT  
DX105R**  
Zungenretractor  
Tongue retractor  
190 mm, 7 1/2"

1/1

**HARTMANN  
OM230R**  
Zungenspatel  
Tongue spatula  
150 mm

1/1

**TOBOLD  
OM235R**  
Zungenspatel  
Tongue spatula  
160 mm

ZUNGENSPEL, BACH- UND DARMSPATEL, HIRNSPEL  
 TONGUE SPATULA, ABDOMINAL AND INTESTINAL SPATULA, BRAIN SPATULA

1/1

**BT751R – BT752R**

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Abdominal and Intestinal  
Spatulas

	Abmessungen Dimensions
<b>BT751R</b>	12 x 200 mm
<b>BT752R</b>	17 x 200 mm

1/1

**FF201R – FF202R**

Hirnspatel, biegsam,  
doppelt endig

Brain spatula, malleable  
double-ended

	Abmessungen Dimensions
<b>FF201R</b>	9 x 200 mm
<b>FF202R</b>	12 x 200 mm

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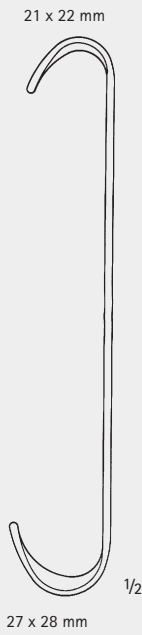
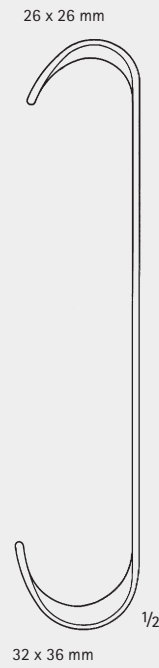
Cheek retractor

Set complete, consisting of:  
BT031R, BT032R, BT033R

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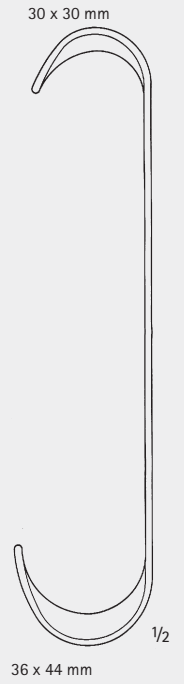
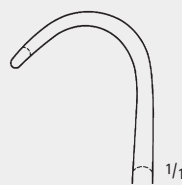
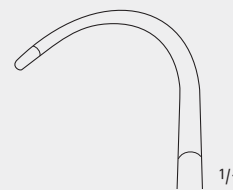
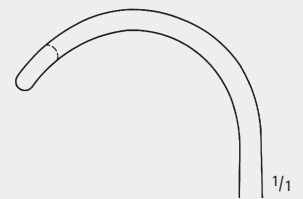
### Retractors for retrograde root resection

Before the use of the retractor, 2 slight deepenings are burred in the jaw. The 2 tips at the working end of D0373R are inserted and will therefore also provide a reliable grip.

RICHTER

D0373R

Retraktor, Blattbreite 15 mm, 45°

Retractor, width of blade 15 mm, 45°

200 mm, 8"

1/1

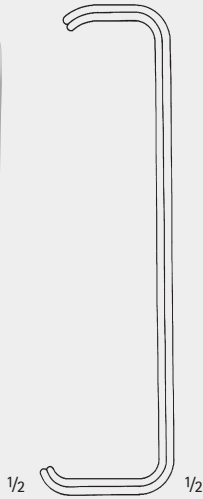
25 x 10 mm  
22 x 10 mm

28 x 12 mm  
32 x 12 mm

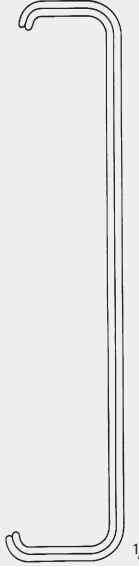
FARABEUF  
BT020R  
Wundhaken  
Retractor  
125 mm, 5"

23 x 16 mm  
20 x 16 mm

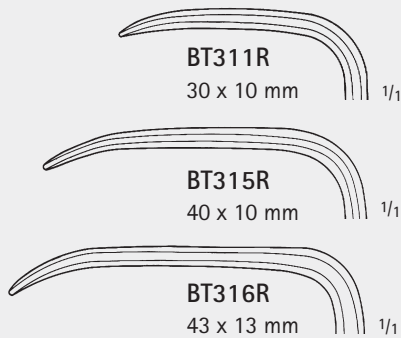
24 x 16 mm  
28 x 16 mm

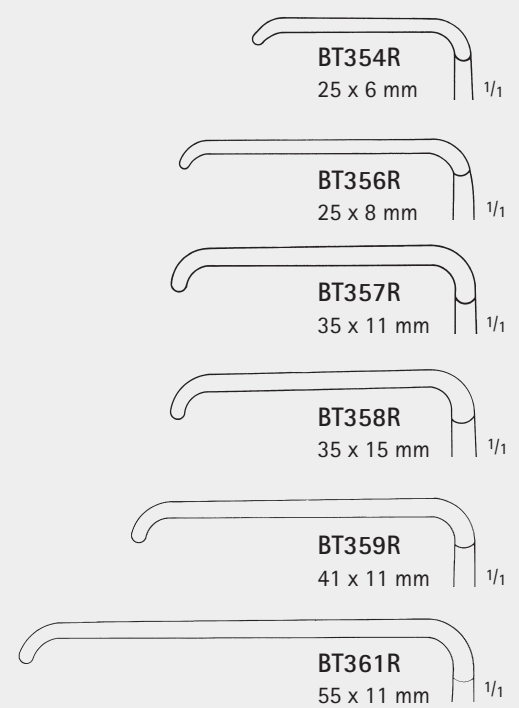
FARABEUF  
BT021R  
Wundhaken  
Retractor  
150 mm, 6"

LANGENBECK  
BT311R - BT316R  
Wundhaken  
Retractor  
230 mm, 9"

KOCHER-LANGENBECK  
BT354R - BT361R  
Wundhaken  
Retractor  
215 mm, 8 1/2"

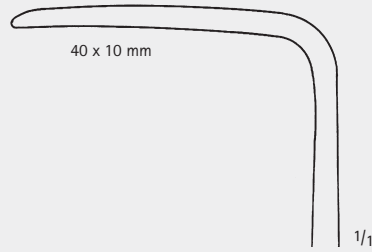
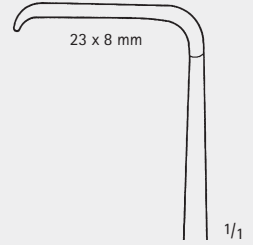
WUNDHAKEN, WANGENHALTER  
RETRACTORS, CHEEK RETRACTORS

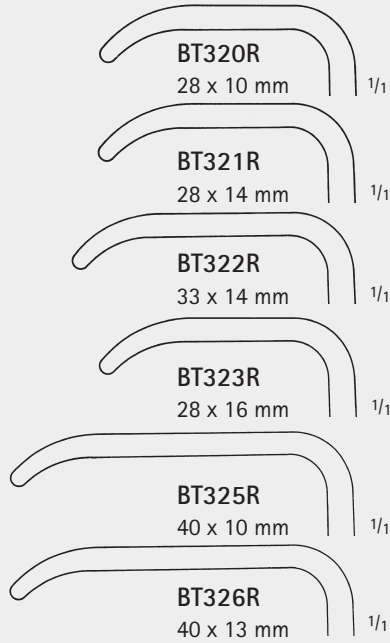
**BT343R**  
Wangenhalter  
Cheek retractor  
215 mm, 8 1/2"

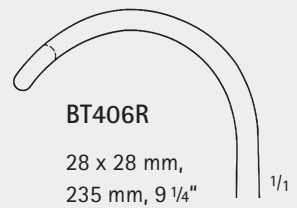
**BT340R**  
Wundhaken  
Retractor  
210 mm, 8 1/4"

LANGENBECK  
**BT320R – BT326R**  
Wundhaken  
Retractor  
210 mm, 8 1/4"

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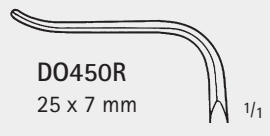
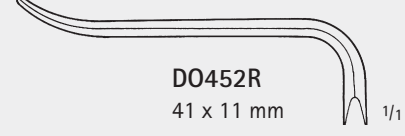
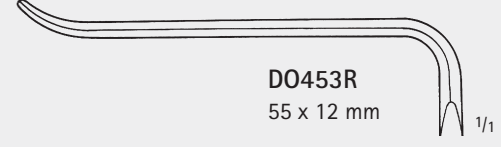
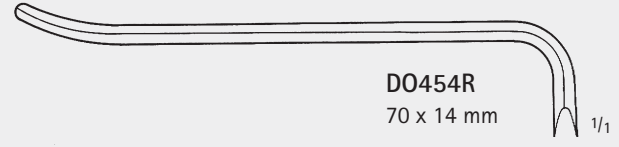
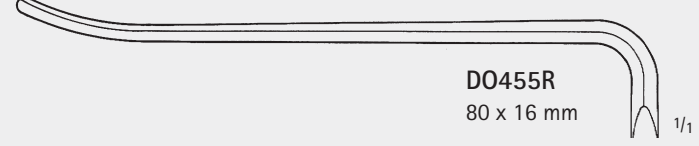
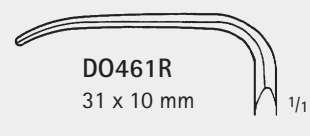
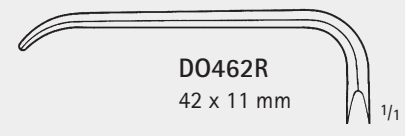
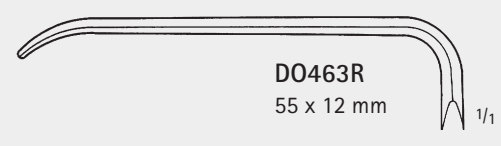
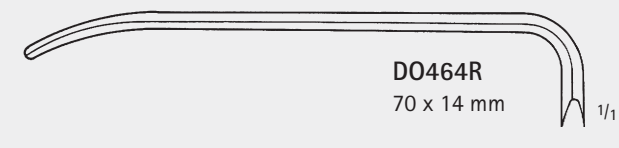
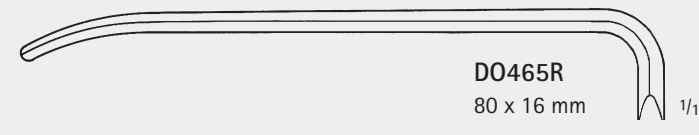
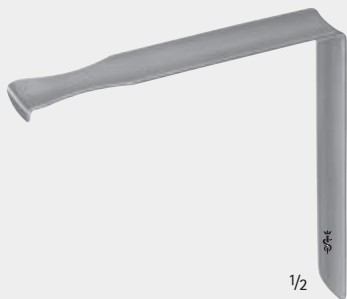
Especially designed for upper impacted wisdom teeth. The design of the retractor enables the retraction of the cheek and mucoperiostal flap as well as one hand simultaneous support of retractor and an additional instrument.

LASTER  
DO472R  
Wangenhaken  
Cheek retractor  
100 mm, 4"

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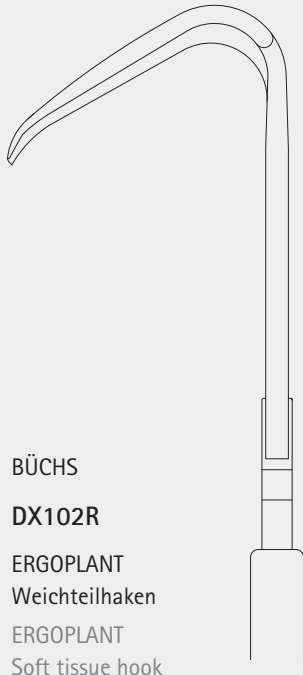
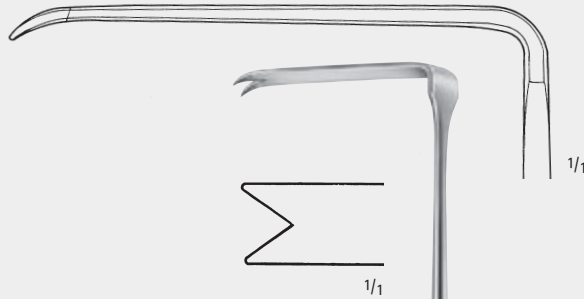
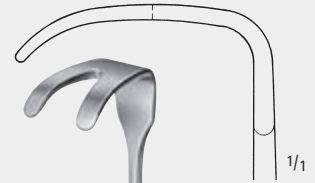
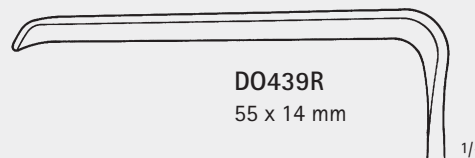
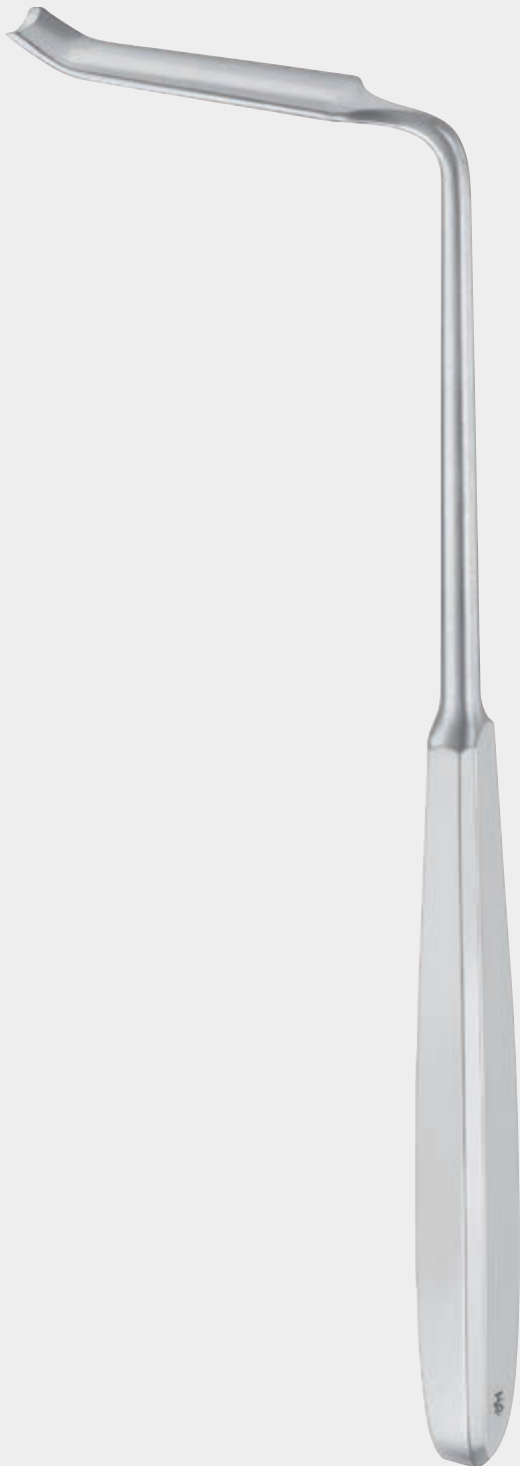
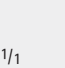
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
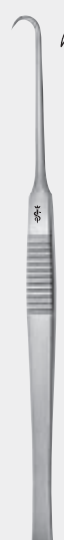
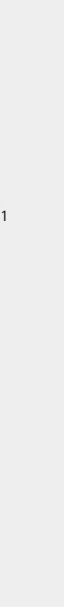
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
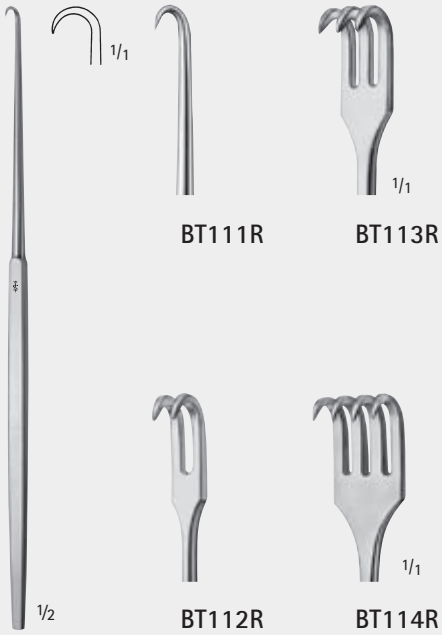
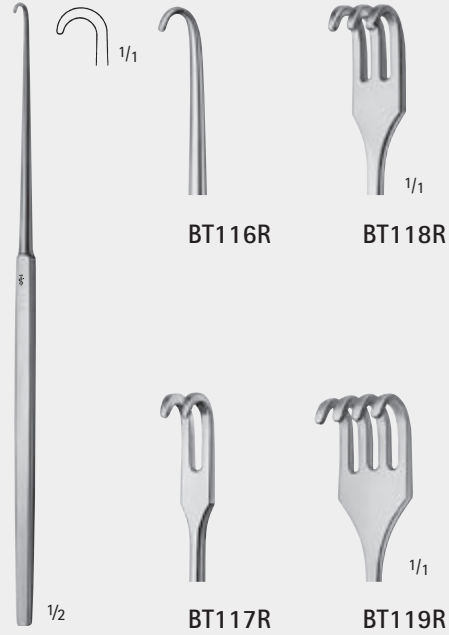
WUNDHAKEN, FEINE WUNDHÄKCHEN  
 RETRACTORS, FINE WOUND RETRACTORS

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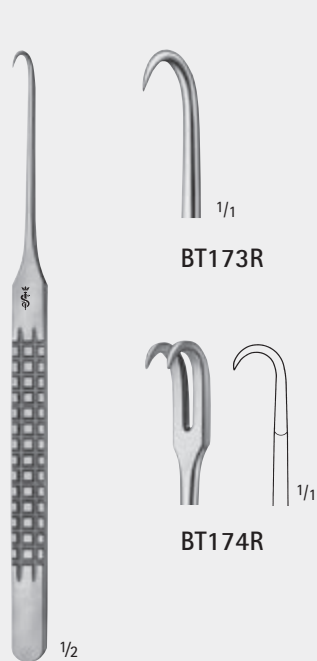
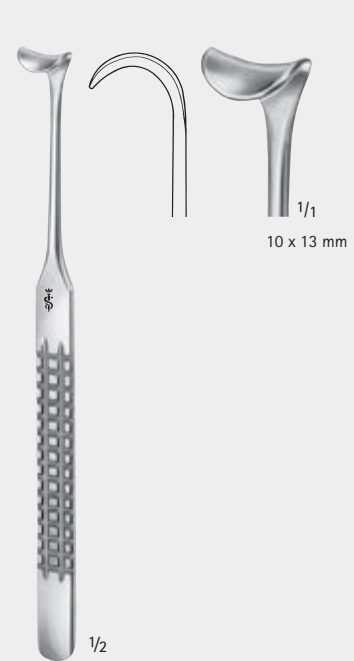
Retraction thread packer,  
flexible

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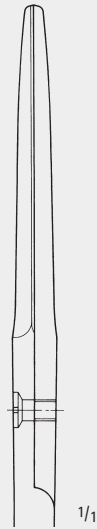
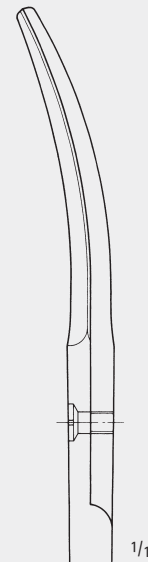
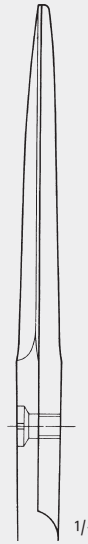
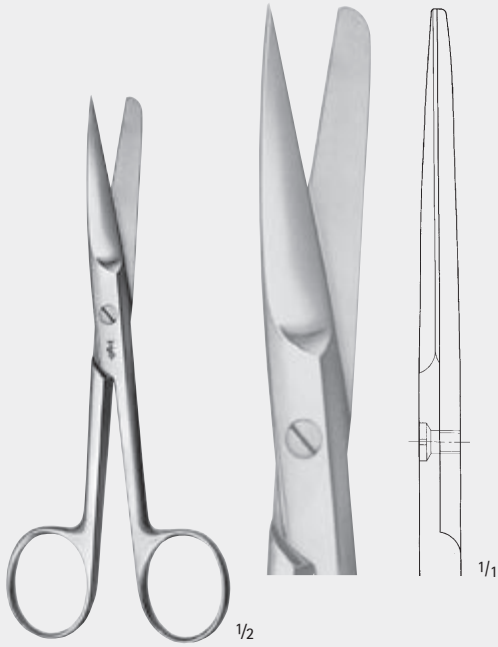
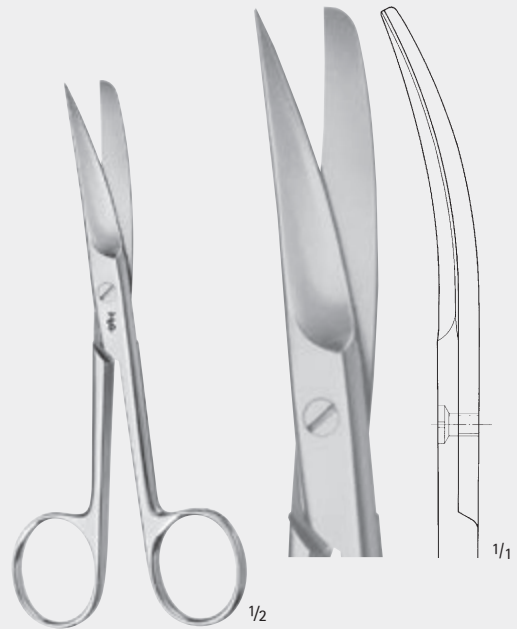
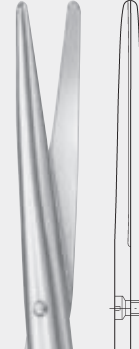
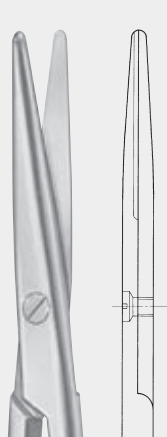
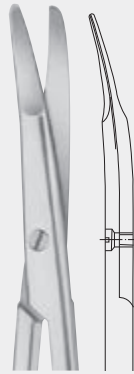
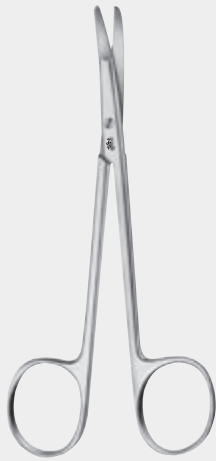
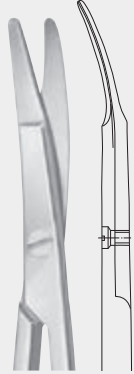
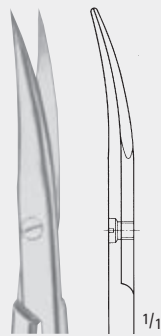
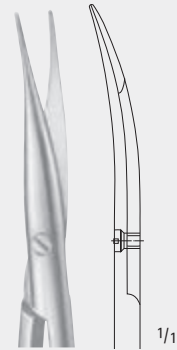
Special flat points for dissecting and  
undermining the skin

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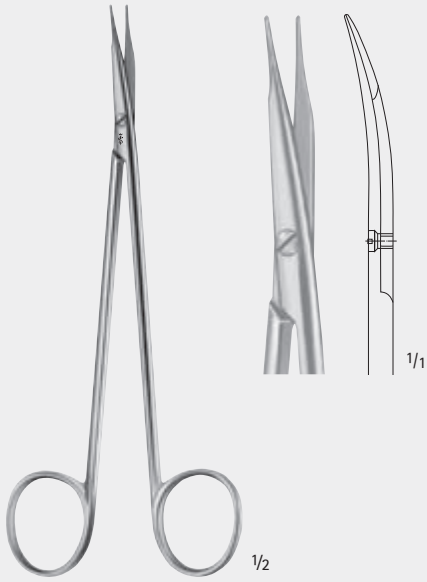
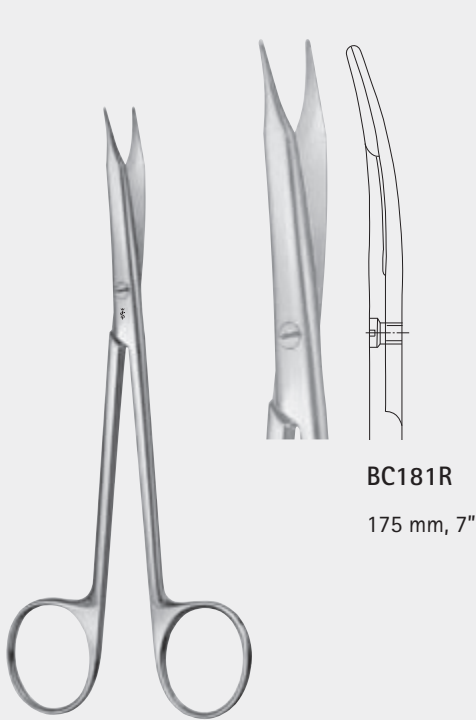
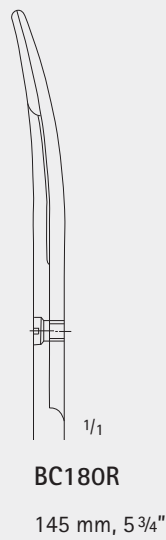
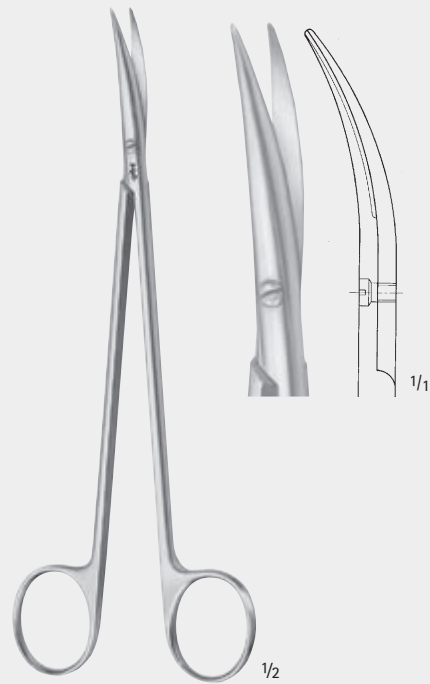
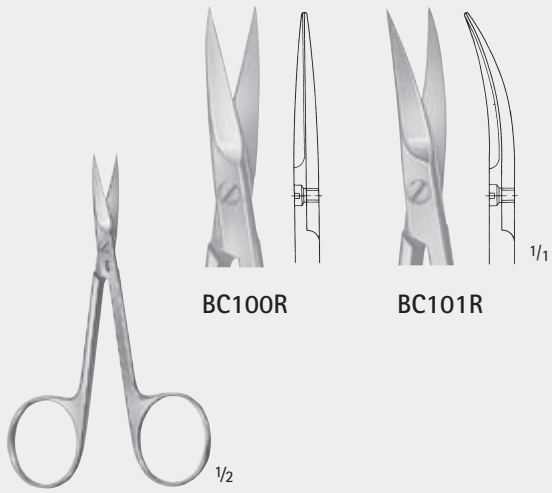
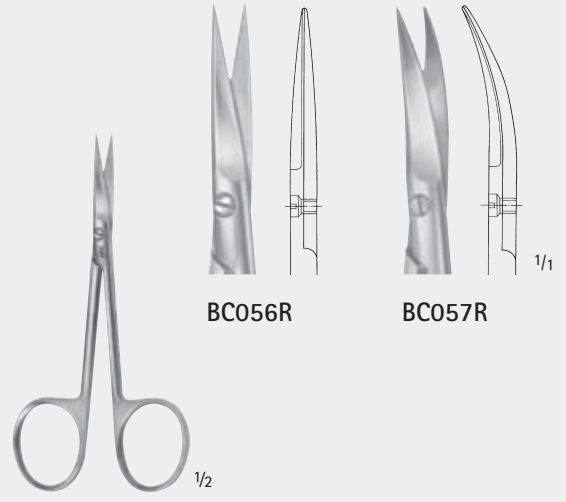
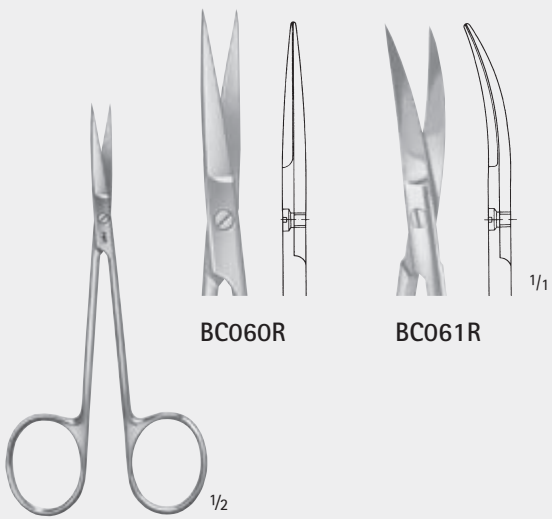
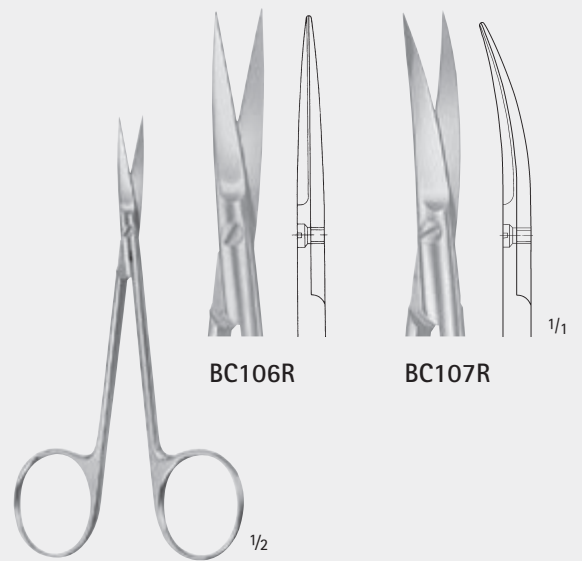
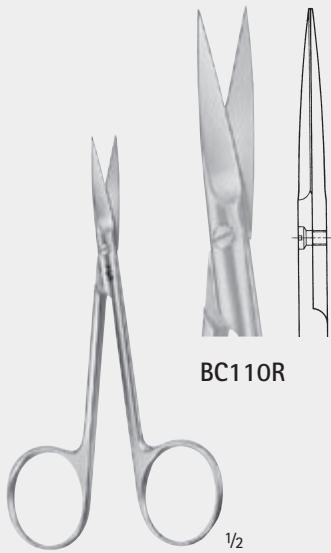
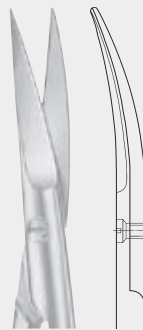
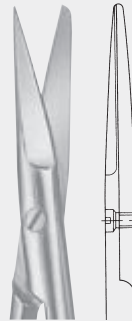
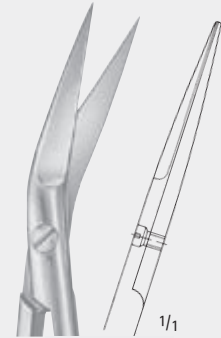
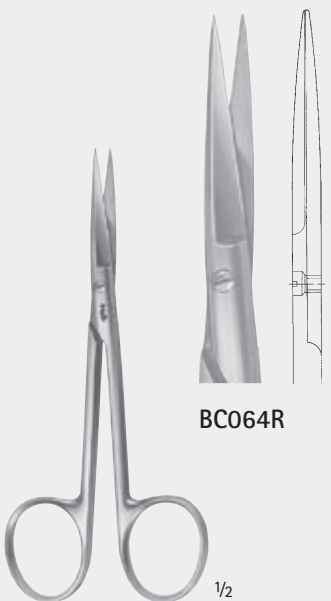
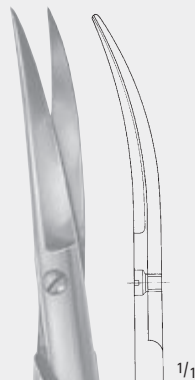
Curved to side,  
especially suited to  
remove ligatures

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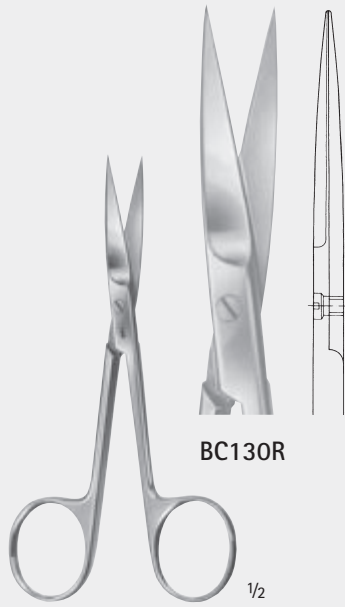
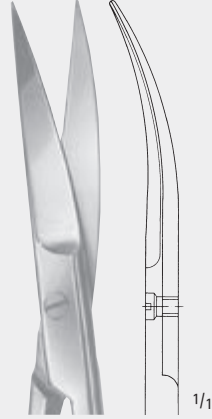
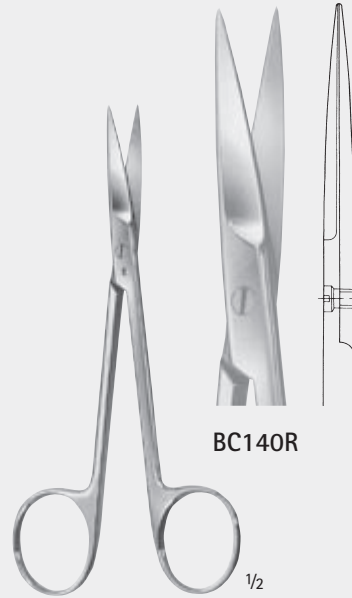
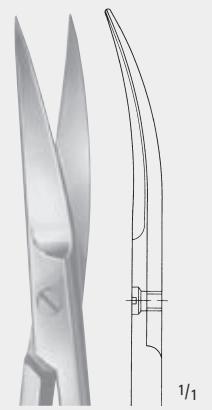
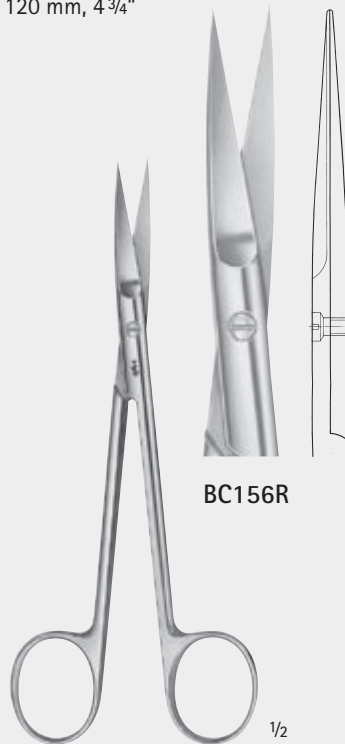
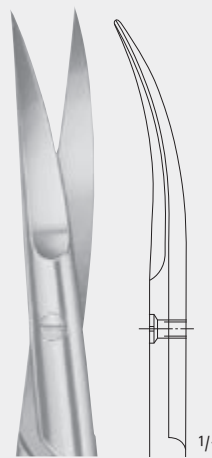
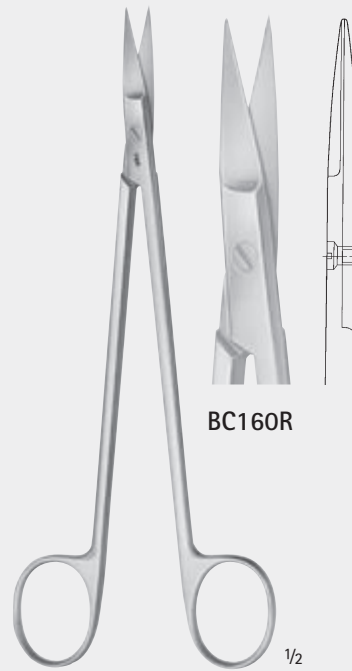
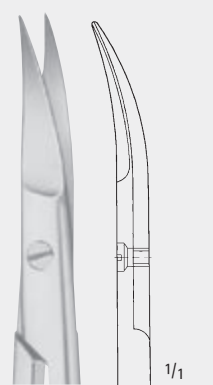
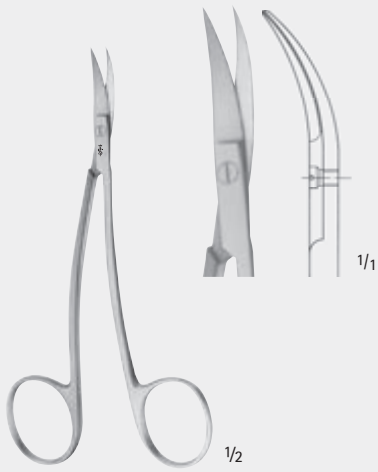
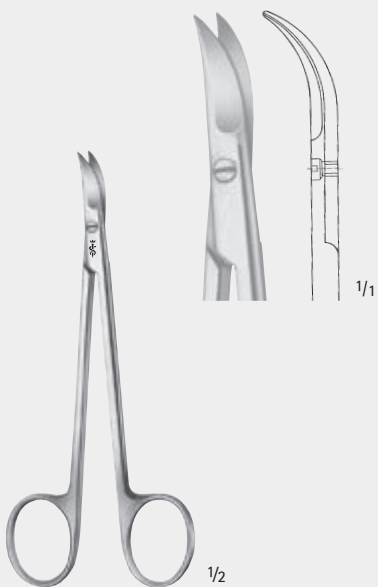
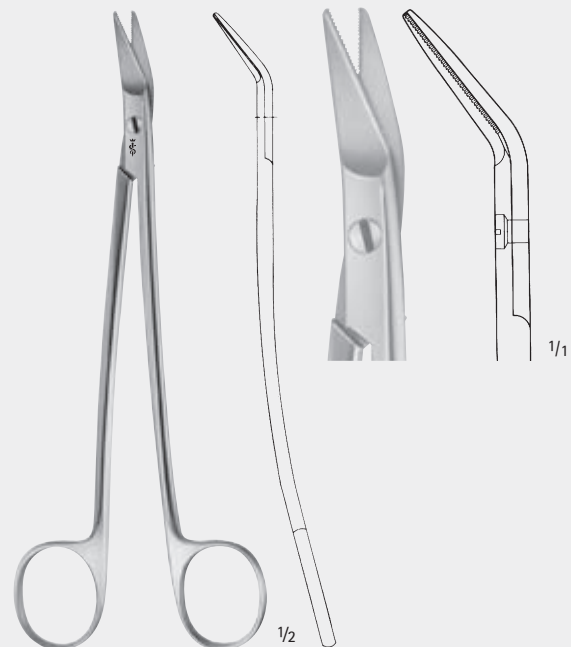
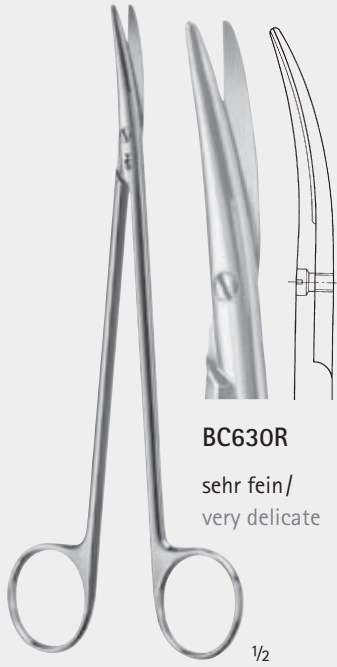
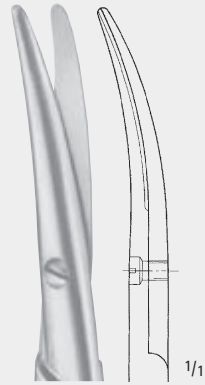
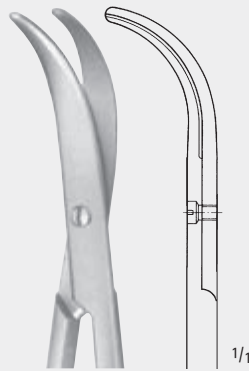
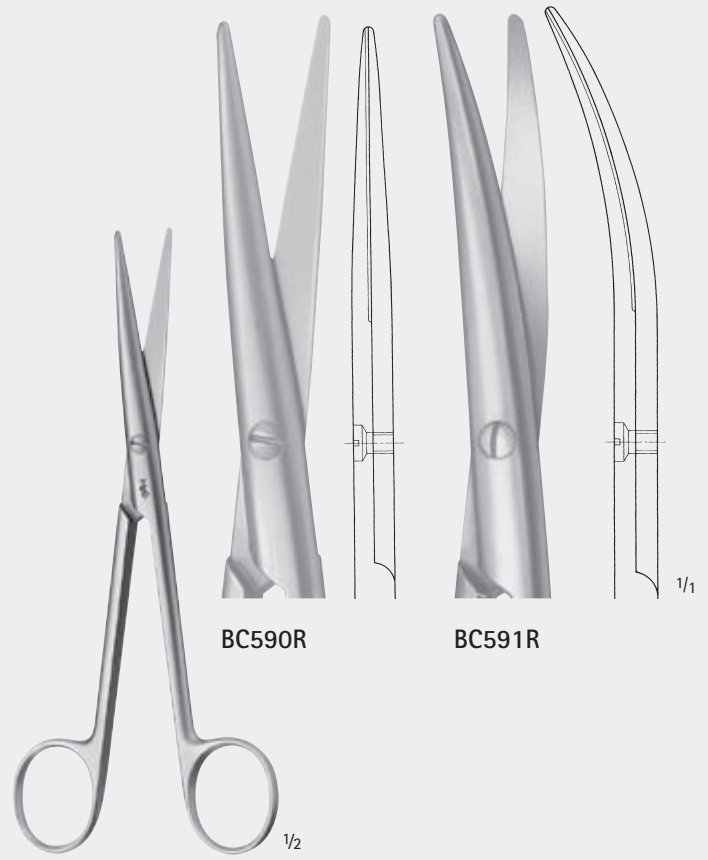
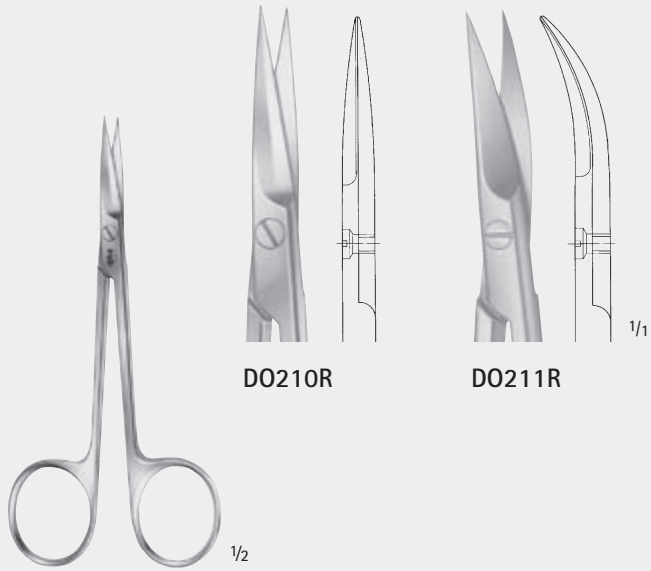
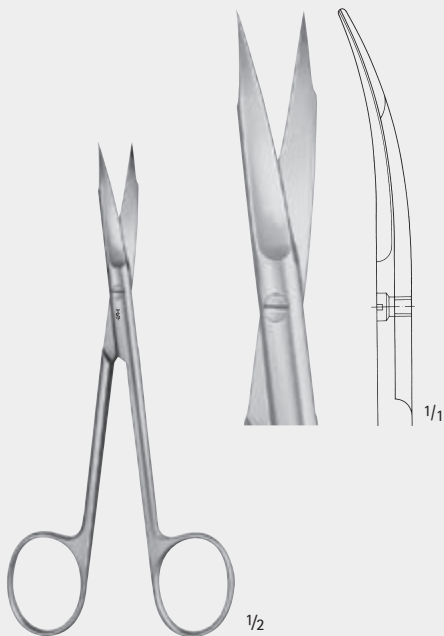
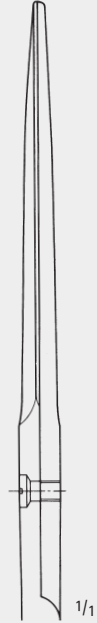
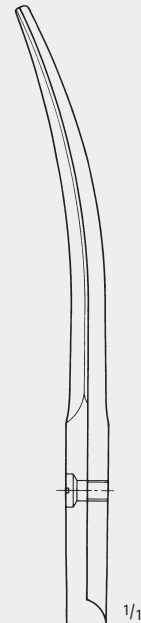
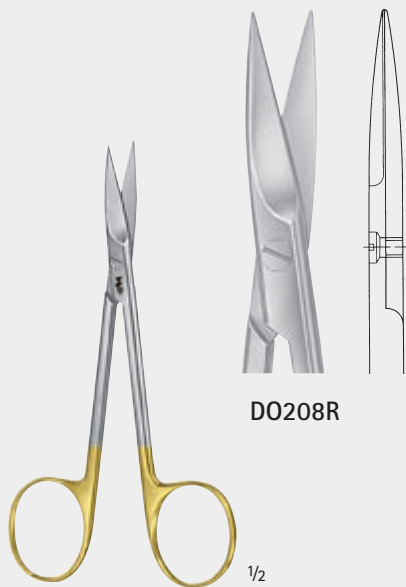
- 3 Jahre Garantie
- Vergoldete Ringe

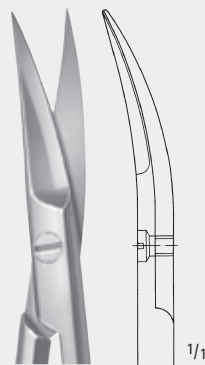
DUROTIP®-Scissors  
 with inserted carbide cutting edges

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- 3 Jahre Garantie
- Vergoldete Ringe

DUROTIP®-Scissors  
with inserted carbide cutting edges

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~W-Cut directly after the catalogue number  
indicates that the resp. item has one serrated  
cutting edge.

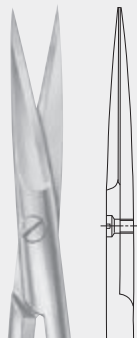
(When ordering please indicate ~W-cut  
together with the catalogue number  
e. g.: BC211W)

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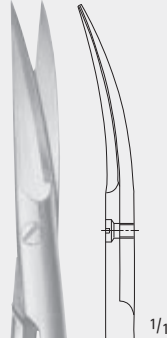
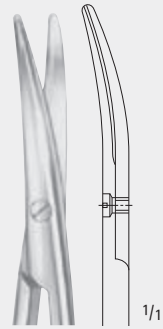
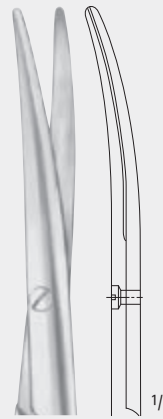
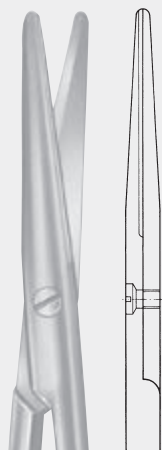
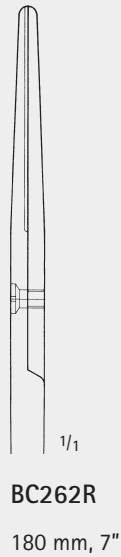
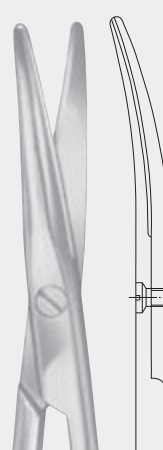
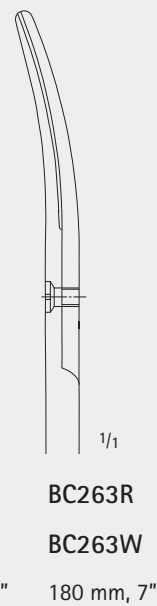
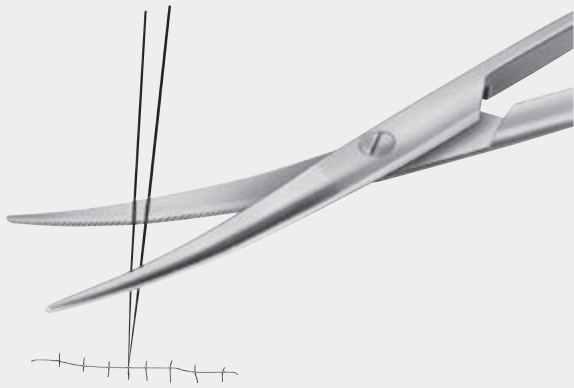
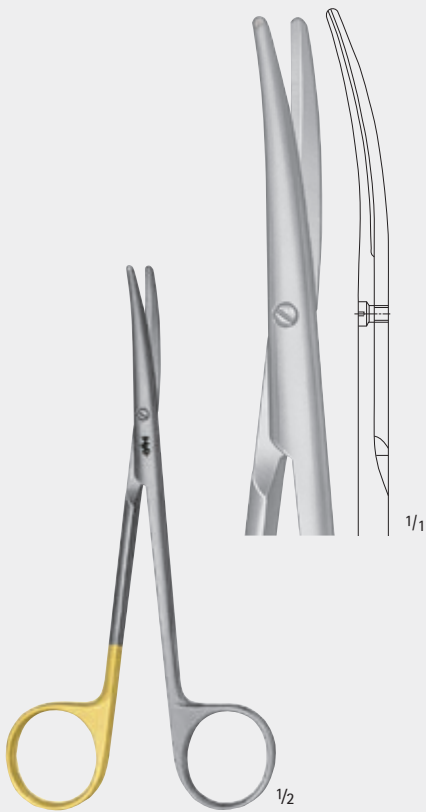
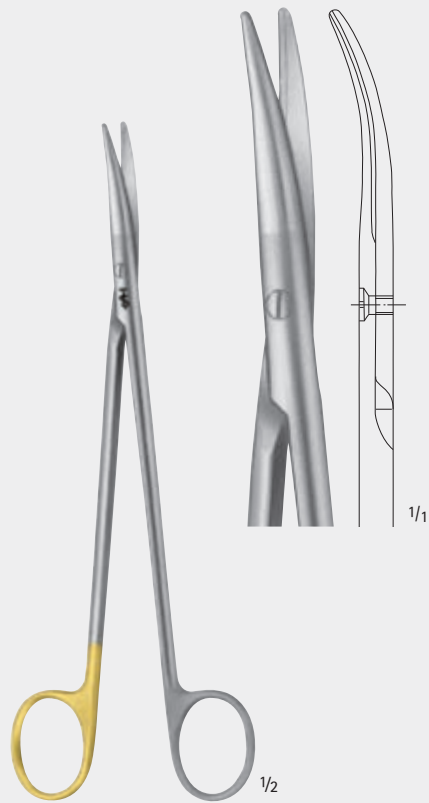
Especially for cutting suture materials

- DUROTIP® ligature scissors are identified by  
1 black and  
1 goldplated ring

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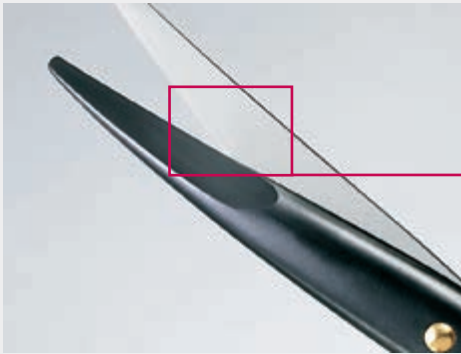
The special qualities of the Noir® dissecting scissors allow dissection that is finer than ever before. Noir®. Gentle dissection with minimum use of force.

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Microtome edge - the extremely sharp cutting edge of the scissors blade facilitates gentle cutting of tissue with minimal use of force.

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The combination of the renowned AESCULAP® tungsten carbide technology with the AESCULAP® Noir® coating results in an extremely high surface hardness and therefore high wear resistance in daily hospital use.

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The extremely hard (2000HV) and low reflection Noir® coating leads to high wear resistance of the instruments and at the same time provides high resistance against corrosion.

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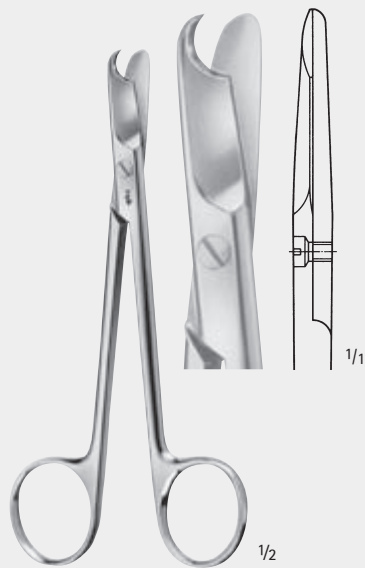
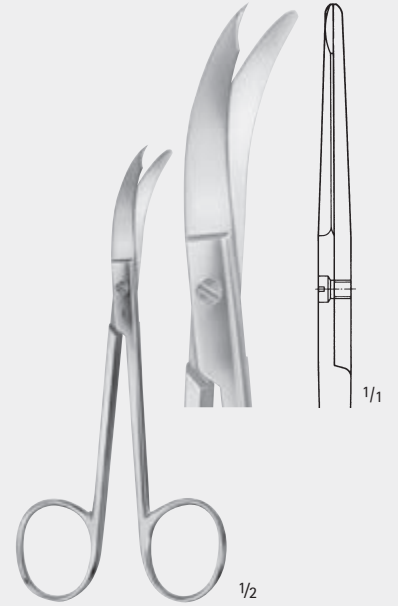
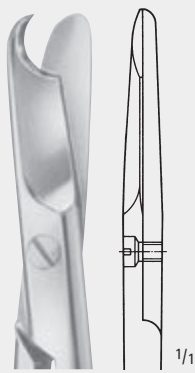
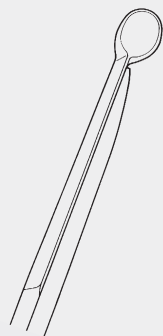
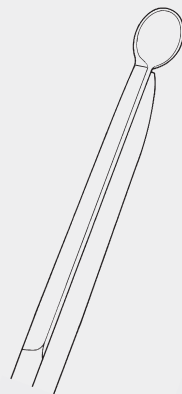
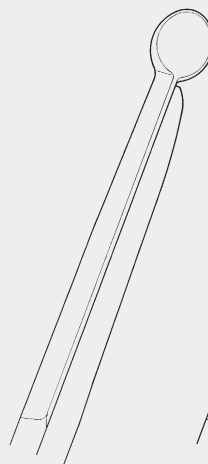
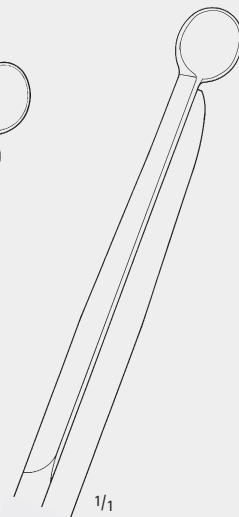
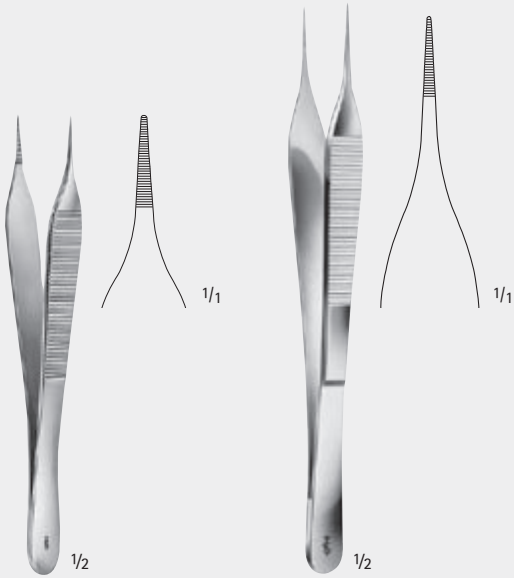
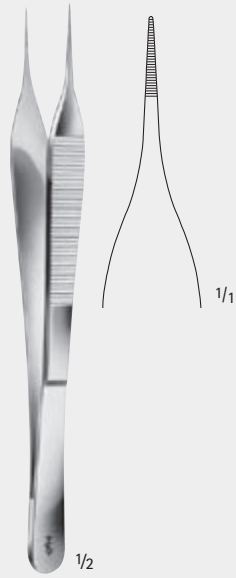
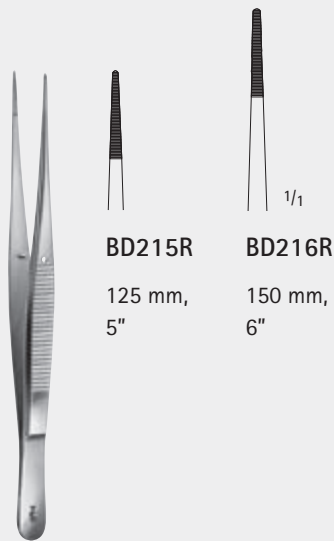
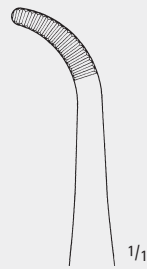
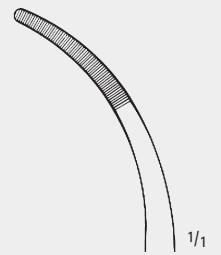
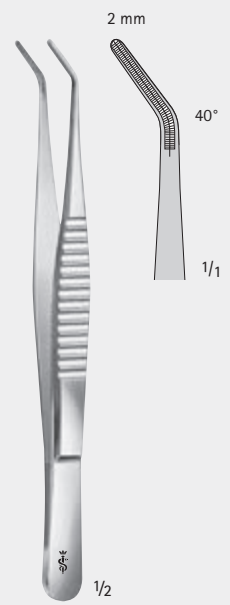
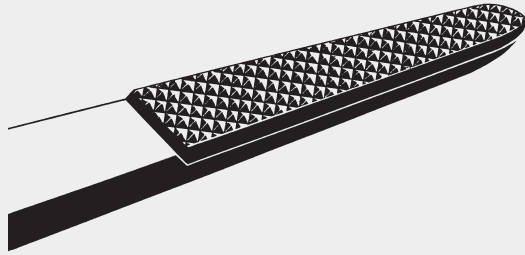
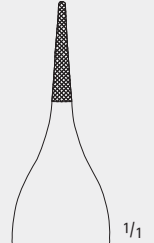
DUROGRIP®-Dissecting Forceps with carbide inserts  
Identification: gold-plated end

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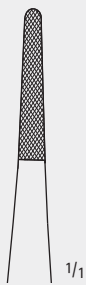
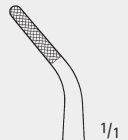
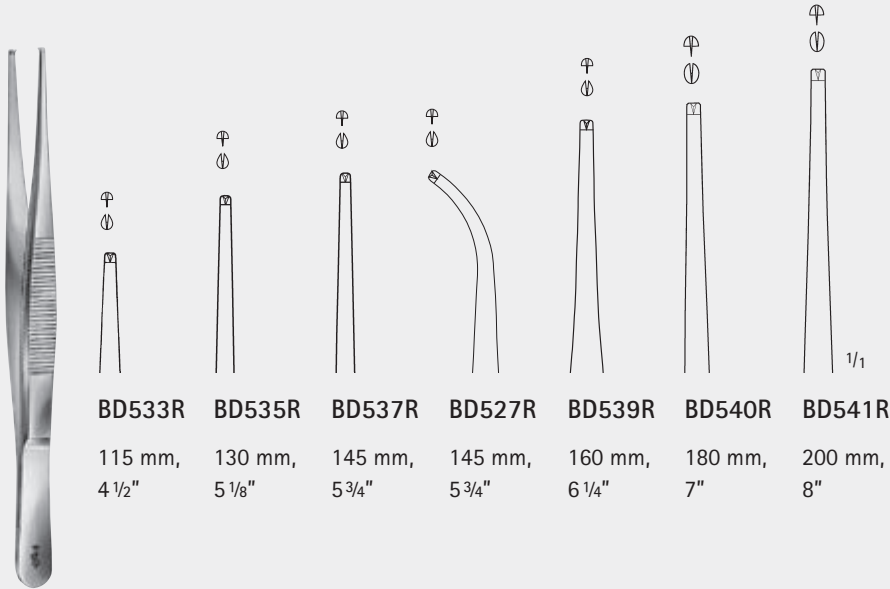
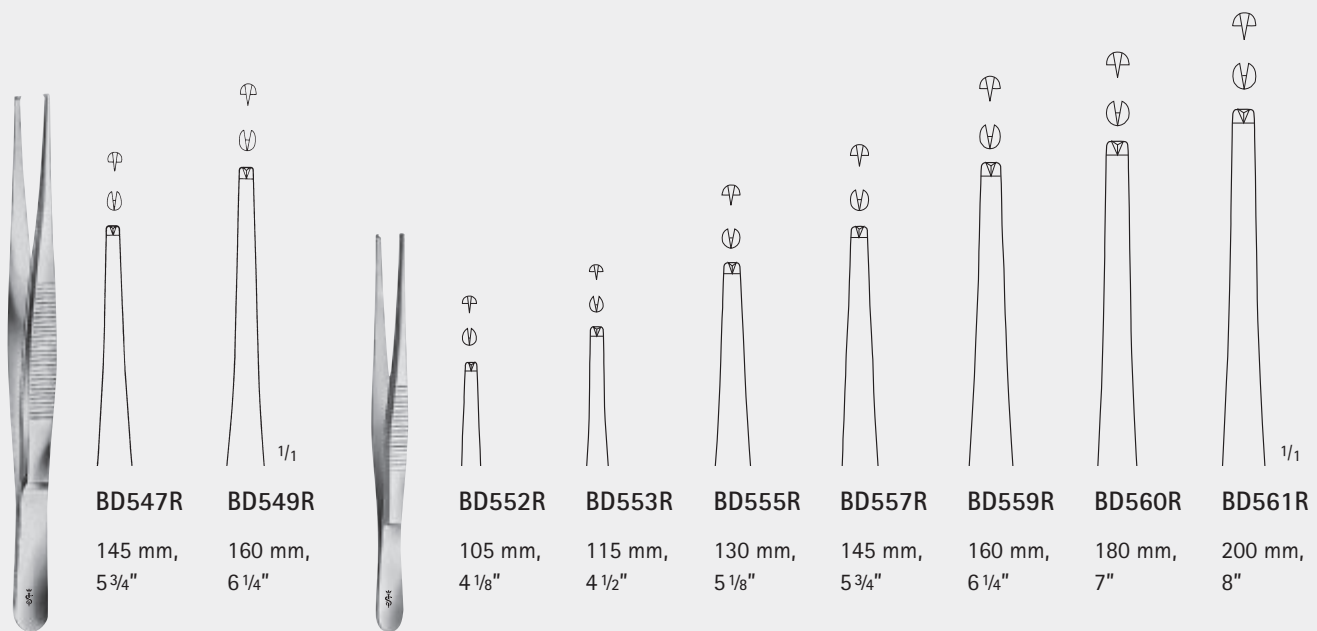
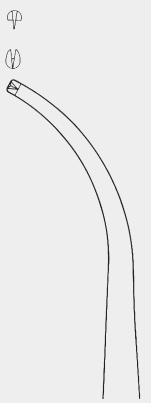
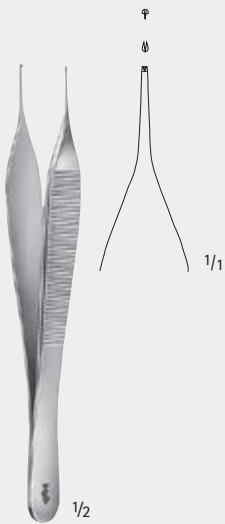
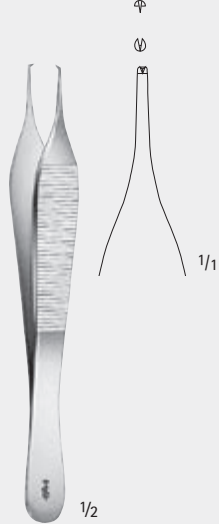
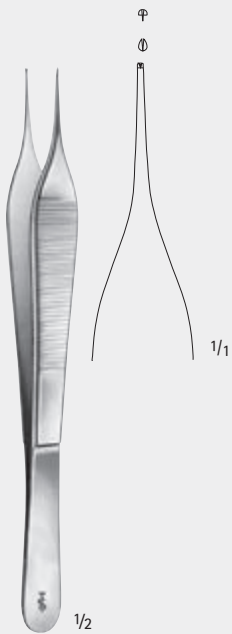
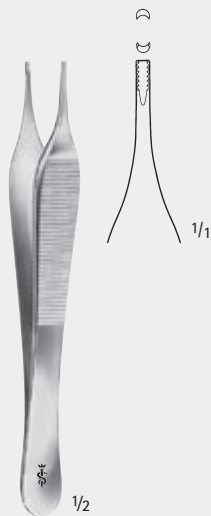
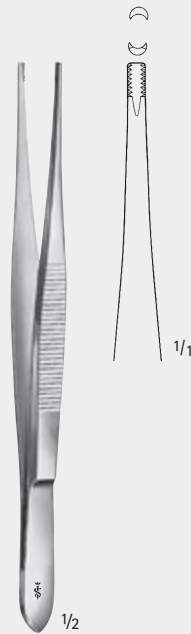
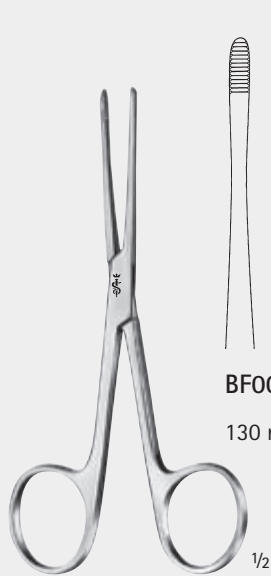
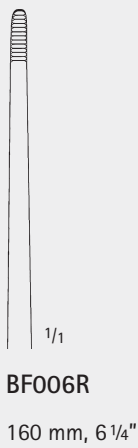
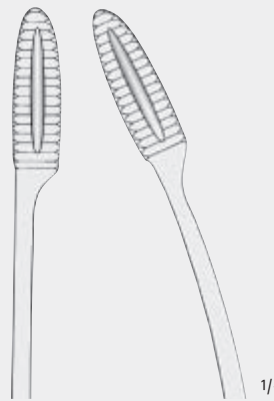
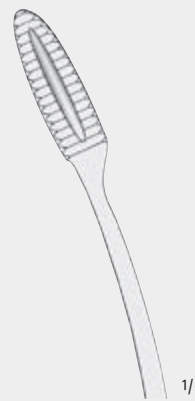
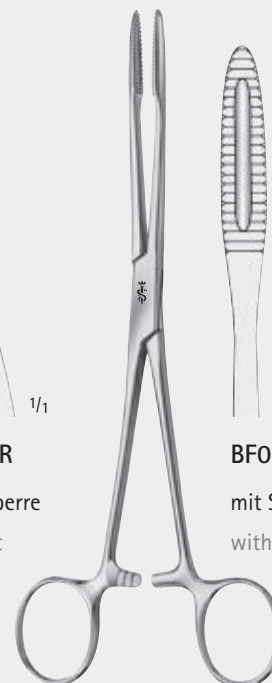
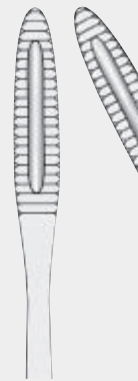
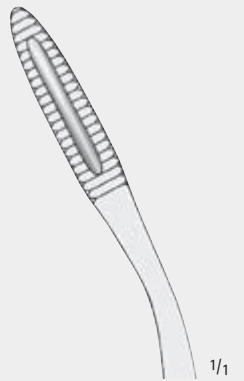
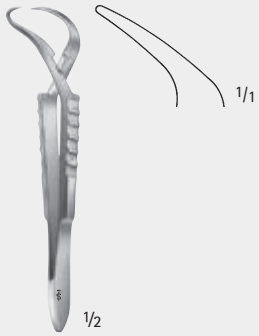
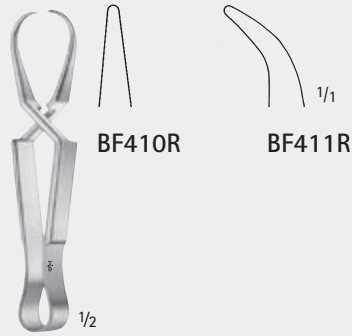
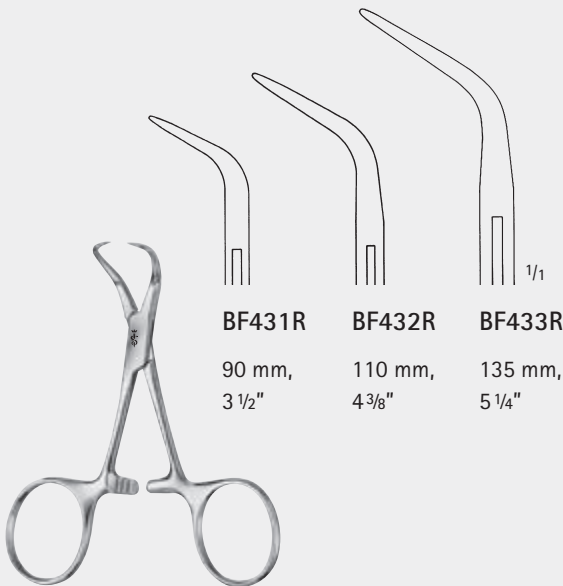
JONES  
**BF407R**  
 Tuchklemme  
 Towel Clamp  
 85 mm, 3 3/8"

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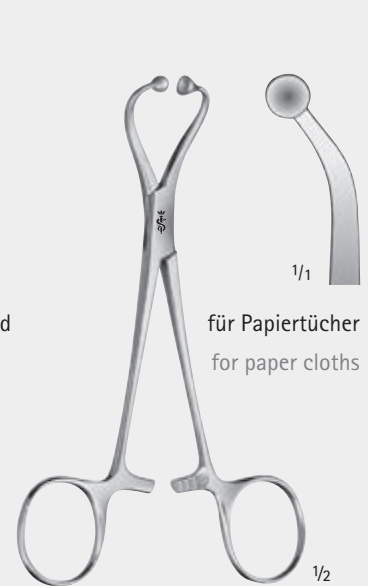
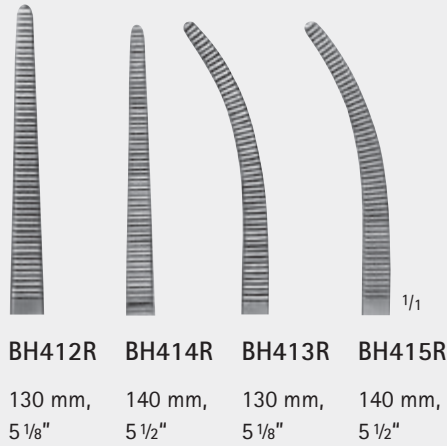
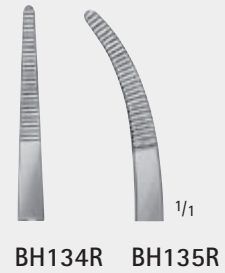
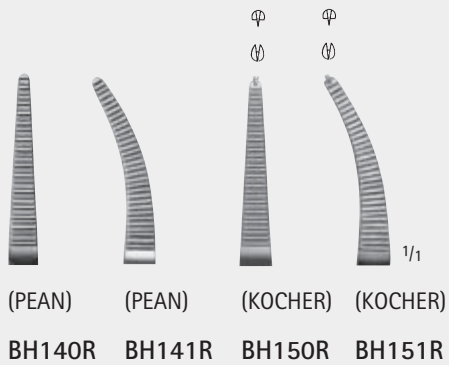
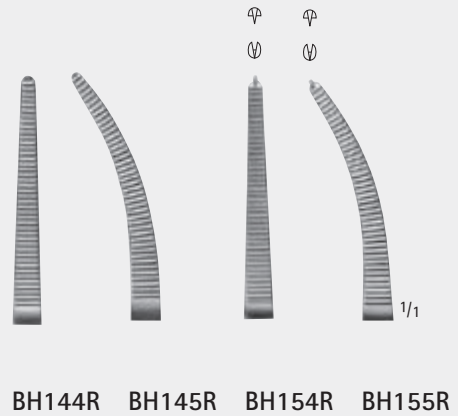
PEAN  
**BH412R – BH415R**  
 Grazile Form  
 Slender type

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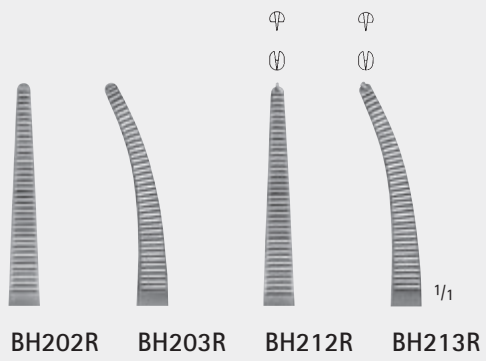
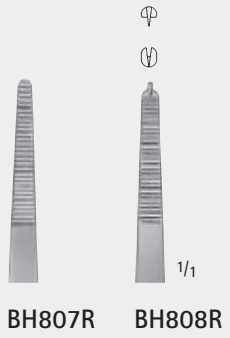
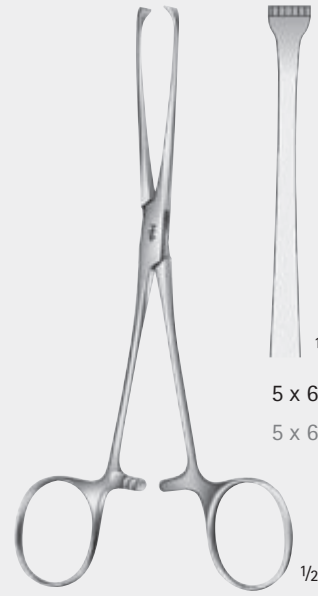
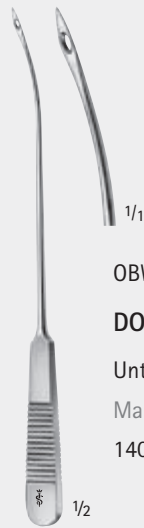
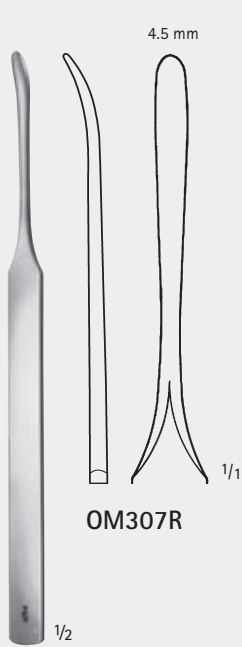
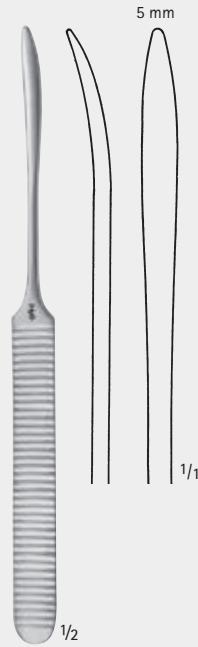
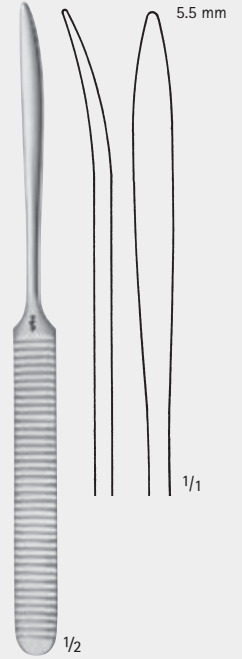
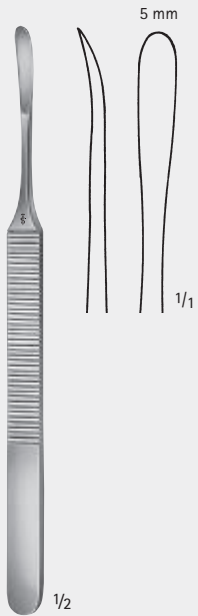
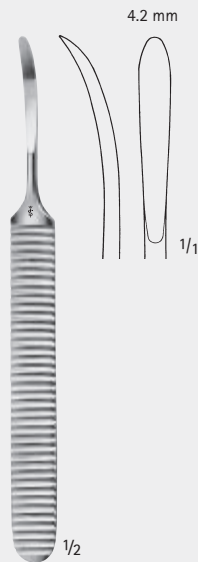
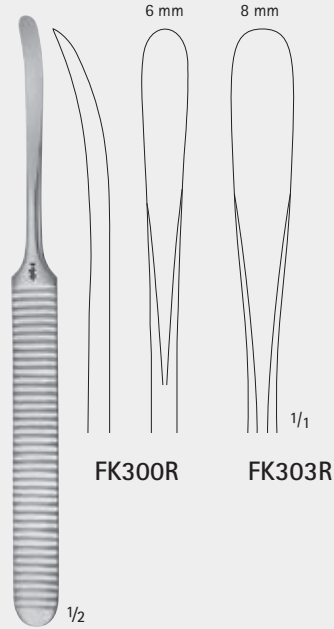
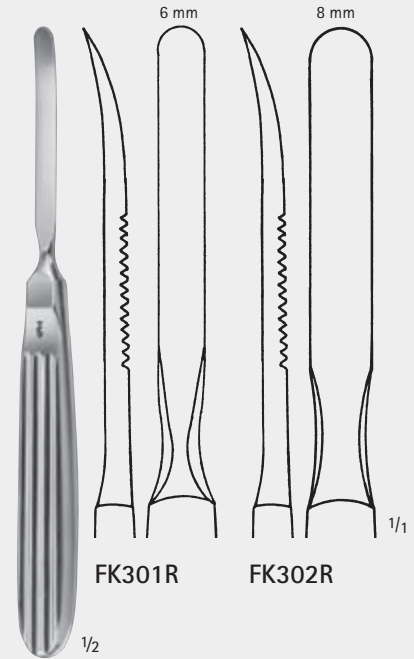
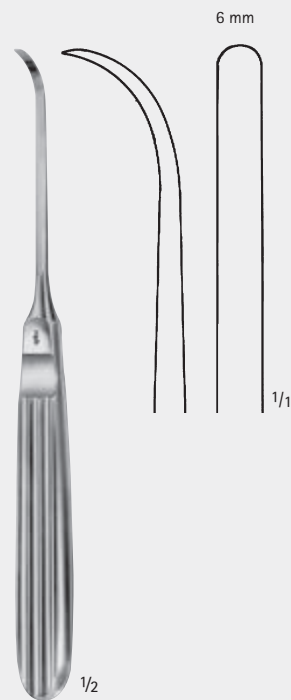
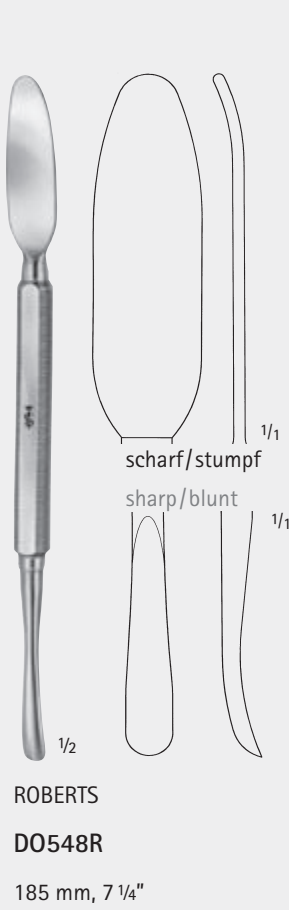
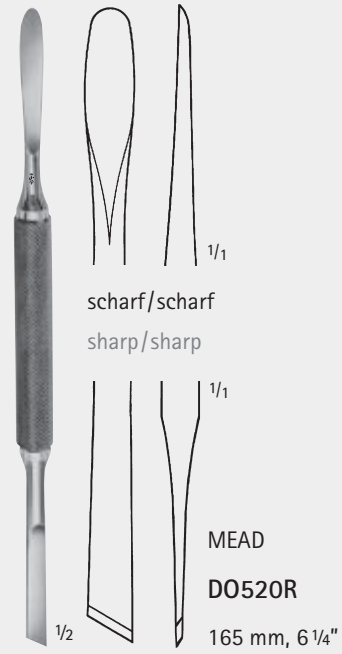
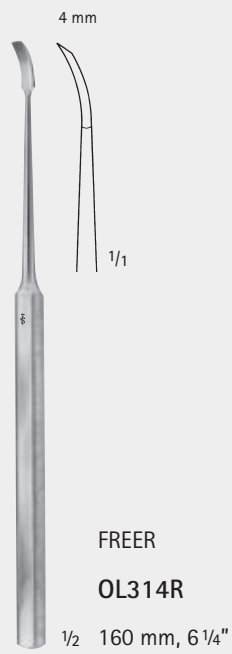
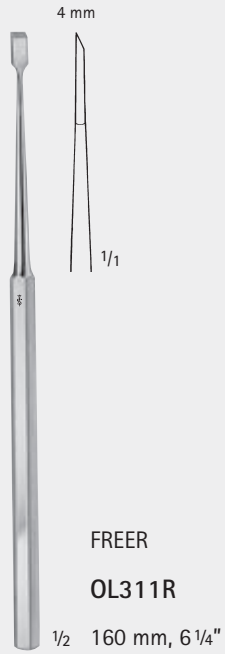
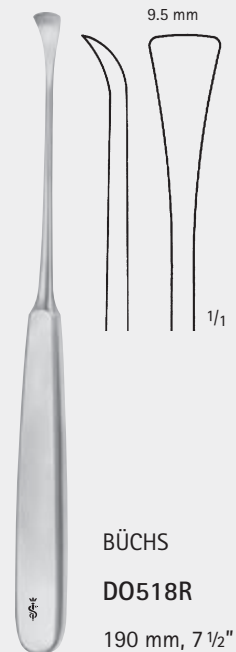
**Range of application:**

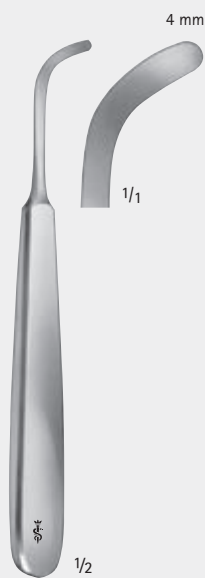
- For loosening parts of the mucosa
- For cyst resection
- For parodontal surgery

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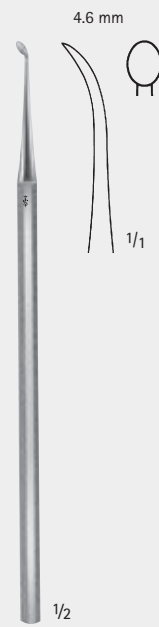
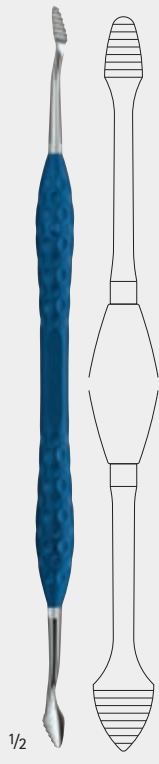
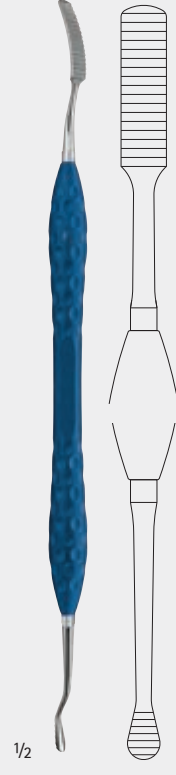
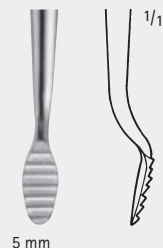
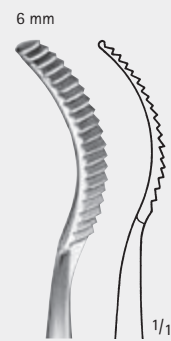
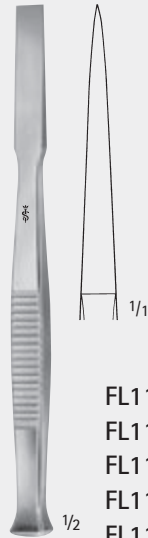
LAMBOTTE

- FL101R 4 mm
- FL102R 6 mm
- FL104R 8 mm
- FL106R 10 mm
- FL108R 12 mm
- FL109R 15 mm

Osteotome

- Osteotomes
- 125 mm, 5"

- FL110R 4 mm
- FL112R 6 mm
- FL114R 8 mm
- FL116R 10 mm
- FL118R 12 mm

Osteotome

- Osteotomes
- 140 mm, 5 1/2"

PARTSCH

- FL173R 3 mm
- FL174R 4 mm
- FL175R 5 mm
- FL176R 6 mm

Flachmeißel

- Chisels
- 140 mm, 5 1/2"

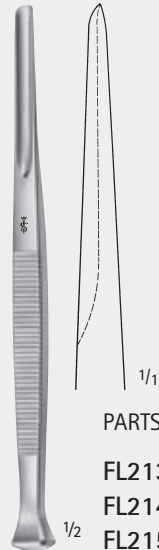
PARTSCH

- FL183R 3 mm
- FL184R 4 mm
- FL185R 5 mm
- FL186R 6 mm
- FL187R 7 mm

Flachmeißel

- Chisels
- 170 mm, 6 3/4"

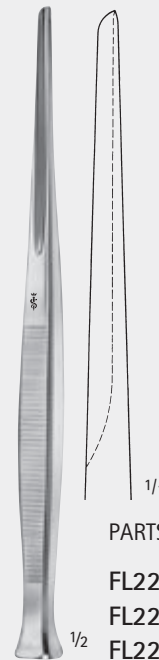
PARTSCH

- FL213R 3 mm
- FL214R 4 mm
- FL215R 5 mm
- FL216R 6 mm

Hohlmeißel

- Gouges
- 140 mm, 5 1/2"

PARTSCH

- FL223R 3 mm
- FL224R 4 mm
- FL225R 5 mm
- FL226R 6 mm
- FL227R 7 mm

Hohlmeißel

- Gouges
- 170 mm, 6 3/4"

FREER  
**OL311R** 4 mm  
Flachmeißel  
Chisel  
160 mm, 6 1/4"

1/1

FREER  
**OL314R** 4 mm  
Flachmeißel, gebogen  
Chisel, curved  
160 mm, 6 1/4"

1/1

LUCAS  
**DO711R** 4 mm  
Flachmeißel  
Chisel  
160 mm, 6 1/4"

1/1

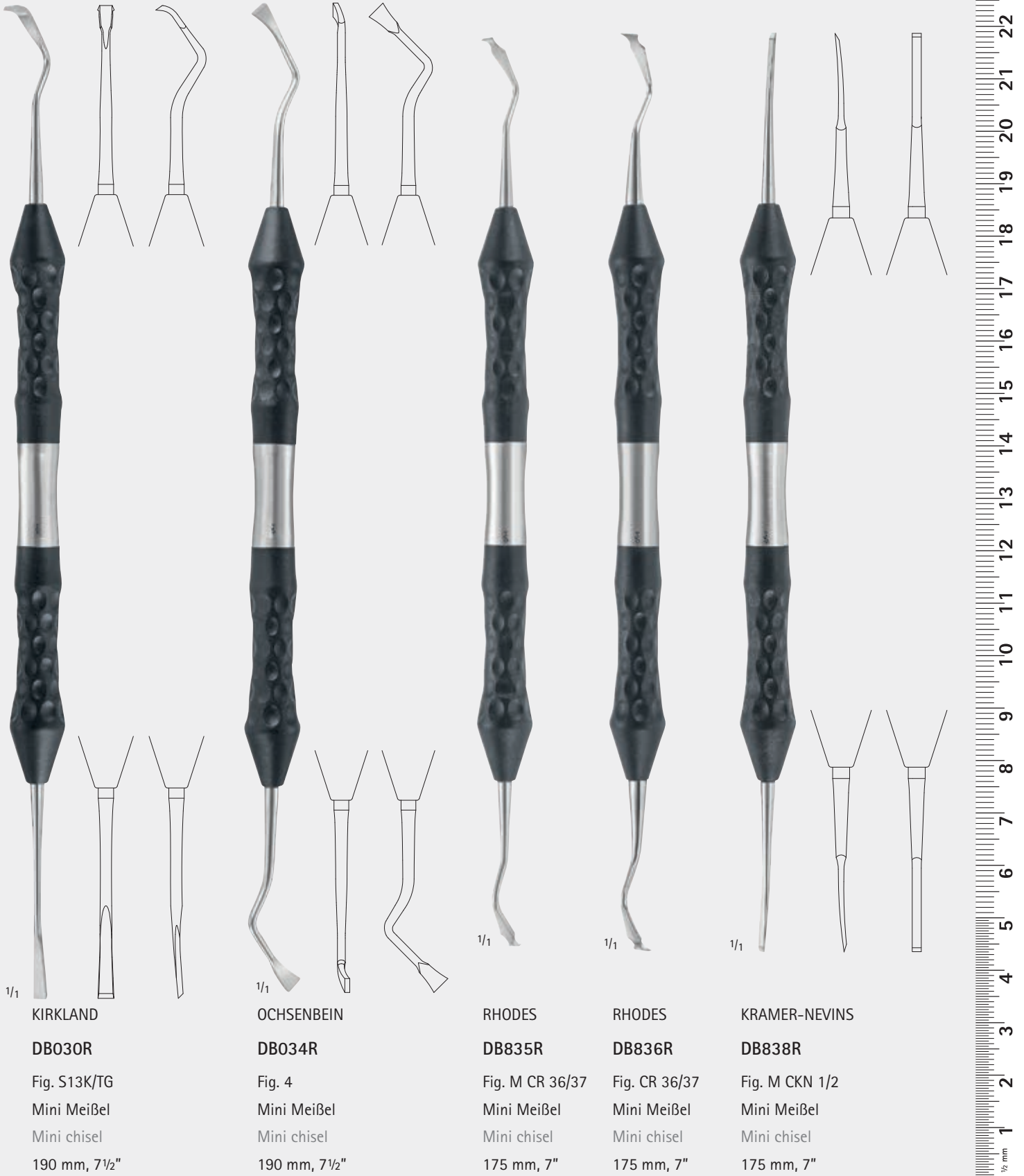
LUCAS  
**DO715R** 3 mm  
Hohlmeißel  
Gouges  
160 mm, 6 1/4"

1/1

MEISSEL, OSTEOTOME  
CHISELS, OSTEOTOMES

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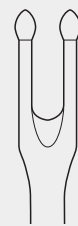
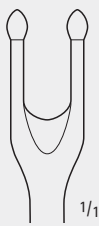
Osteotome

185 mm, 7 1/4"

STÖSSEL, MEISSEL, OSTEOTOME  
TAMPERS, CHISELS, OSTEOTOMES

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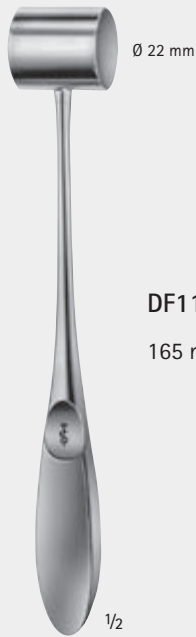
Weights refer to the heads of the mallets

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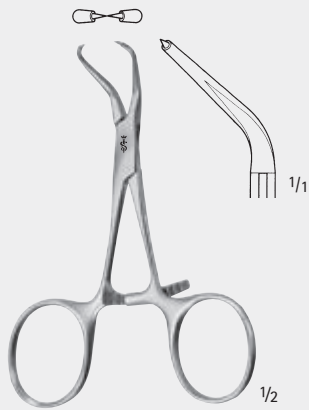
ERGOPLANT Mallet,  
with interchangeable plastic disks  
200 mm, 8"

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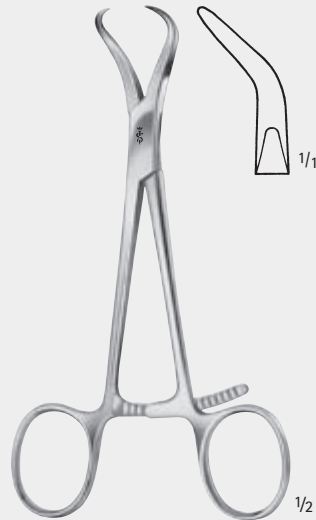
Reposition forceps with stepped tips,  
long ratchet

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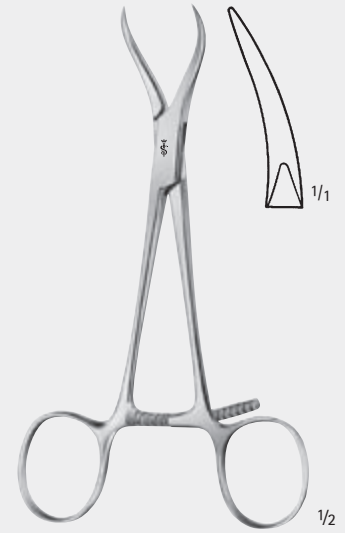
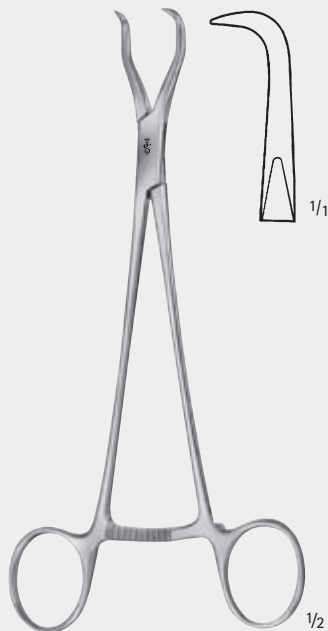
Reposition forceps,  
especially for maxillary reposition

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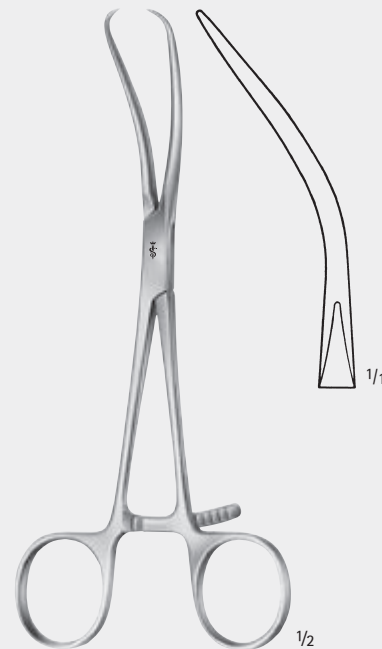
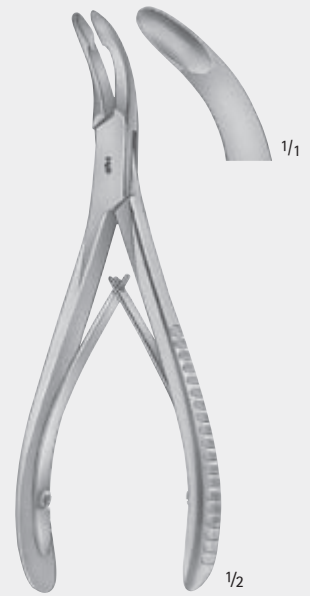
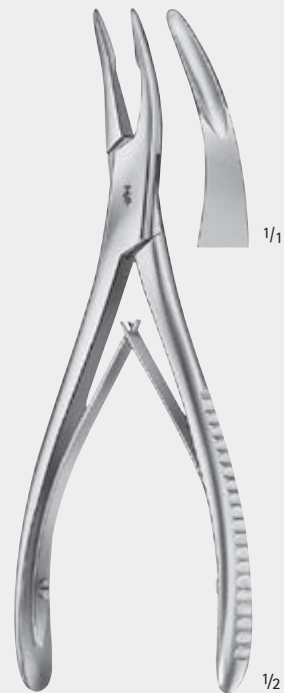
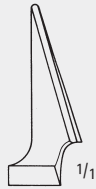
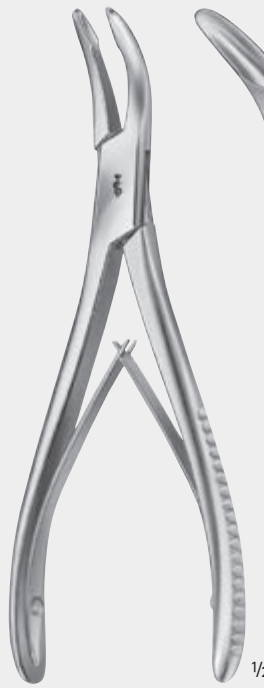
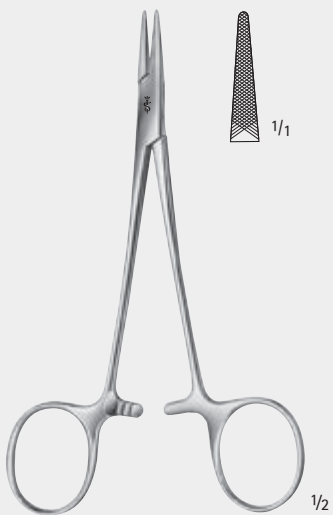
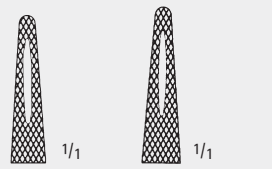
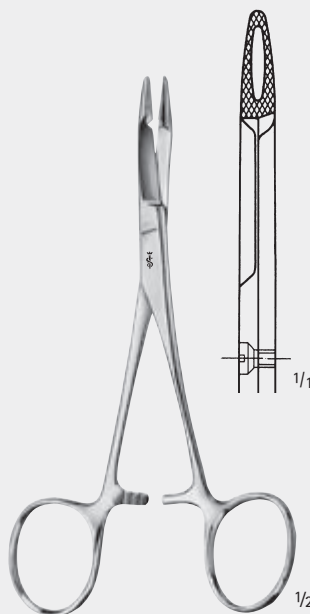
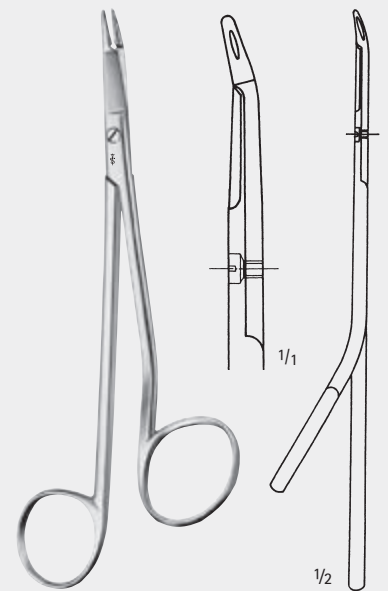
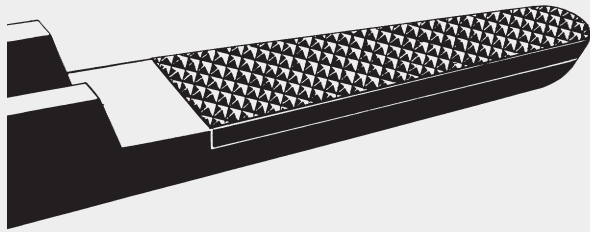
AESCULAP® needle holders have jaws with carbide inserts and according to the use required the resp. structure can be chosen. A smooth jaw surface as well as 3 different serrations enable a „grip“ of the respective needle.

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For holding tissue and membranes. The slit in the working tip serves for precise positioning of the needle.

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9/0 – 11/0  
6/0 – 10/0  
4/0 – 6/0  
bis 3/0

We recommend carbide inserts

with smooth jaw surface for suture material with 0.2 mm pitch of serration for suture material 0.4 mm 0.5 mm

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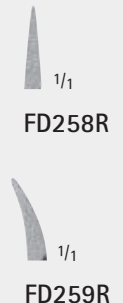
DUROGRIP® Needle holders with carbide inserts

Identification: gold-plated rings, handle ends and ratchets

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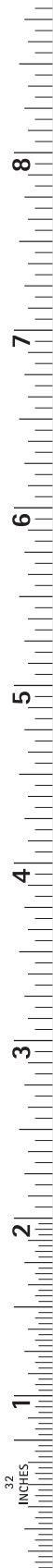
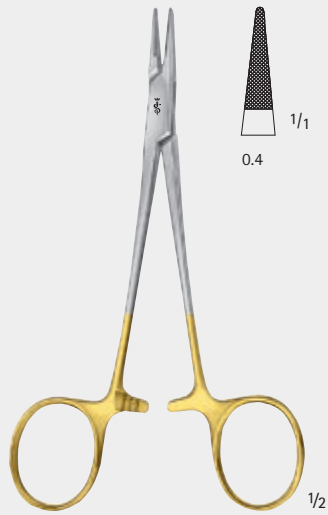
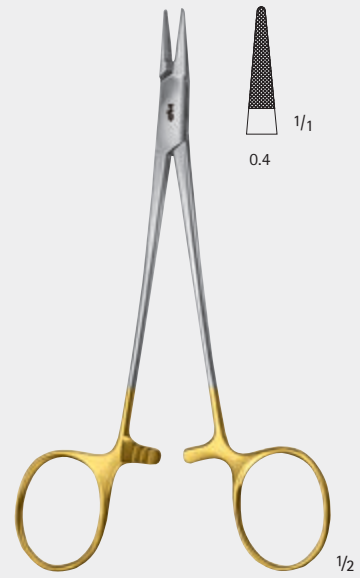
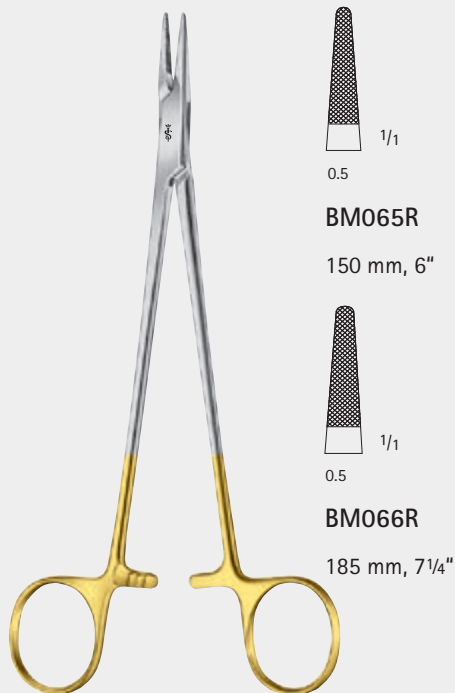
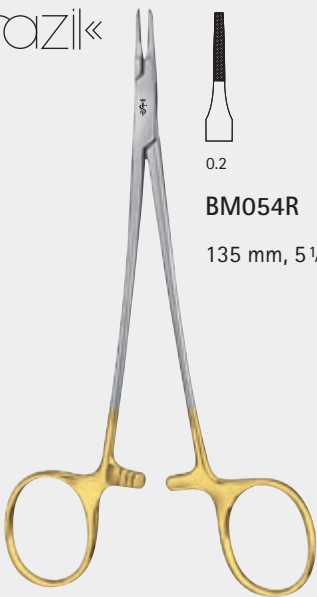
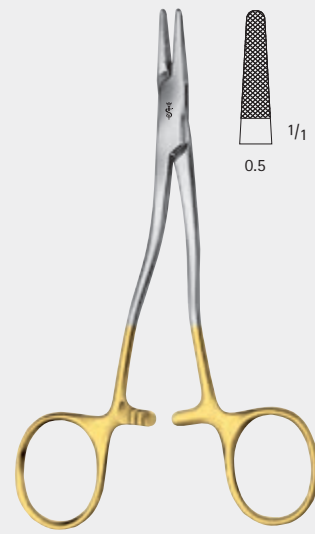
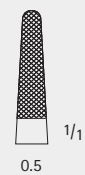
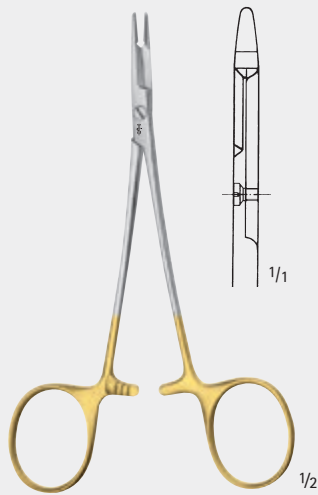
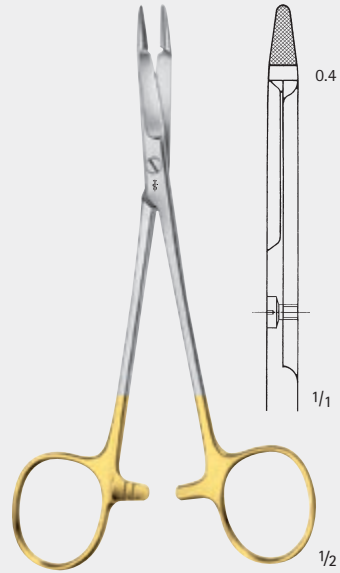
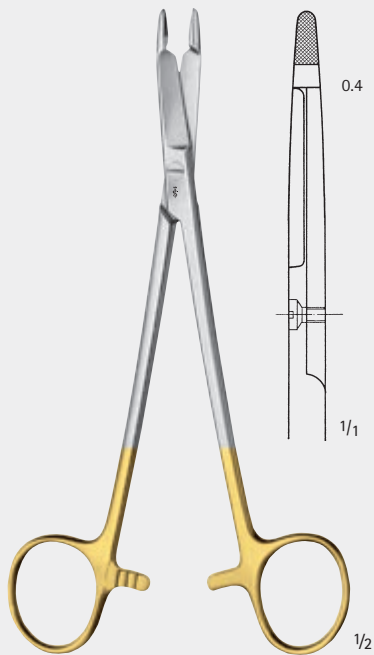
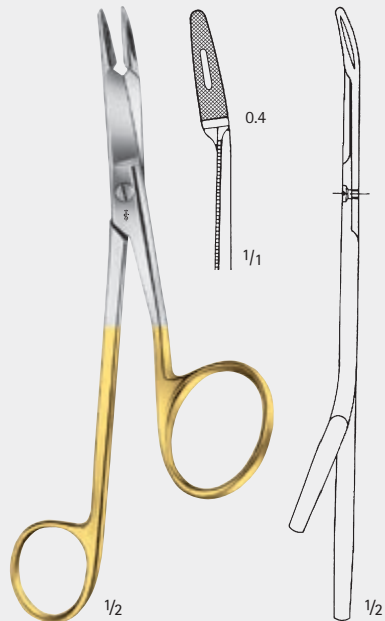
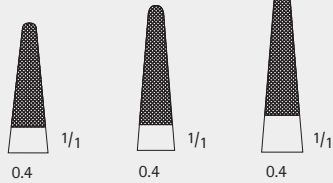
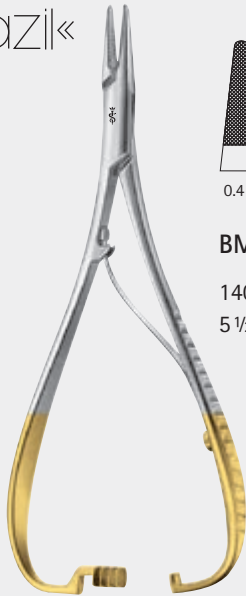
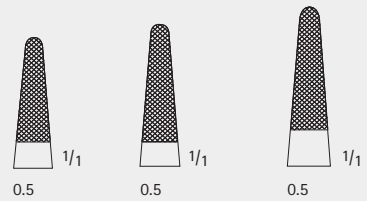
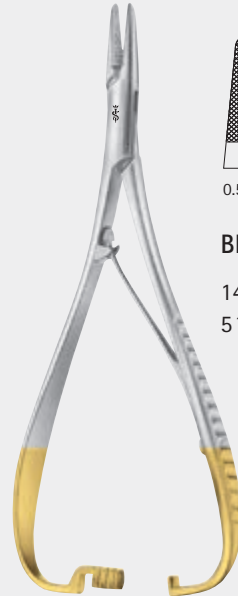
**BL930R** 65 x 13 mm

**BL932R** 95 x 16 mm

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Needle case complete,  
with interior box

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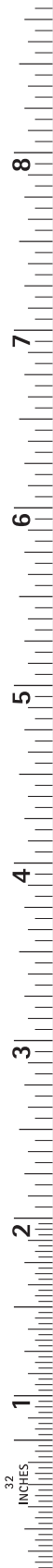
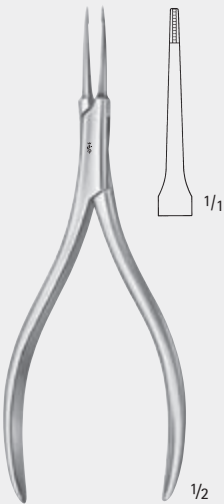
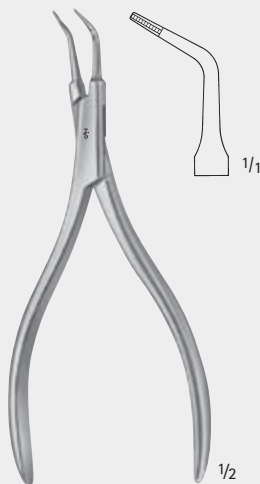
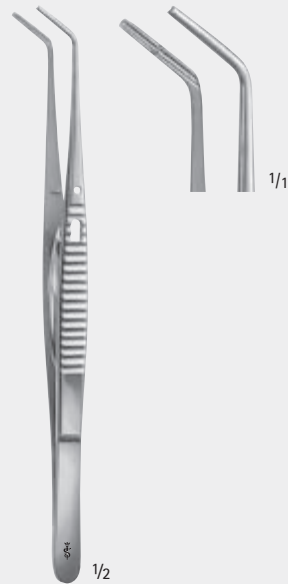
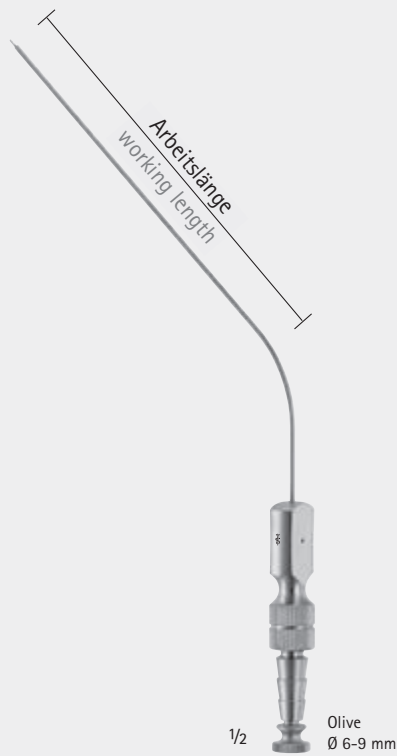
With finger cut-off and stylet, thumb piece parallel to curve, with LUER hub

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With finger cut-off and stylet, thumb piece 90° to curve, with LUER hub

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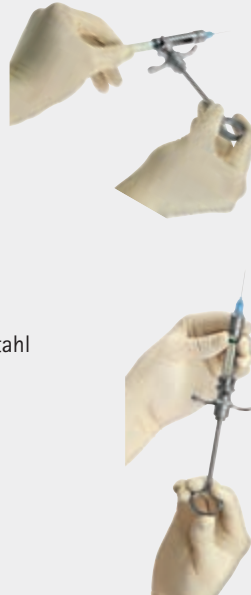
AESCULAP® Cartridge Syringe „Aspira-plus II“ made of stainless instrument-steel (no chrome), with aspiration device

After having inserted the ampulla and the cannula, the syringe is immediately ready for injection.

By a slight twist of the piston rod to the right the double corkscrew tip will grasp the foam rubber of the ampulla.

This enables a controlled aspiration.

### Advantages:

- Processing by using corrosion- and acid resistant instrument steel instead of chrome material
- Cartridge insert and ejection by folding back the bush

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Needle attachment, long, suitable for all disposable needles with a thread diameter of 6 mm and long cartridge pick (e.g. Sterican® Z, B. Braun etc.)

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AESCULAP® Single Use Surgical Instruments

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For further details see  
brochure No. **C82002**

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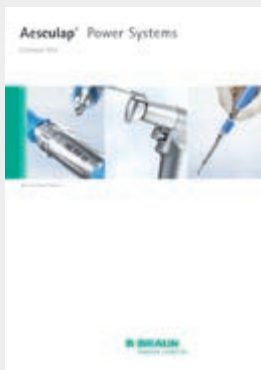
For further details see  
brochure No. **4008000**

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For further details see  
brochure No. **O22702**

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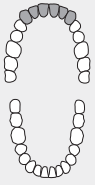
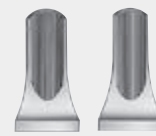
For upper incisors and canines, wide

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For upper crowded incisors and canines

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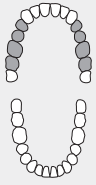
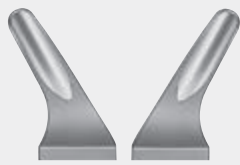
The tooth forceps is positioned between the 7th and 8th tooth. By pressing the working ends together, the wisdom tooth can be removed.

ZAHNZANGEN, KINDERZAHNZANGEN, AMERIKANISCHES MODELL  
 TOOTH FORCEPS, TOOTH FORCEPS FOR CHILDREN, AMERICAN PATTERN

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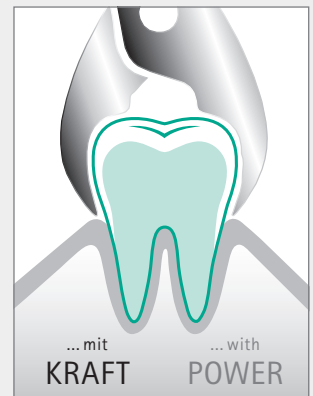
#### THERE ARE DIFFERENT WAYS TO EXTRACT A TOOTH ...

The importance of tissue preserving extraction is growing strongly. Especially in the field of implantology tissue preserving can avoid complex bone augmentation (1, 2). ExtraLux instruments enable easy and comfortable tissue preserving extraction of all teeth by

- Sharp working tips: Convenient cutting of the Sharpey bundles
- Thin working tips: Bone preserving cutting of the Sharpey's fibers  
Easy access to the Periodontology gap
- Ergonomics: Enables a better control of force
- Haptics: Instruments are also handy intra-operatively

(1) Muska E, Walter C, Knight A, Taneja P, Bulsara Y, Hahn M, Desai M, Dietrich T. Atraumatic vertical tooth extraction: a proof of principle clinical study of a novel system. Oral Surg Oral Med Oral Pathol Oral Radiol. 2013 Nov;116(5):303-10.

(2) Sharma SD, Vidya B, Alexander M, Deshmikh S. Periosteum as an Aid to Atraumatic Extraction: A Comparative Double Blind Randomized Controlled Trial. J Maxillofac Oral Surg. 2015 Sep;14(3):611-5.

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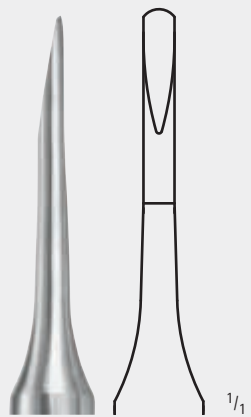
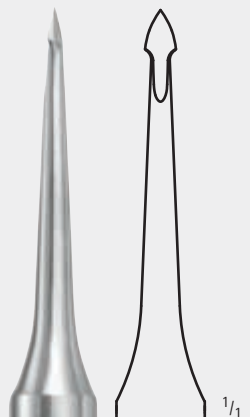
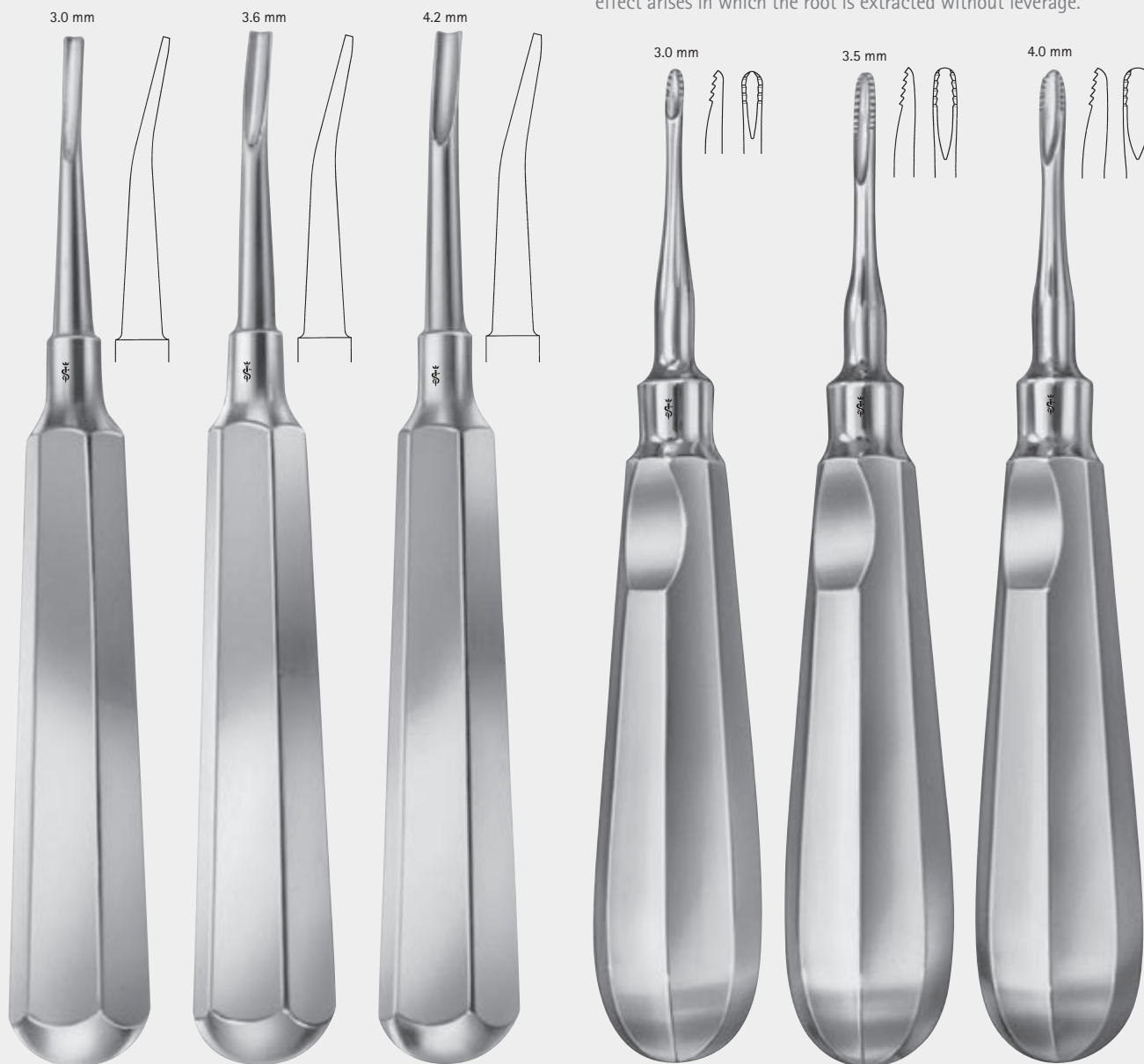
With a slight rotary movement, the LINDO-LEVIAN root elevator is pushed between alveolus and root until the groove becomes lodged in the roots. By pulling the root elevator straight down, the barb effect arises in which the root is extracted without leverage.

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By rotating the handle (perpendicular to its long axis) the luxation force is directed distally to elevate the tooth out of the alveolus.

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The luxator is inserted between the wisdom tooth and the second molar pushing it firmly to the level of the gingival tissue.

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**Important note:**

The luxation of the wisdom tooth is not achieved by using the second molar as the leverage point for the instrument. The instrument should be used with a rotational force in distal direction to luxate the tooth in a buccal movement.

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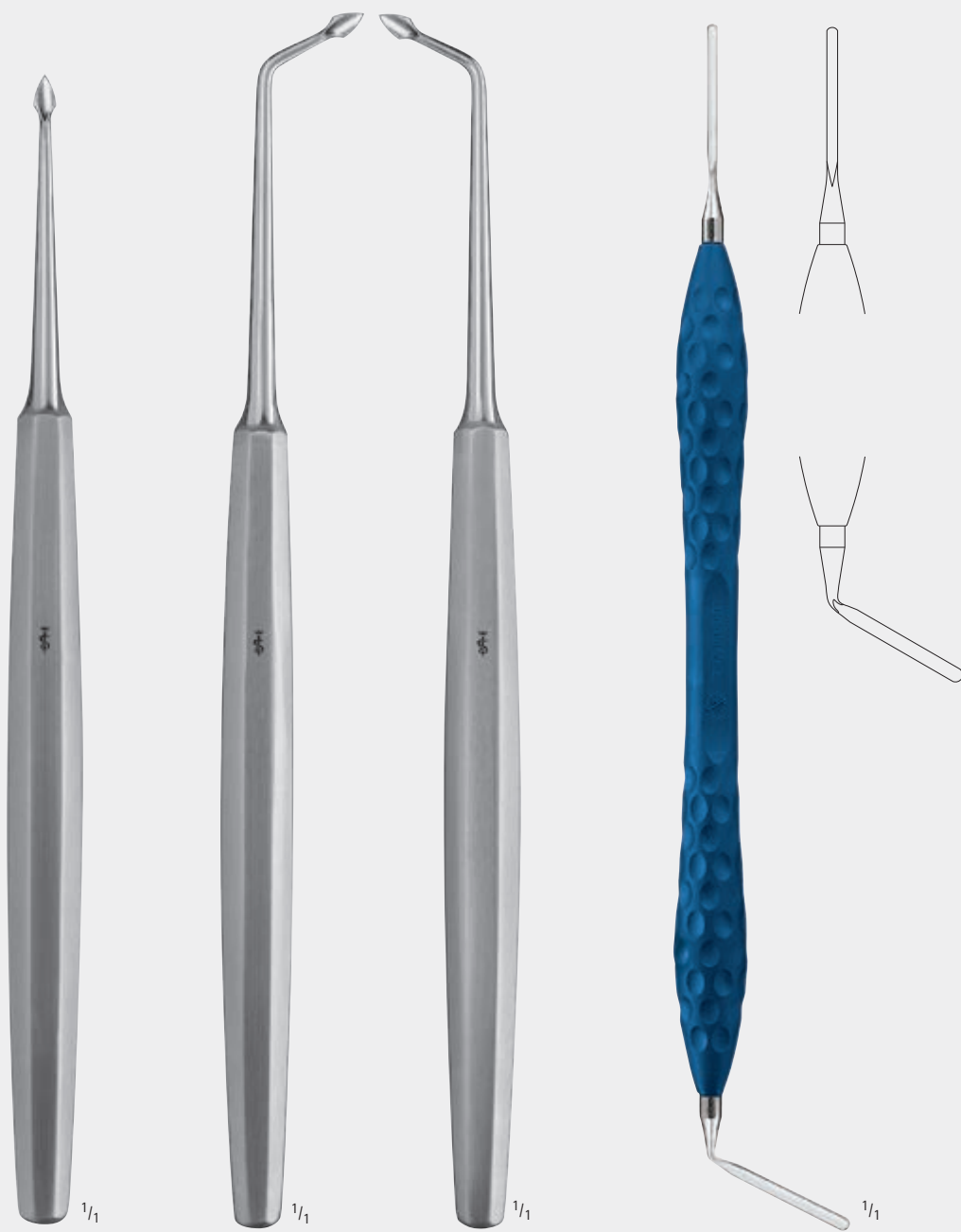
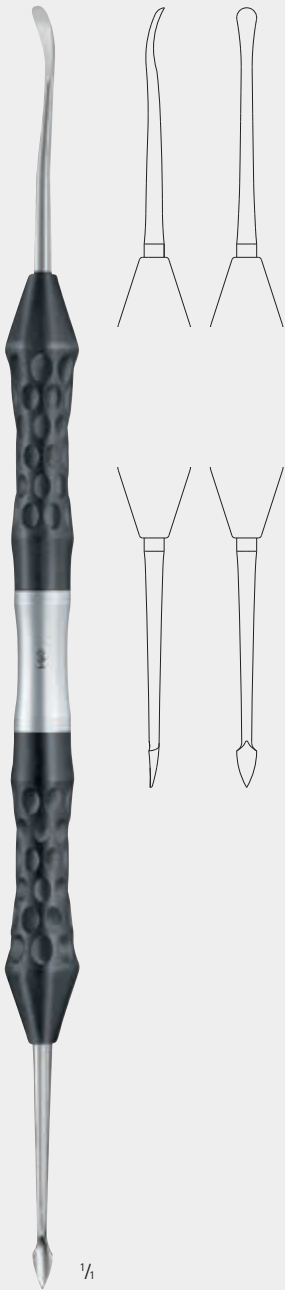
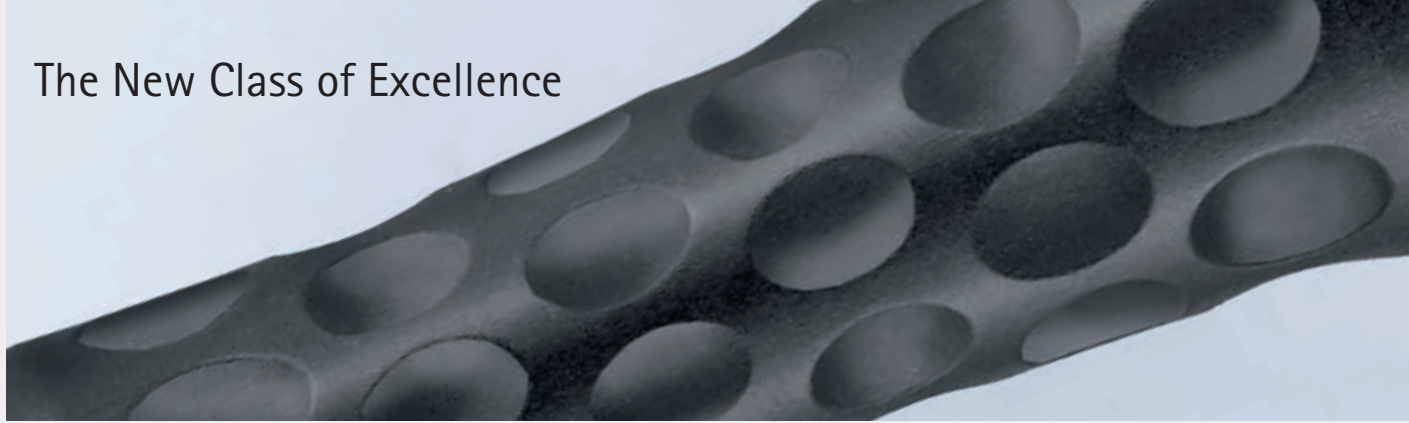
Continuous corrosion-resistant  
instrument steel

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Instrument tight-covered  
by a PEEK handle

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In close collaboration with **Dr. Dr. Andres Stricker, Freiburg University**, an instrument range especially designed for dental implantology has been developed for ergonomics concerning the working tips and handle for any specific indication. ERGOPLANT thus enables working efficiently with minor physical strain under practice conditions. All ERGOPLANT instruments are suitable for steam sterilization 134°C (Autoclav).

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Titanium-microblade holder with round handle, light. Easily rotated and easy to handle for precision incisions.

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Sales unit: **PAK** = Package of 1 piece

Mikrochirurgische Klinge, stainless steel, steril verpackt

Microsurgical blade, stainless steel, sterile packed

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Scalpel blades, vacuum heat treated, sterile packed

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Scalpel blade holder with round handle

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**PRICHARD**  
**DX200R**  
Raspatorium  
Raspatory  
200 mm, 8"

BASIS INSTRUMENTE  
BASIC INSTRUMENTS

PRICHARD

DB020R

Raspatorium, grazil  
Raspatory, delicate

195 mm, 7<sup>3</sup>/<sub>4</sub>"

MOLT

DB023R

Raspatorium  
Raspatory

190 mm, 7<sup>1</sup>/<sub>2</sub>"

FREER

DB025R

Raspatorium, scharf / stumpf  
Raspatory, sharp / blunt

190 mm, 7<sup>1</sup>/<sub>2</sub>"

BUSER  
**DX301R**  
Mikro-Raspatorium  
Micro-Raspatory  
180 mm, 7 1/2"

FRETWURST  
**DB024R**  
Raspatorium/Desmotom  
Raspatory/Desmotomes  
190 mm, 7 1/2"

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Atraumatic dissecting of the flap by means of the papilla elevator without any damage to the papillae.

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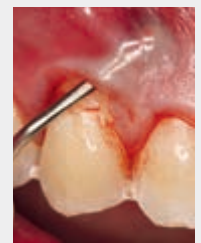
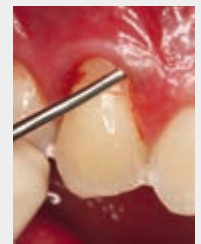
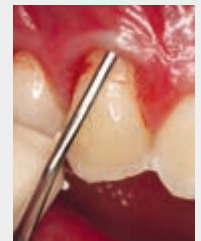
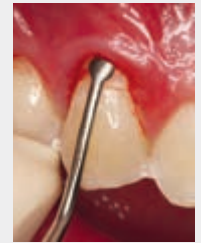
The straight tunnelling instrument is mainly used in the upper jaw, in the frontal and lateral region. This allows large areas of the gingiva to be undermined using both tunnel and sulcular techniques.

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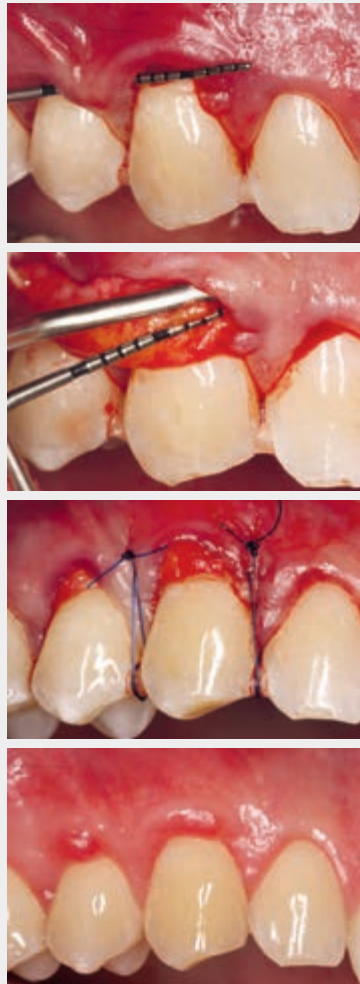
The angled tunnelling instrument is primarily used for preparing lateral, buccal regions. It can also be used distally, without tearing the mucosal tissue and also for the lower jaw.

195 mm, 7<sup>3</sup>/<sub>4</sub>"

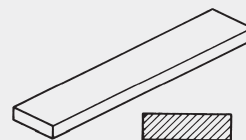
JG598

Sterilit® i AESCULAP®-Schleifstein-Öl

Inhalt: 50 ccm

Sterilit® i AESCULAP® oil for grinding stones

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Original Arkansas grinding stone,  
for grinding on tunnelling instruments

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- 3 Jahre Garantie
- Vergoldete Ringe

DUROTIP®-Scissors with inserted carbide cutting edges

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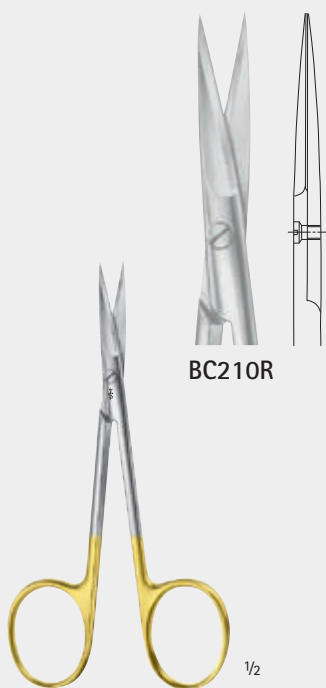
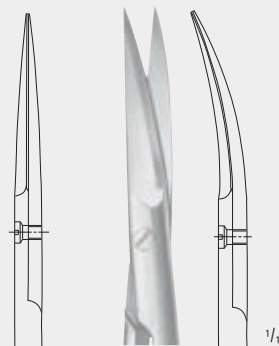
**W**-Cut directly after the catalogue number indicates that the resp. item has one serrated cutting edge. (When ordering please indicate **W**-cut together with the catalogue number e.g.: BC211W)

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DUROGRIP® Needle holder  
for suture material 4/0 – 6/0

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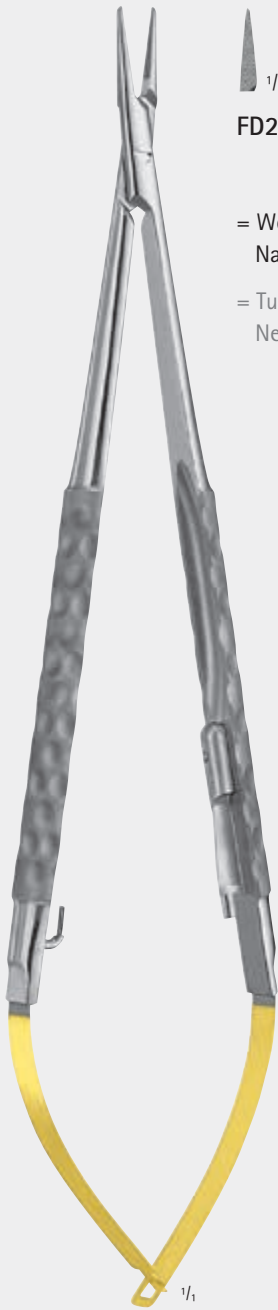
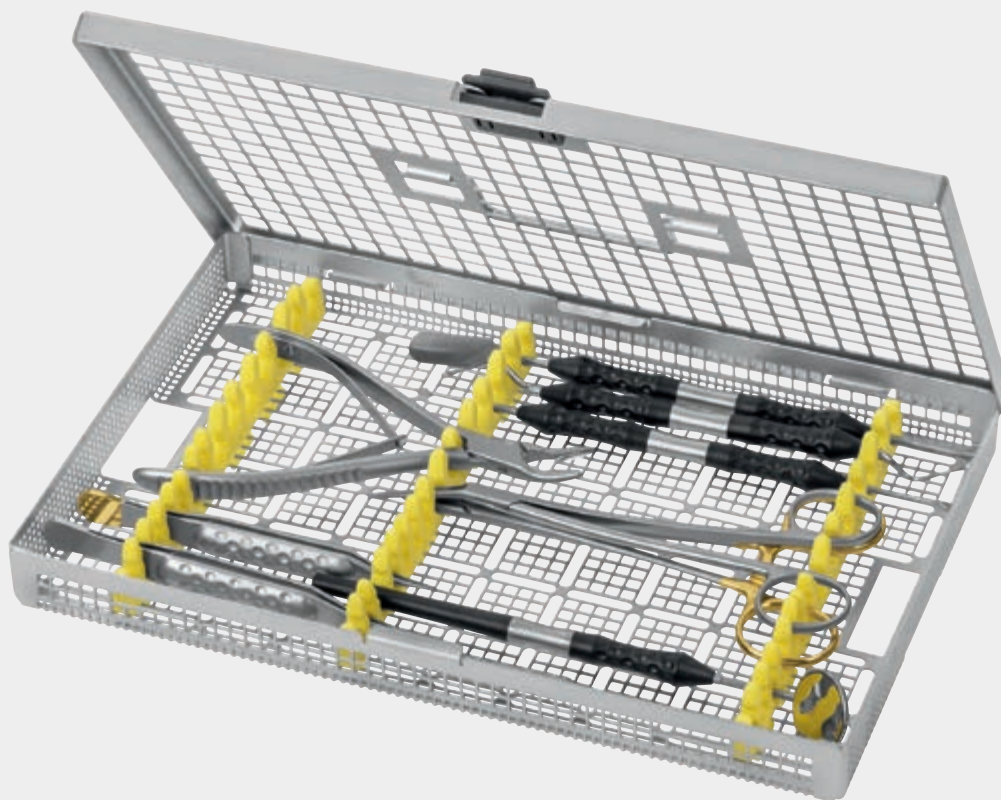
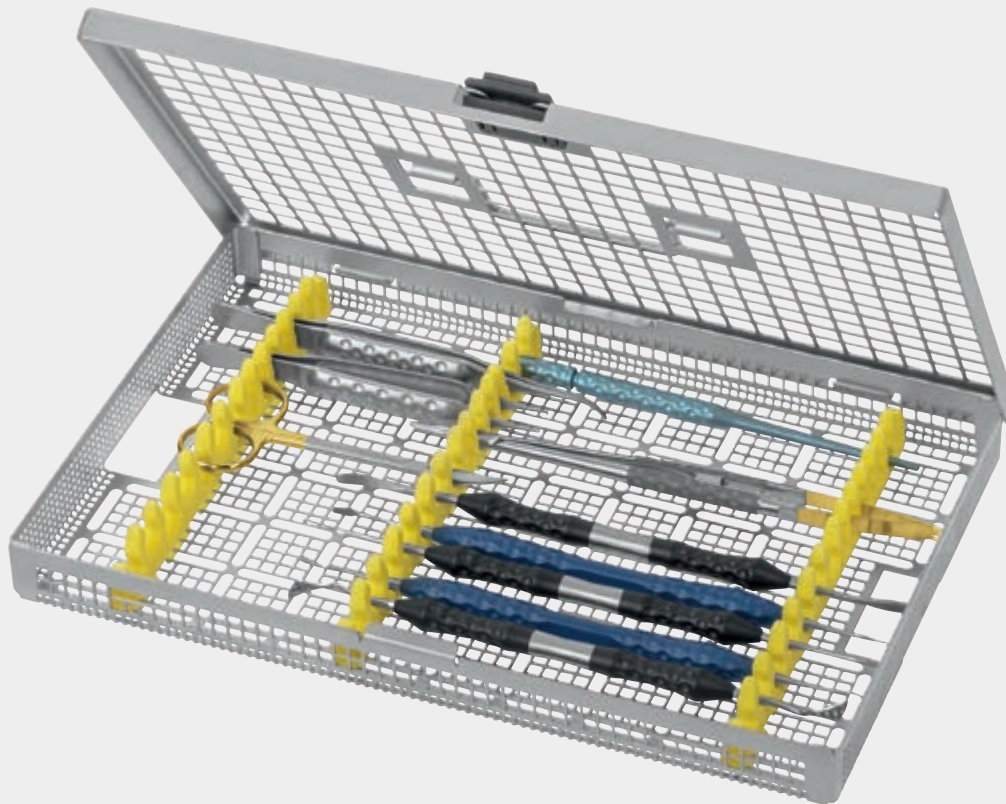
**DX705**

AESULAP® Basic Kit for dental implantology, consisting of:

- BB515 1 PAK = Package of 100 scalpel blades, sterile, Fig. 15
- BB063R 1 Scalpel handle, No. 3
- DX094 1 Ergoperio Mouth mirror holder
- DA036R 1 PAK = Package of 12 Rhodium-Mouth mirror, Ø 22 mm
- DX051R 1 ERGOPLANT Tissue forceps
- DX052R 1 ERGOPLANT Dissecting forceps
- DX201R 1 ERGOPLANT Raspatory, BUSER
- DX200R 1 ERGOPLANT Raspatory, PRICHARD
- DX220R 1 Ergoperio Bone curette, LUCAS
- BM013R 1 DUROGRIP® Nadelhalter, BABY-CRILE-WOOD
- DO219R 1 Gum scissors, LOCKLIN
- FO409R 1 Bone rongeur, LUER-FRIEDMANN
- JG381R 1 Tray (273 x 176 x 30 mm),  
suitable for dental container JN092

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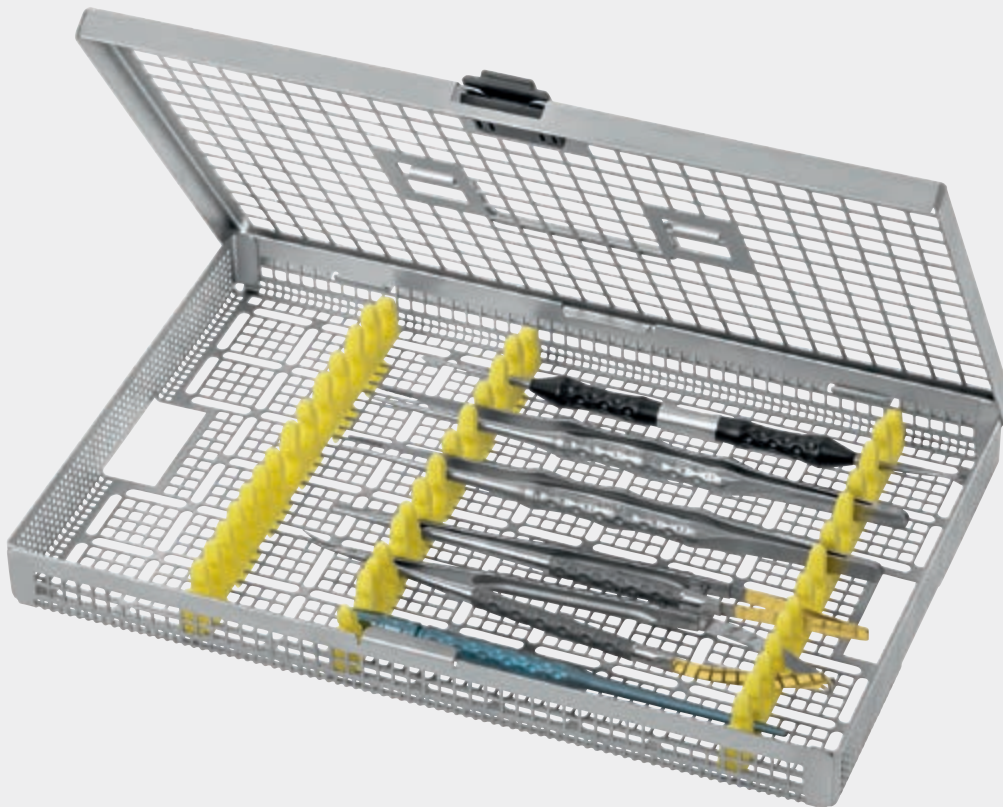
**DX707**

AESCU LAP® Soft Tissue Kit for dental implantology, consisting of:

- BB045T 1 Titanium scalpel handle
- BB367R 1 PAK = Package of 10 micro scalpel blades, sterile, Fig. 367
- BB063R 1 Scalpel handle, No. 3
- BB515 1 PAK = Package of 100 scalpel blades, sterile, Fig. 15
- DX050R 1 ERGOPLANT Tissue forceps
- BM003R 1 DUROGRIP® Needle holder
- DX450R 1 ERGOPLANT Gingivectomy knife, KIRKLAND
- DB025R 1 Ergoperio Raspatory FREER
- DX405R 1 ERGOPLANT Atraumatic papilla elevator, PHW
- DX401R 1 ERGOPLANT Spatula, sharp, STRICKER
- DX402R 1 ERGOPLANT Spatula, HEIDEMANN
- BC211W 1 Delicate scissors, curved
- DX055R 1 ERGOPLANT Soft tissue forceps
- JG381R 1 Tray (273 x 176 x 30 mm), suitable for dental container JN092

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**DX720**

Micro Soft Tissue Set, consisting of:

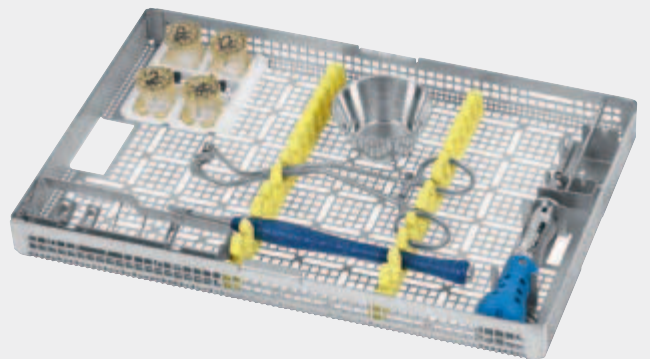
- BB045T Titanium-microblade holder, 145 mm
- DX301R Micro-Raspatory, 180 mm
- DX070R Micro atraumata forceps, 180 mm
- FD252R Micro ligature scissors, curved, one blade serrated, 180 mm
- DX060R Micro dissecting forceps, 180 mm
- BM003R DUROGRIP® Needle holder, 180 mm
- JG381R Tray, 274 x 172 x 30 mm

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For advanced alveolar crest atrophies less than three millimeters wide it is preferable to pursue a two-stage approach, in which, an autologous bone graft from appropriate autologous donor sites should be transplanted to the deficient alveolar crest atrophy region prior to the actual implantation. This is exactly what the AESCULAP® Bone Fixation Set was developed for.

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Screw magazine with 8 screws, self-taping, cross recess, Ø 1.5 mm

### Standard Titanium Screws 1.5 mm

#### High stability

- No weakening of the material due holding pins in the screw
- Titanium-aluminium alloy Ti-6Al-4V ensures high stability of the screws

#### Small & flat screw head

- The screw head is easy to countersink in the bone transplant

#### Micro thread

- Allows a good fixation even in very thin residual bone

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**Micro screws 1.2 mm**

Micro screw head = Standard screw head

- Reduces the risk of screw fracturing

**High stability**

- No weakening of the material due holding pins in the screw
- Titanium-aluminium alloy allows high stability of the screws

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Screw magazine with 8 screws, self-taping, cross recess, Ø 1.2 mm

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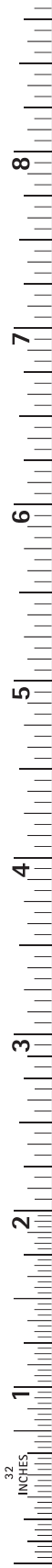
**Screw magazine**

**Mechanical cleanable**

- The new open magazine storage system enables mechanical cleaning of the implants

**Simple documentation**

- The batch numbers are printed on the screw magazines

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## Screwdriver

### Ergonomic

- Reduces the force required for tightening and loosening the screw

### Soft tissue sleeve

- The screw safety sleeve reduces the danger of soft tissue injuries

### Screw fall protection

- The screw safety sleeve serves as a screw fall protection

### Clean

- The open design enables efficient cleaning

### Friction

- Enables a strong retention for the screw

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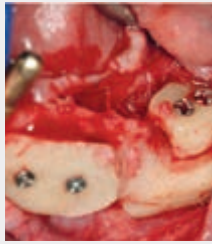
### Bone fixation forceps:

- **Universal**

Universal application in the upper and lower jaw.

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- **Adaptable**

The working tips adapt to the respective anatomical situation.

- **Delicate**

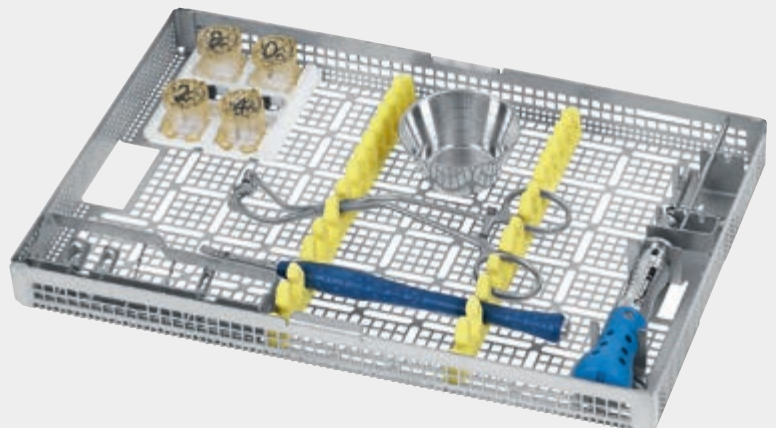
The delicate working tips enable drilling and screwing even in smaller bone blocks.

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Ø 1.5 mm

Länge / length

DX646T	6 mm	-
DX648T	8 mm	DX608T
DX650T	10 mm	DX610T
-	12 mm	DX612T
-	14 mm	DX614T

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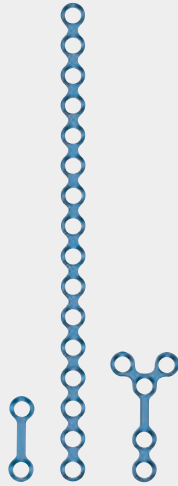
Screw magazine with 8 screws,  
self-taping, cross recess

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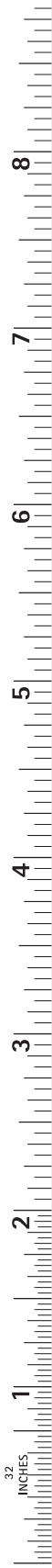
Bone graft preparation forceps

- enables to fix the bone graft for  
recontouring
- flexible and thin branches enables  
the adaption to the size of the  
bone graft

**DX629R** Ø 1.0 mm, 31 / 12 mm  
**DX630R** Ø 1.25 mm, 31 / 16 mm  
**DX631R** Ø 1.5 mm, 27 / 12 mm  
 Spiralbohrer mit Dentalschaft  
 Twist drill with dental shaft

**DX605R**  
 Schraubendrehergriff  
 Screw driver handle

**DX606R**  
 Schraubendreherklinge  
 Screw driver blade

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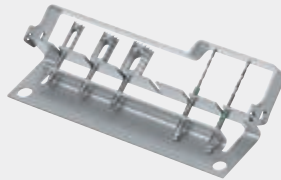
Rack for screw magazines  
(Rack without contents)

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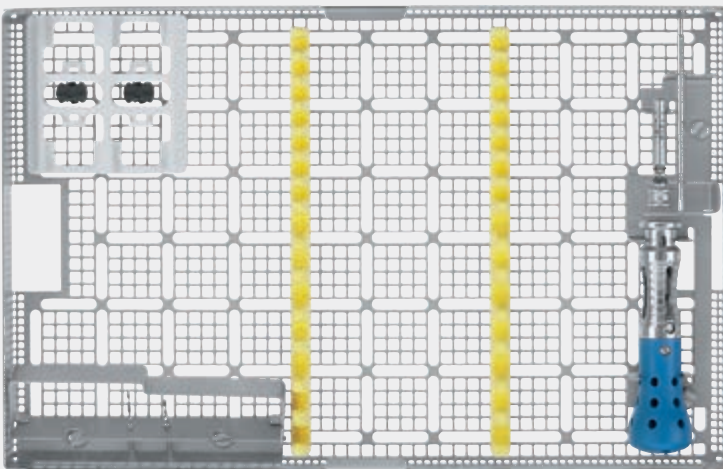
Rack for twist drills  
and tools  
(Rack without contents)

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**DX715**

Bone Fixation Set including tray with the  
following contents:

- JG403R Dental tray
- DX620 Rack for screw magazines
- DX621R Rack for twist drills
- DX630R Twist drill, Ø 1.25 mm
- DX631R Twist drill, Ø 1.5 mm
- DX715200 Rack for screw driver
- DX605R Screw driver
- DX606R Screw driver blade
- JG391 Silicone instrument rack (2x)

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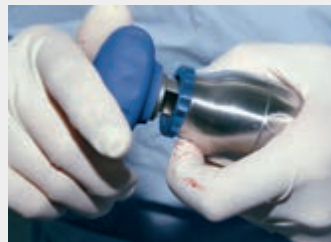
Permanent filter for sterile container,  
Order unit: PAK = Pack with 2 pieces

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Autogenous bone in augmentation has been established as the „golden standard“. The region where to gain the bone (Bone  
Harvesting) and the required consistence for augmentation can  
vary, depending on the case. Here the ERGOPLANT Micro Bone  
Mill shows its flexibility whereat a high yield of bone-grist is  
always ensured.

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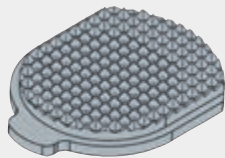
- ERGOPLANT Micro bone mill
- Small and handy
  - Interchangeable cutting disks
  - Cutting disks available for finer or coarser bone particles

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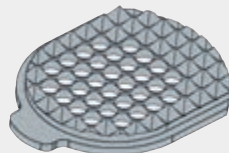
Cleaning fork for punching out  
valuable bone material of all cutting  
disk holes at once in a row.

**DX810R**  
ERGOPLANT  
Reinigungsinstrument fein  
für DX802R  
ERGOPLANT  
Bone mill cleaner fine  
for DX802R

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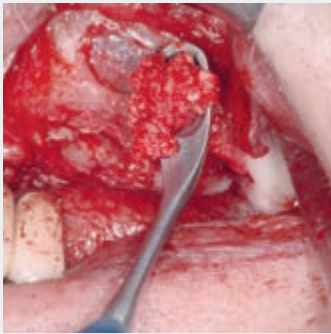
**DX800**

AESULAP® Micro Bone Mill Kit,  
consisting of:

- DX801R 1 ERGOPLANT Micro bone mill  
(incl. DX803R cutting disk, coarse)
- DX802R 1 ERGOPLANT Cutting disk, fine
- DX810R 1 ERGOPLANT Bone mill-cleaner, fine
- DX811R 1 ERGOPLANT Bone mill-cleaner, coarse
- JF284R 1 Tray (274 x 172 x 60 mm)  
with storage aid for micro bone mill,  
suitable for dental container JN095
- TE888 1 Packing stencil (only available for DX800)

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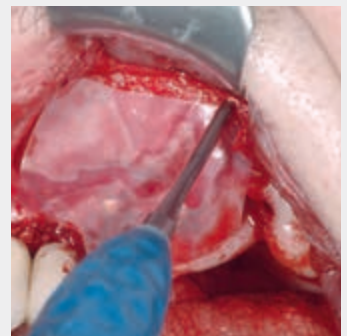
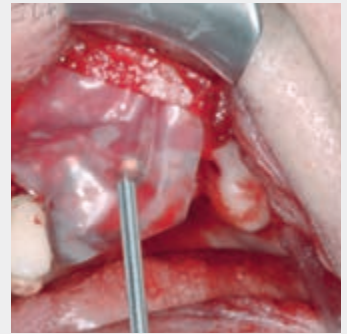
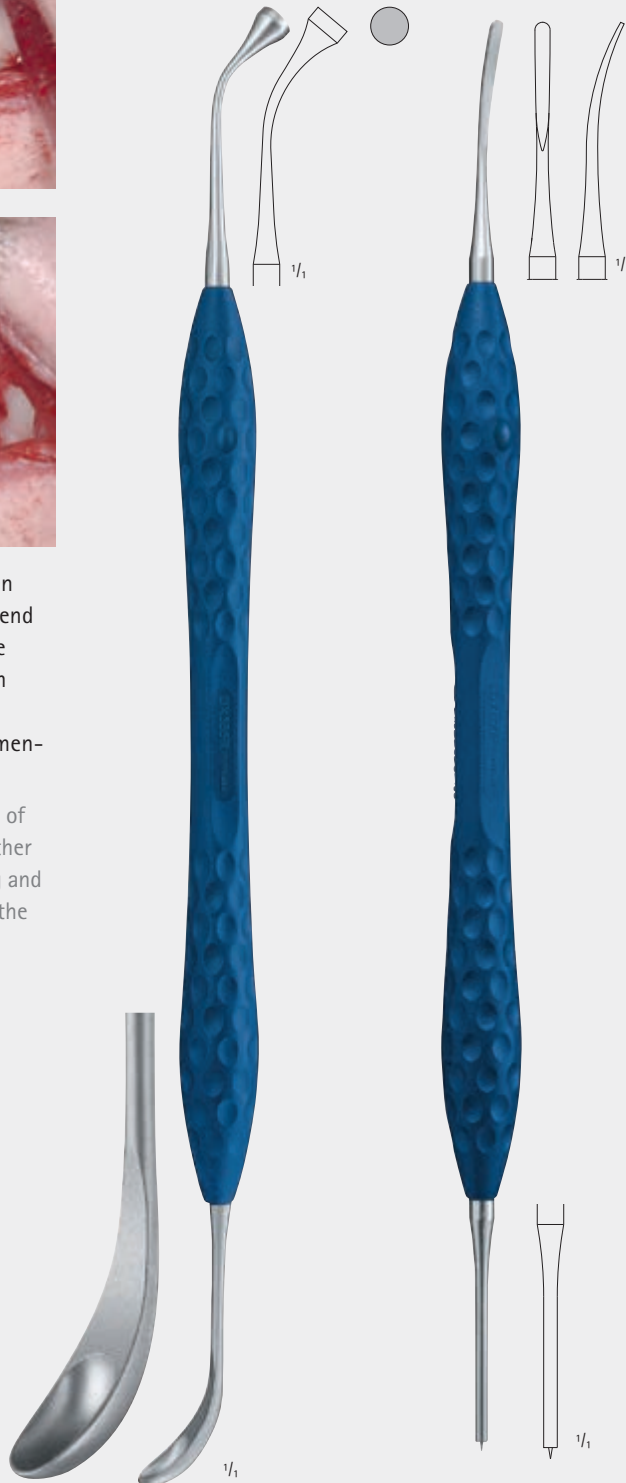
Enables carrying and placing of enough bone material. The other working tip serves for pushing and plugging bone material into the augmentation area.

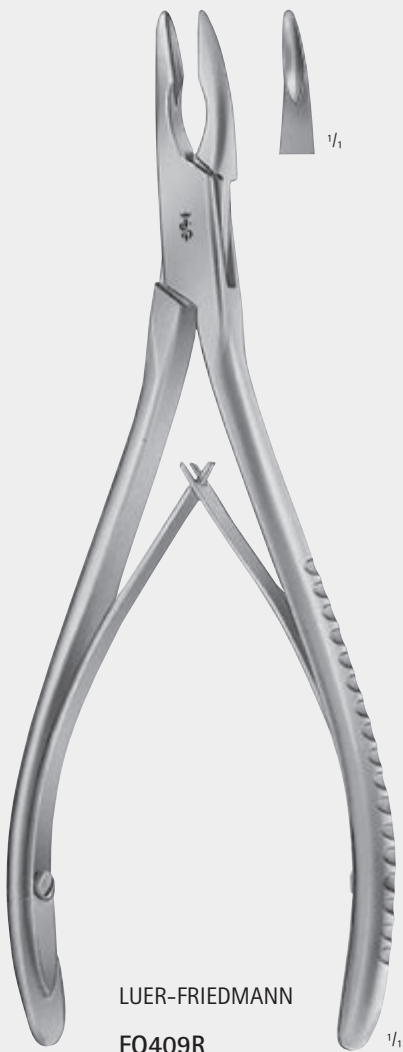
**STRICKER**  
**DX555R**  
ERGOPLANT  
Aufbringinstrument für Knochenmaterial  
ERGOPLANT  
Applicator for bone material  
195 mm, 7 3/4"

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The small surface of the pointed tip avoids the sticking of the humid membrane. The small spatula on the other end, helps to take out bone material of, for instance, the bone files DX252R – DX253R, as shown on page 300.

**STRICKER**  
**DX600R**  
ERGOPLANT  
Membran Platzierungsinstrument  
ERGOPLANT  
Membrane placing instrument  
190 mm, 7 1/2"

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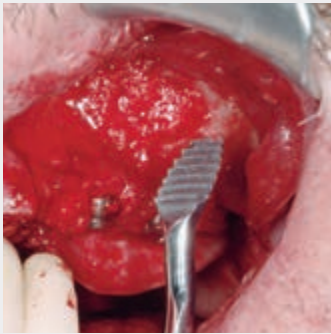
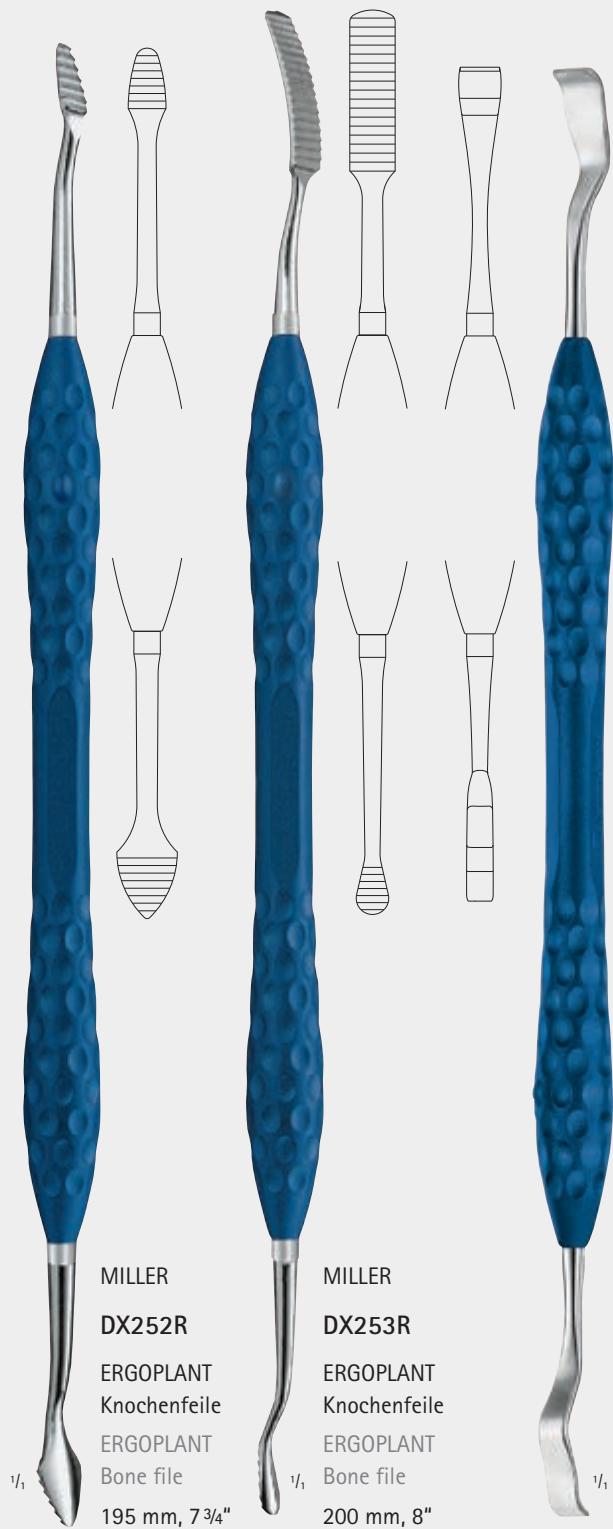
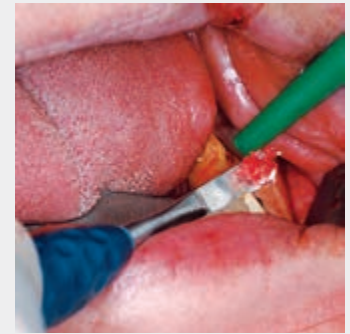
If smaller quantity of bone material is required, with the bone files DX252R – DX253R it can be harvested directly from the jawbone, without additional trauma for the patient. The quantity is often sufficient, for augmentation around the implant-neck.

MILLER  
**DX252R**  
ERGOPLANT  
Knochenfeile  
ERGOPLANT  
Bone file  
195 mm, 7 3/4"

MILLER  
**DX253R**  
ERGOPLANT  
Knochenfeile  
ERGOPLANT  
Bone file  
200 mm, 8"

BUSER  
**DX254R**  
ERGOPLANT  
Knochenschaber, 3 / 5 mm  
ERGOPLANT  
Bone raspatory, 3 / 5 mm  
190 mm, 7 1/2"

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With this raspatory fine bone particles can be harvested by simply scratching over the jawbone.

ERGOPLANT – KNOCHENGEWINNUNG / BONE SPLIT  
 ERGOPLANT – BONE HARVESTING / BONE SPLIT

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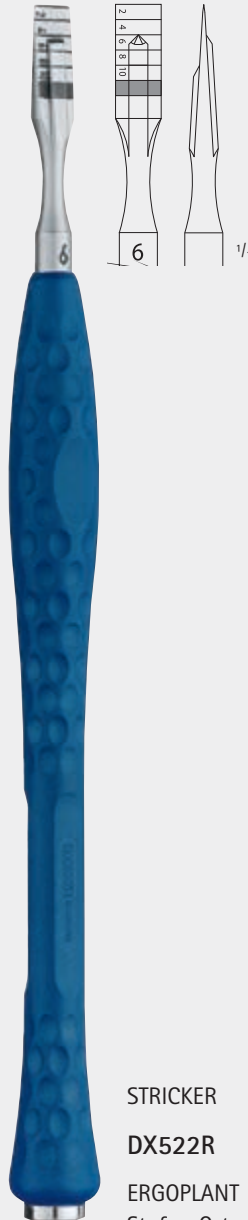
ERGOPLANT Mallet 135 g with interchangeable plastic disks

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The Step-Osteotome was specially developed for reliable bone-splitting with simultaneous stepwise widening. Due to the specially designed working tips, the maxillary crest will be first expanded towards palatinal and only afterwards towards vestibular. In the same time spongy bone in the center will be condensed. After removing the Step-Osteotome, a pre-selection point for the implant bur can be clearly seen.

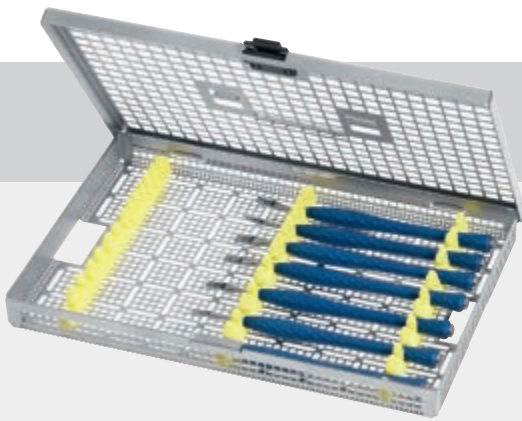
**DX510R**  
ERGOPLANT  
Osteotom  
gerade, 3 mm  
ERGOPLANT  
Osteotome  
straight, 3 mm  
160 mm, 6 1/3"

**DX511R**  
ERGOPLANT  
Osteotom  
gerade, 4 mm  
ERGOPLANT  
Osteotome  
straight, 4 mm  
160 mm, 6 1/3"

**DX512R**  
ERGOPLANT  
Osteotom  
gerade, 6 mm  
ERGOPLANT  
Osteotome  
straight, 6 mm  
160 mm, 6 1/3"

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**DX708**

AESCU LAP® Bone Split Kit for dental implantology,  
consisting of:

- DX510R 1 ERGOPLANT Osteotome, straight, 3 mm
- DX511R 1 ERGOPLANT Osteotome, straight, 4 mm
- DX512R 1 ERGOPLANT Osteotome, straight, 6 mm
- DX513R 1 ERGOPLANT Osteotome, straight, 8 mm
- DX517R 1 ERGOPLANT Osteotome, curved, 4 mm
- DX518R 1 ERGOPLANT Osteotome, curved, 6 mm
- DX522R 1 ERGOPLANT Step-Osteotome, 6 mm
- JG381R 1 Tray (273 x 176 x 30 mm),  
suitable for dental container JN092

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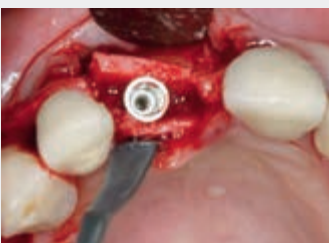
The maxillary crest is splitted transversal by the Step-Osteotome.

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The bone split is kept apart by the retractor.

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The implant can be positioned by using the existing bone.

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After splitting the maxillary crest the bone split is closing within a short period of time. This leads to an unwished and unnecessary bone loss when drilling afterwards. This causes the risk of an implant that is not exactly positioned and additional augmentative treatments could be necessary. The bone split is kept apart by the retractor. Thus, the existing bone can be, used better to position the implant.

STRICKER  
**DX505R**  
ERGOPLANT  
Bone Split Retractor-  
Applizierinstrument  
ERGOPLANT  
Applicator for  
bone split retractors

**DX506T**  
ERGOPLANT  
Bone Split Retraktor  
aus Titan Legierung,  
3 x 5 mm  
ERGOPLANT  
Titanium alloy  
bone split retractor,  
3 x 5 mm

**DX507T**  
ERGOPLANT  
Bone Split Retraktor  
aus Titan Legierung,  
4 x 6 mm  
ERGOPLANT  
Titanium alloy  
bone split retractor,  
4 x 6 mm

**DX509R**  
Lagerung für 4 Bone Split Retraktoren,  
zur Montage in AESCULAP® Dental Sieb-  
trays, inklusive 2 Befestigungsstiften  
Rack for 4 bone split retractors,  
can be easily fastened in AESCULAP®  
dental trays, incl. 2 fixation pins

ERGOPLANT – BONE CONDENSING / BONE SPLIT  
 ERGOPLANT – BONE CONDENSING / BONE SPLIT

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
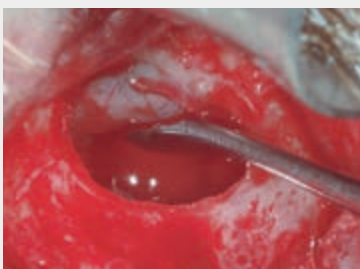
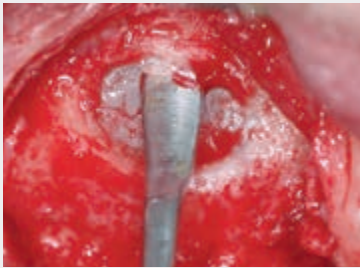
**DX709**

AESCULAP® Bone Condenser Kit for dental implantology, consisting of:

- DX530R 1 ERGOPLANT Bone condenser, straight, 1.5 – 2.2 mm
- DX531R 1 ERGOPLANT Bone condenser, straight, 2.2 – 2.8 mm
- DX532R 1 ERGOPLANT Bone condenser, straight, 2.8 – 3.5 mm
- DX533R 1 ERGOPLANT Bone condenser, straight, 3.5 – 4.2 mm
- DX534R 1 ERGOPLANT Bone condenser, straight, 4.2 – 4.8 mm
- DX535R 1 ERGOPLANT Bone condenser, bayonet, 1.5 – 2.2 mm
- DX536R 1 ERGOPLANT Bone condenser, bayonet, 2.2 – 2.8 mm
- DX537R 1 ERGOPLANT Bone condenser, bayonet, 2.8 – 3.5 mm
- DX538R 1 ERGOPLANT Bone condenser, bayonet, 3.5 – 4.2 mm
- DX539R 1 ERGOPLANT Bone condenser, bayonet, 4.2 – 4.8 mm
- JG381R 1 Tray (273 x 176 x 30 mm),  
suitable for dental container JN092

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Separating the maxillary sinus membrane carefully from the bony base. Finally the sinus window will be pushed with the other working tip gently into the hollow maxillary cave.

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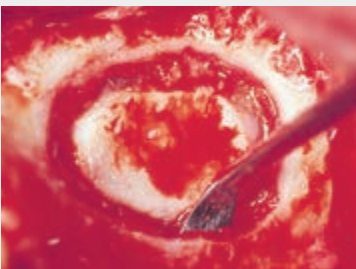
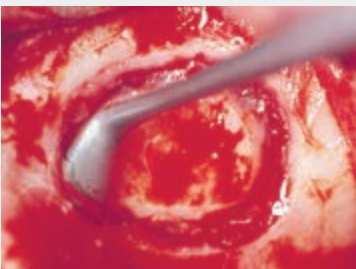
The Schneiderian membrane of the sinus can be detached with elevator D0571R by 1 – 2 mm along the total cortical bone.

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Elevator D0573R is particularly useful in the lower part for detaching the membrane further. The same goes for the caudal area, especially when the membrane is 4 – 5 mm above the base of the maxillary sinus and the maxillary sinus is built up with granular material.

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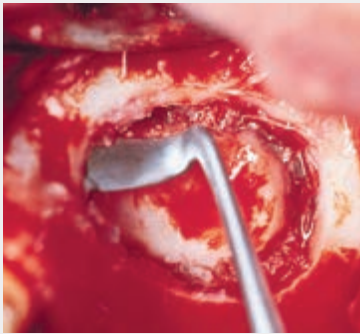
Elevator DO575R makes it possible to lift the membrane further in mesial region. One working tip is specially designed for the right maxillary sinus and the other for the left maxillary sinus.

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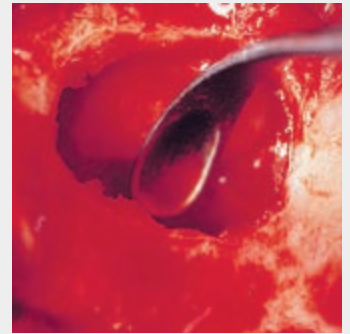
With elevator DO574R, the sinus membrane is further elevated in the upper and posterior part of the maxillary sinus wall.

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With the aid of elevator DO572R, the sinus membrane is further detached over a broad area in the inner region, especially along the roof of the palate. Condenser DX555R permits positioning of the filling material.

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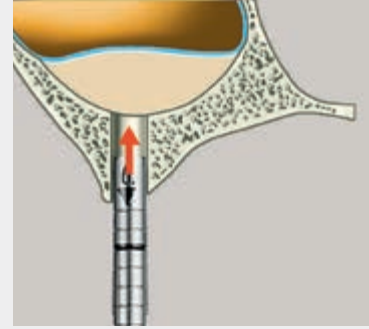
This illustration shows the technique of the percrestale sinus elevation.

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**DX710**

AESCULAP® Sinutome Kit for dental implantology,  
consisting of:

- DX570R 1 ERGOPLANT Sinutome, straight, Ø 2.2 mm
- DX571R 1 ERGOPLANT Sinutome, straight, Ø 2.8 mm
- DX572R 1 ERGOPLANT Sinutome, straight, Ø 3.5 mm
- DX573R 1 ERGOPLANT Sinutome, straight, Ø 4.2 mm
- DX574R 1 ERGOPLANT Sinutome, straight, Ø 4.8 mm
- DX575R 1 ERGOPLANT Sinutome, bayonet, Ø 2.2 mm
- DX576R 1 ERGOPLANT Sinutome, bayonet, Ø 2.8 mm
- DX577R 1 ERGOPLANT Sinutome, bayonet, Ø 3.5 mm
- DX578R 1 ERGOPLANT Sinutome, bayonet, Ø 4.2 mm
- DX579R 1 ERGOPLANT Sinutome, bayonet, Ø 4.8 mm
- JG381R 1 Tray (273 x 176 x 30 mm),  
suitable for dental container JN092

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Recall Set consisting of  
DB180T, DB181T, DB182T  
and JG387R

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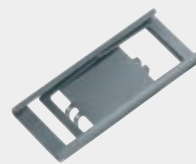
Holding device for  
identification labels

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Plastic identification labels,  
white, with 10-digit barcode  
for praxiDoc or DIOS software  
applications, labeling as specified

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These instruments were developed to hold back lips, cheeks, tongue and lingual mucous membrane. Due to the ergonomic handle design, a more relaxed working with minor physical strain is possible, even in long operations.

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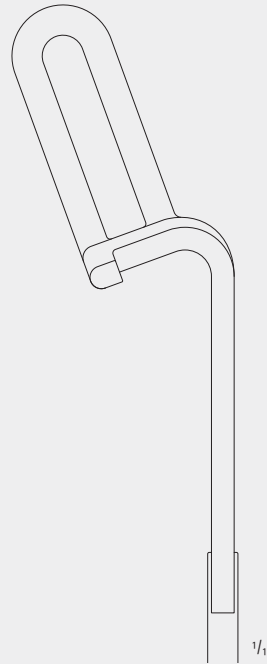
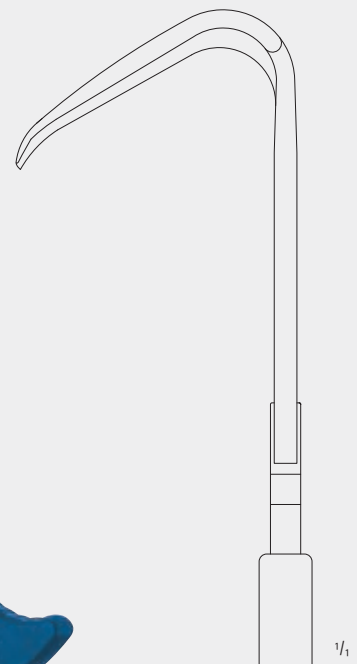
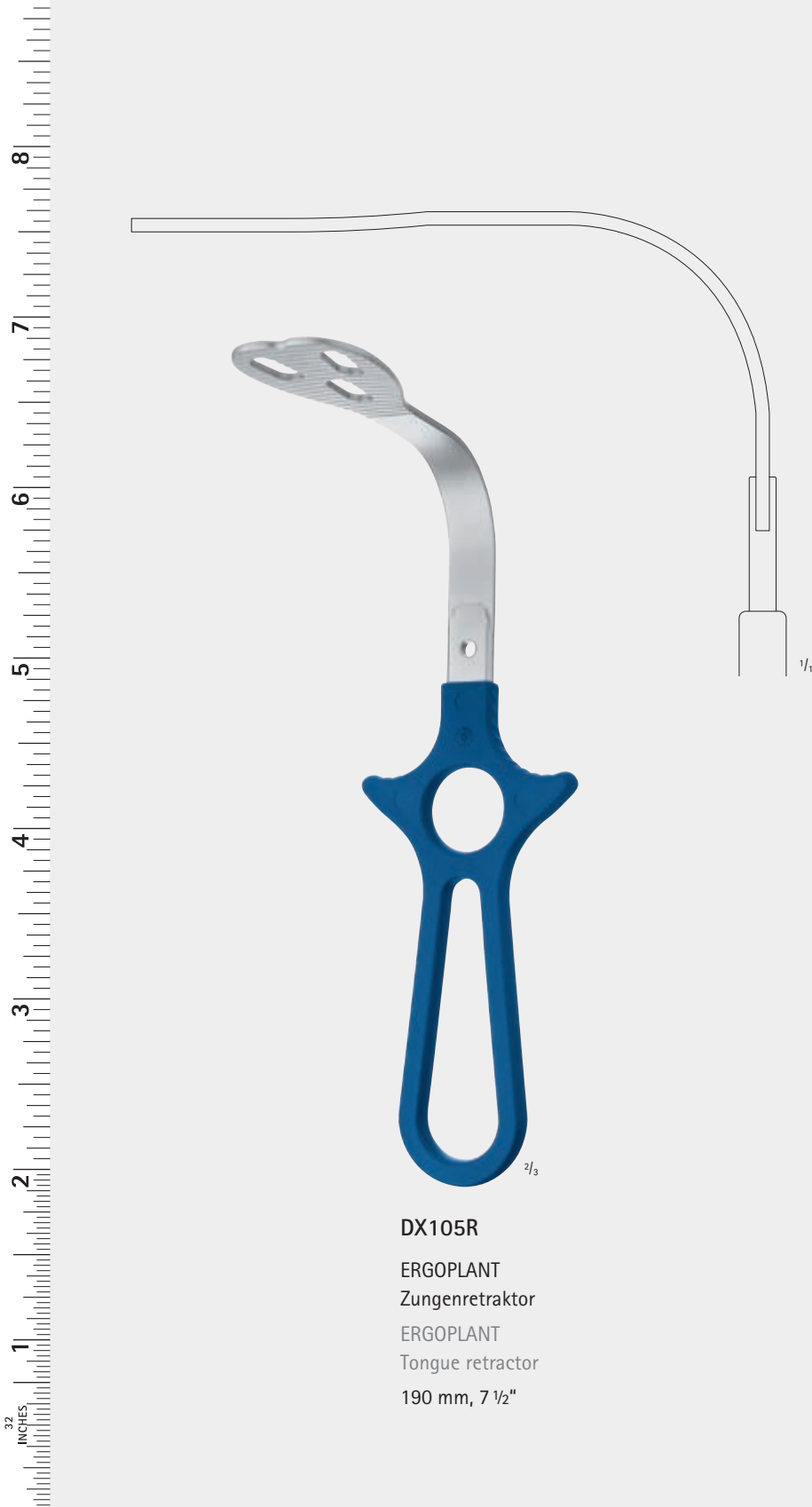
Sales unit: PAK = Package of 2 pieces  
 suitable for steam sterilization 134°C  
 (Autoclav)

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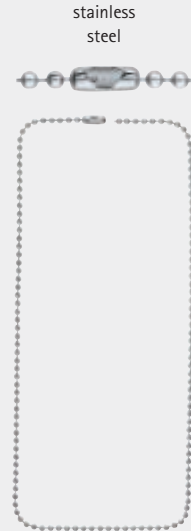
ERGOPLANT  
 Silicone mouth wedge  
 Size: S

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ERGOPLANT  
 Silicone mouth wedge with metal core  
 Size: M

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
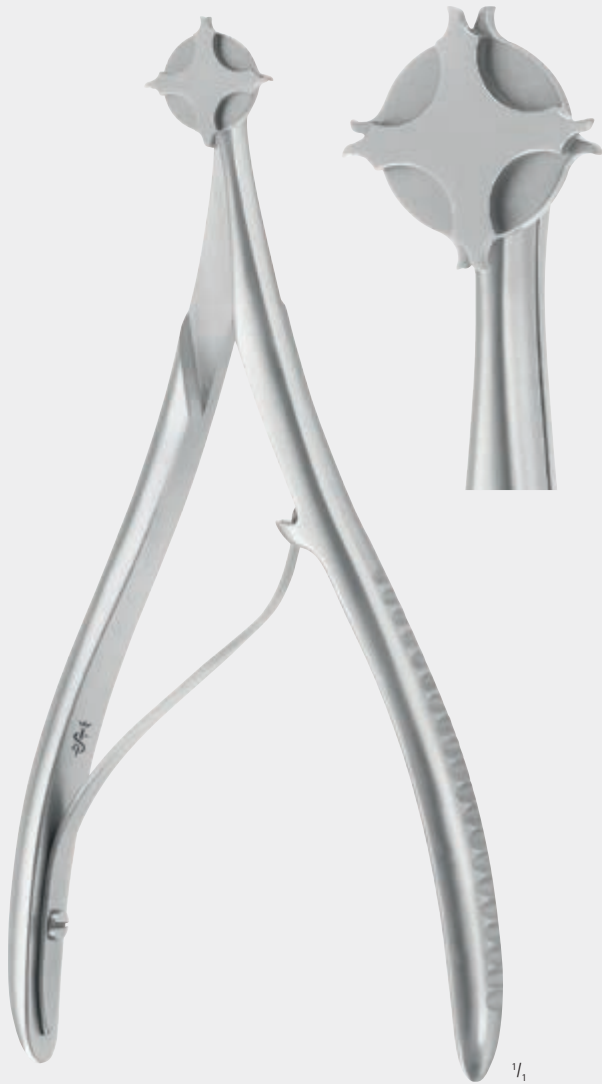
ERGOPLANT  
 Silicone mouth wedge with metal core  
 Size: L

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At the beginning the crown is slit parallel with a relatively thin instrument.

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To produce parallel slit edges which can be easily gripped with the pliers, a cylindrical instrument with Ø of approx 1.0 mm is guided perpendicular to the tooth center line.

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The teeth of the crown widening pliers are positioned in order to spread the crown.

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Finally the crown will be removed in the conventional way.

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### Our tip:

The slit walls which are produced must be perpendicular. Always cut through the bottom crown edge and, if possible, extend the slit up to the masticatory surface. In the case of bridges, move the slits approximately 1/3 towards the suspension element.

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JESCO®-Form impression trays, impression trays acc. WÖSTMANN perforated pattern. Using plastic or resilient impression compounds, these materials will penetrate the holes and give the necessary support for a correct impression. The impression trays are made of stainless steel.

The impression trays for lower jaw acc. WÖSTMANN in addition have the following characteristics and advantages:

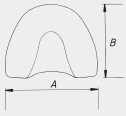
- Modified tray shield and changed tray curve
- Lengthened tray with simultaneous reduction of transversal expansion
- All teeth are seized, incl. the third molares
- Sufficient and uniform thickness of the impression compounds in the area of lingual undercut of the teeth

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\* The shapes of the impression trays are based on an analysis of naturally toothed patients of:

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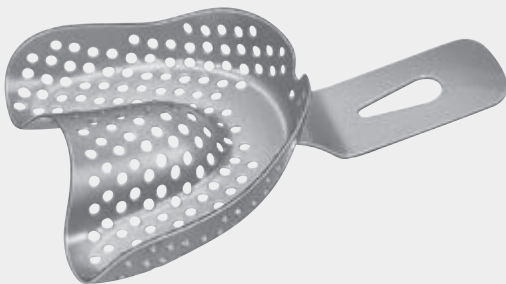
Using plastic or resilient impression compounds, these materials will penetrate the holes and give the necessary support for a correct impression.

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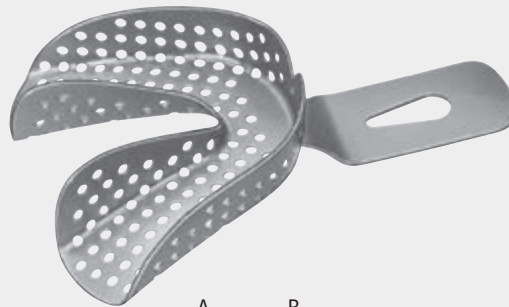
JESCO®-Form impression trays  
for toothed upper jaws

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Retraction thread packer, flexible

175 mm, 7"

**DF131R**

150 mm, 6"

**DF138R**

165 mm, 6 1/2"

**DF150R**

besonders flexibel  
especially flexible  
185 mm, 7 1/4"

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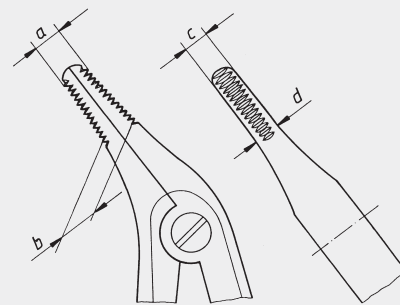
Telescope pliers grip the inner crown so that the outer one can be separated easily. To avoid damage to the crown in too stern a pressure, the set screw can be adjusted.

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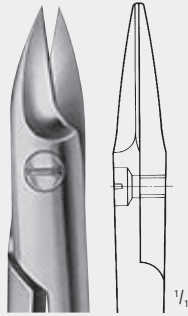
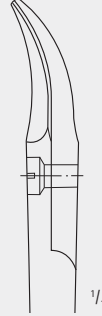
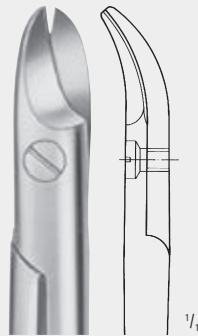
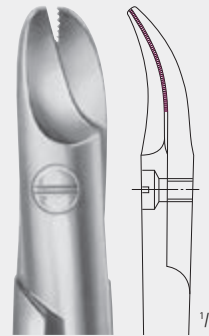
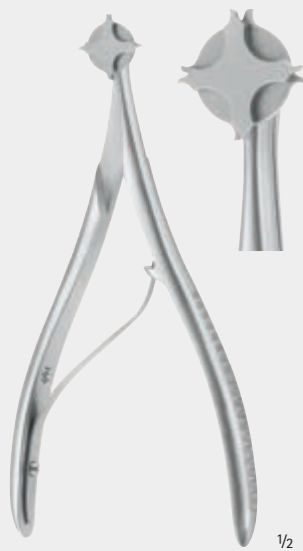
Crown widening pliers, for removing crowns and bridges

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Copring, temporary measure and crown removing pliers

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Crown and bridge remover, complete, with 3 points

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Light wire pliers

- for shaping and closing loops
- with 3 holding grooves
- max. wire size .028"

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Bird beak pliers

- with tungsten carbide inserts (can be replaced)
- max. wire size .028"

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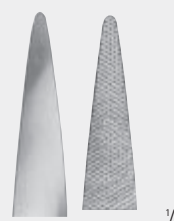
Flat pliers

- mouth parts serrated
- with short mouth parts, for better transmission of force

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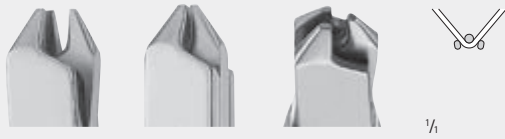
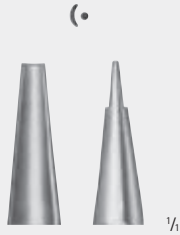
Wire bending pliers  
for bending loops

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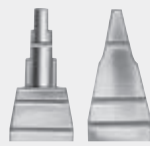
Triangular bending clamp pliers

- for fast and exact bending of arrow-head clasps
- max. wire size: 0.7 mm (.028")

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Loop bending pliers

- for loops of any dimension
- particularly suitable for labial bows
- max. wire size: 0.7 mm (.028") springy

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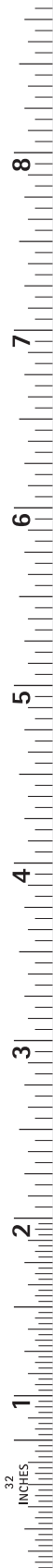
Wire scissors, 1 blade serrated  
for soft wire up to Ø 1.2 mm,  
for hard wire up to Ø 0.6 mm

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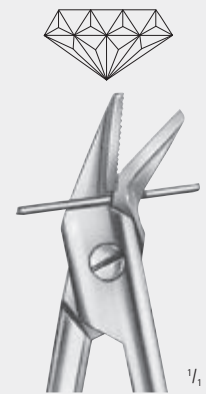
Wire scissors, 1 blade serrated  
for soft wire up to Ø 1.2 mm,  
for hard wire up to Ø 0.8 mm

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Wire cutter  
 for soft wire up to Ø 1.5 mm,  
 for hard wire up to Ø 1.0 mm

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Wire cutter  
 for soft wire up to 0.5 mm,  
 for hard wire up to 0.3 mm

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Wire cutter  
 for soft wire up to 0.6 mm,  
 for hard wire up to 0.4 mm

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Wire cutter  
for soft wire up to Ø 2.0 mm  
for hard wire up to Ø 1.5 mm

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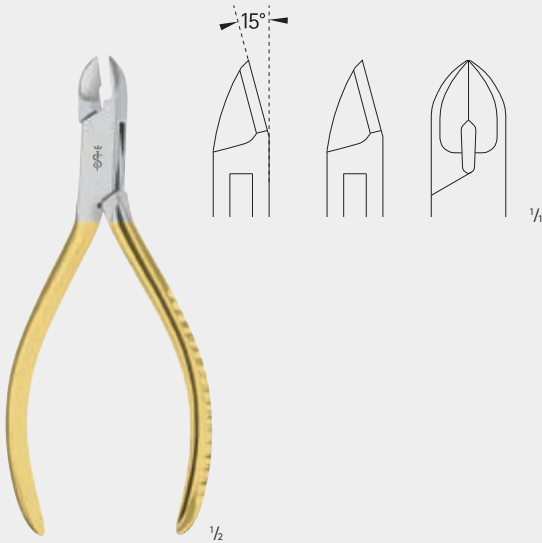
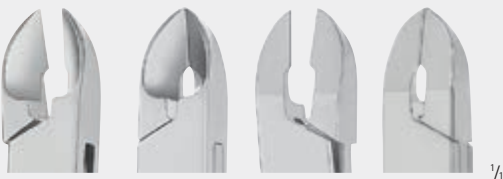
Wire cutter  
for soft wire up to Ø 3.0 mm  
for hard wire up to Ø 2.2 mm

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Hard wire cutter

- with tungsten carbide inserts  
(can be sharpened/replaced)
- for cutting all orthodontic wires
- wires can be cut without deformation
- max. wire size: .022 x .028"

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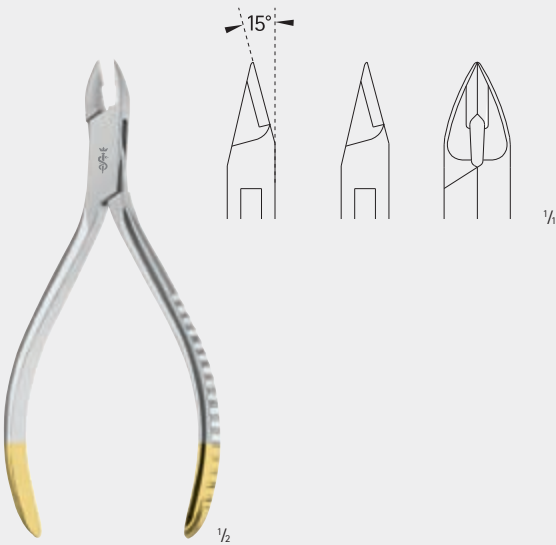
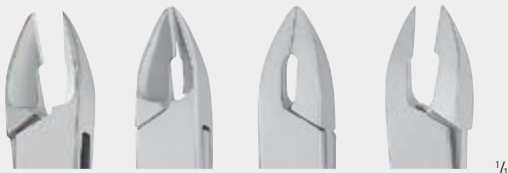
Distal end cutter

- with tungsten carbide inserts  
(can be sharpened/replaced)
- for intraoral cutting, with safety hold,  
from wire size .014" upwards
- max. wire size: .021 x .025"

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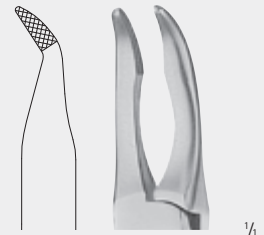
Ligature cutter

- with tungsten carbide inserts  
(can be sharpened/replaced)
- fine cutting jaws and their 15° angular offset enable cutting close to the bracket
- max. wire size: .014"

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Universal orthodontic pliers

- used as a retaining instrument and to insert archwires in the tube
- enables easy intraoral activation of archwires
- short jaws for better transmission of force

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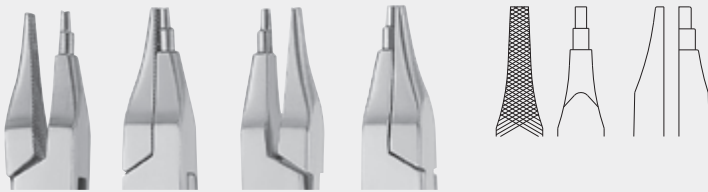
Torque pliers

- with tungsten carbide inserts (can be replaced)
- for exact, sharp-edged bending thanks to sturdy tungsten carbide inserts
- max. wire size: .028"

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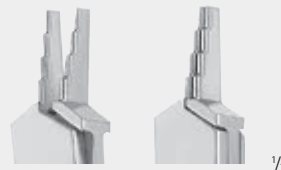
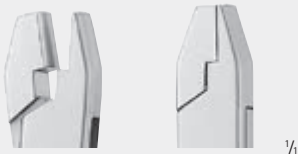
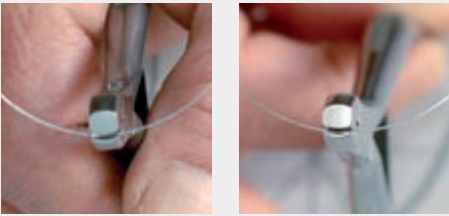
#### Loop forming pliers

- sturdy, rounded jaw part in size 2 and 2.5 mm (for the inner diameter of the loop)
- flat jaw part serrated for good fixation of the archwire
- sufficient room for the reliable and torque-free formation of loops
- max. wire size: .019 x .025"

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Arch bending pliers

Step loop bending pliers

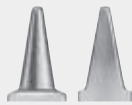
- for forming the desired curvature
- the curved design of the jaws allows the curvature of the wires to be adjusted in small steps
- the spring-loaded handle opens the pliers automatically, which facilitates the work
- suitable for all orthodontic wires

- Steps of 3, 4, 5 and 6 mm enable exact, even bending, e. g. for utility arches

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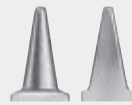
Band removing pliers  
To remove bands, the plastic tip is  
placed on the occlusal surface and  
the metal beak is placed under the  
tube or cleat, and the bands are  
removed in small steps with  
alternating forces buccally and  
lingually/palatally.

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Band remover

- with tungsten carbide insert (can be replaced)
- replaceable plastic tip
  - to plug in (Art.no. **DP744202**)
  - to screw (Art.no. **DP744205**)

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Combination clamp forceps

- straight
- mouth parts serrated
- for positioning brackets
- spindle on forceps end is used for adjusting the brackets

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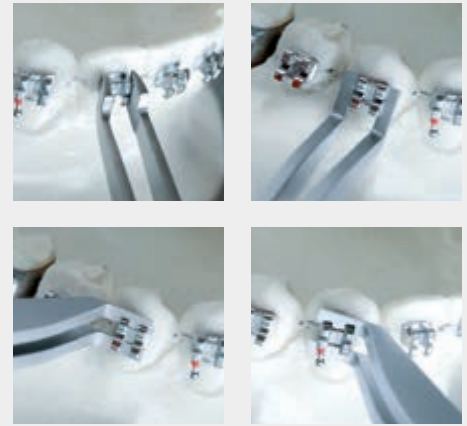
Combination clamp forceps

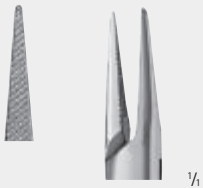
- angled
- mouth parts serrated
- particularly suitable for positioning brackets in the posterior teeth
- spindle on forceps end is used for adjusting the brackets

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Holds the clamped brackets automatically. The delicate forceps tips can be placed mesially/distally and also under the wings. The end of the forceps can be used to adjust the bracket position.

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Alastik holder (MATHIEU)

- with delicate, serrated mouth parts
- for applying rubber ligatures
- Spring enables single-handed use
- the single-step locking mechanism facilitates the work and makes it easier to remove the clamp from the patient

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Forceps, straight

- mouth parts serrated
- for applying rubber ligatures

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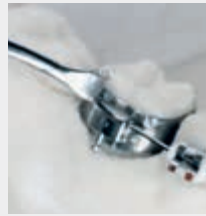
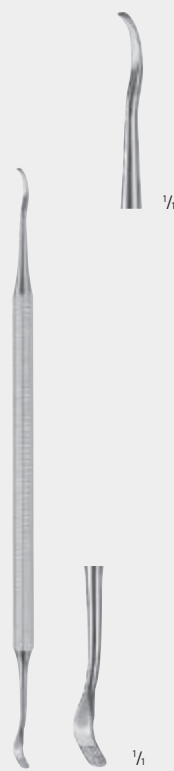
Scaler

- for removing adhesive and cement residues

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Scaler/Pusher

- for positioning and removing bands and for removing cement and adhesive residues

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Orthodontic probe

- short, strong tip
- among other things, suitable for removing ligatures and adjusting brackets

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Flat pliers

- mouth parts serrated
- with short mout parts, for better transmission of force

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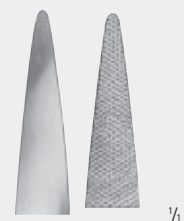
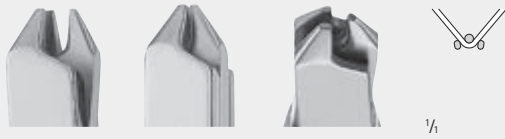
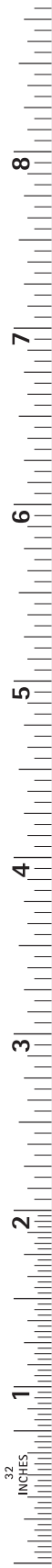
Three-finger pliers ■ For use with hard wires and spring hard wires in the application of orthodontic appliances, quad-helix appliances, Nance appliances and many other appliances

- max. wire size: 0.7 mm (.028")

**ADERER  
DP316R**  
125 mm, 5"  
Dreifingerzange

- Max. Drahtstärke: 0,9 mm (.036")

Three-finger pliers ■ max. wire size: 0.9 mm (.036")

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## SYSTEMATIC INSTRUMENT REPROCESSING PROCEDURE

The primary intention of hygienic measures in dental practice is to safeguard the health of patients and personnel. To ensure safety it is particularly important to observe the principles of hygiene when cleaning and sterilizing instruments. To maintain a safe, efficient and economical workplace, clear procedures for hygiene must be specified and followed. The AESCULAP® Sterile Container System is a proven and excellent aid in the standardized management of instruments throughout the dental practice workflow, from sterile processing through transport, storage and use.

## STERILGUTKREISLAUF – INSTRUMENTENAUFBEREITUNG STERILE GOODS CYCLE – INSTRUMENT REPROCESSING

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## FOUR SMART REASONS TO CHOOSE THE AESCULAP® STERILE CONTAINER SYSTEM

- **Prevention of infections**  
Of patients and staff through preservation of instrument sterility and closed transport of used instruments
- **Economy**  
Low total cost of ownership due to reusable packaging, minimal processing costs and long service life
- **Comfortable handling**  
Simple, quick and reliable
- **Transparency**  
Standardized, effective documentation of your sterile supply workflow

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### 4 | Kontrollieren, pflegen, verpacken Inspection, care and maintenance and packing

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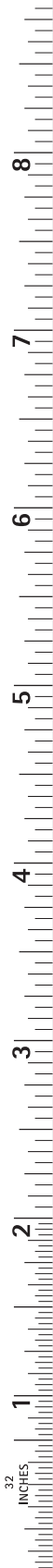
## INNOVATION FACTORY IN TUTTLINGEN – INNOVATION MEETS TRADITION

The AESCULAP® sterile container system has been manufactured at the Tuttlingen / Germany site for more than 45 years.

With a modern industrial architecture, at the Innovation Factory, which was opened in June 2015, AESCULAP® acts on the B. Braun corporate values: be innovative, increase efficiency and live sustainably.

The goal is an intelligent factory. This is characterized by adaptability, efficient use of resources and ergonomic design.

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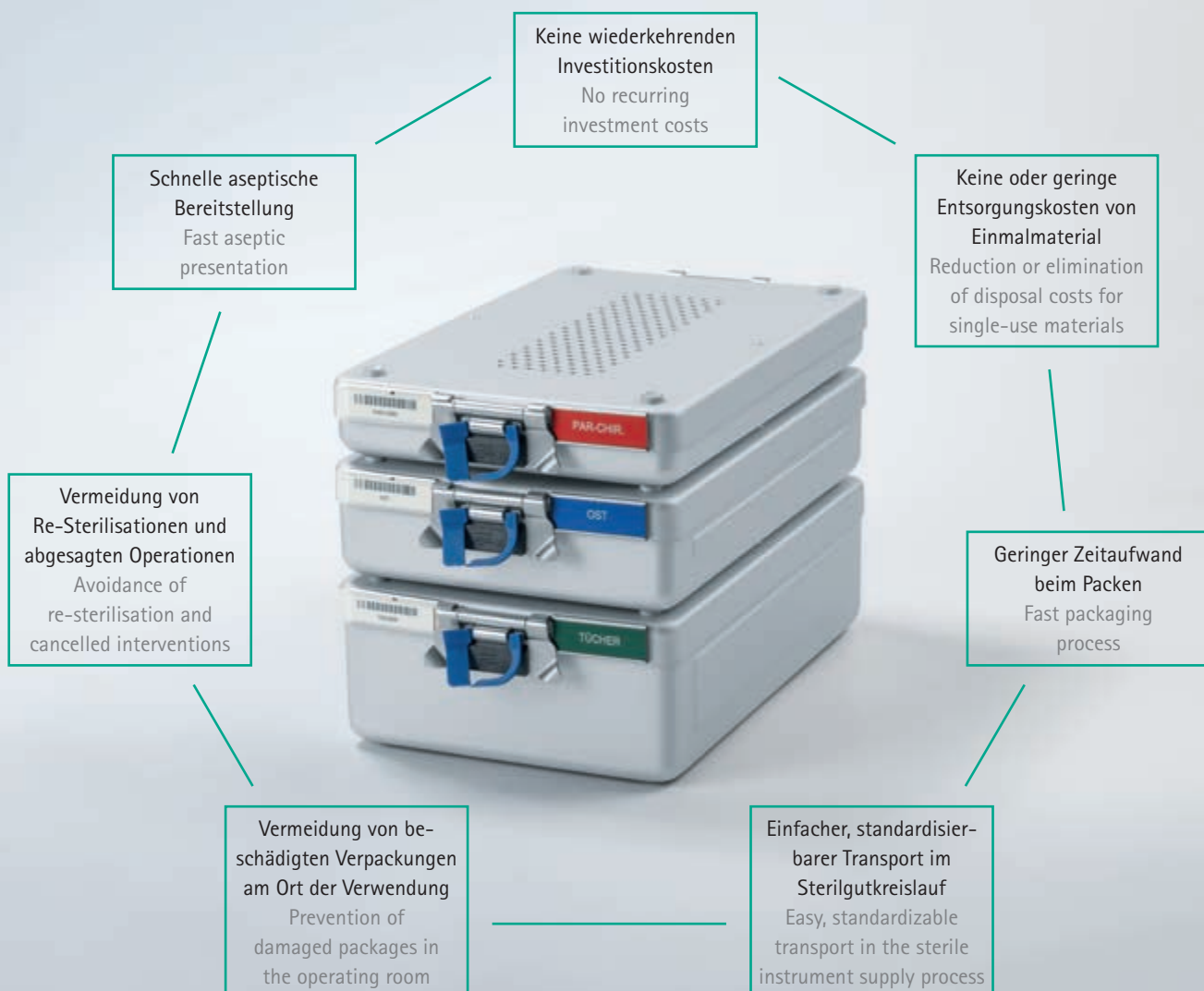
The solid, robust design and the manufacturing technology used by AESCULAP® enable the sterile containers, instrument baskets and all the other components to be manufactured with very precise dimensions and a high level of stability.

All the components of an AESCULAP® sterile container comply with the national and international requirements and standards (e. g. DIN EN ISO 11607-1).

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## QUALITY AT A GLANCE

The tension locks are also designed as handles for fast, contamination-free lid removal when sterile goods are presented for use.

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The lock plates can be fixed at any position and turned through 90° into the lid. Springs integrated into the lock plates guarantee that the sterile container always remains consistently closed.

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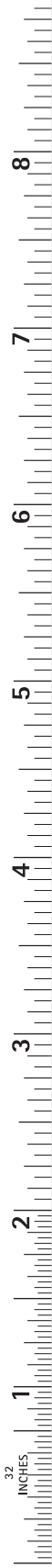
The containers can be stacked, reducing the amount of space required in the sterile goods cycle.

The seal prevents accidental opening of the container after sterilization.

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## AESCULAP® STERILE FILTERS

- The filter material conforms to the requirements of DIN EN ISO 11607
- Reliable: easy to check
- Advantageous: they facilitate good drying results within the container

The AESCULAP® PTFE permanent filter also contributes to efficiency and economy with the following features:

- Extremely resistant and durable
- Validated for 1,000 use cycles

Closure of the dental containers is ensured by the ring-shaped silicon seals around the inside of the lid. The filter holder has an extremely durable silicon seal which keeps the filter firmly in place:

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(1) Final report: Determination of the effectiveness of an Automatic Cleaning Process.  
19-February 2009.

## EFFECTIVE AND CLEAN

The wide perforation ensures optimal water jet permeation  
and avoids rinsing blind spots.

*„The microbiological and organic contamination is satisfactorily  
reduced (1)!“*

This is the result of extensive testing carried out by the  
independent laboratory Medical Device Testing.

PAR surgical instrument sets in AESCULAP® trays contaminated  
with test germs and organic matter were used for testing.

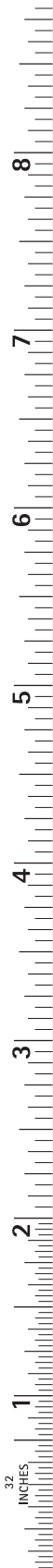
Consequently, the mechanical cleaning and thermal  
disinfection before sterilization were rated as being adequate  
in accordance with DIN EN ISO 17664.

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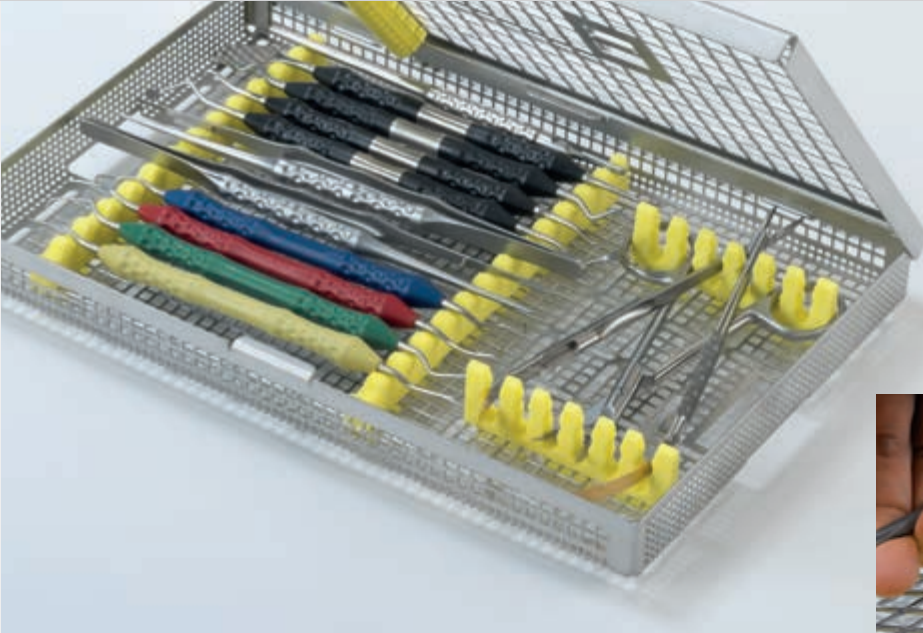
## ORGANIZATION

Your instruments are stored in the smallest possible space,  
safely fixed in their positions.

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## ECONOMICAL

Mechanical processing with the AESCULAP® tray system optimizes your workflow. The stainless steel material makes AESCULAP® trays a robust and durable asset. The simple and robust locking system can be easily opened and closed with one hand.

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## FLEXIBILITY

Position the silicone racks in the optimal arrangement for your instruments – in seconds, without any additional tools.

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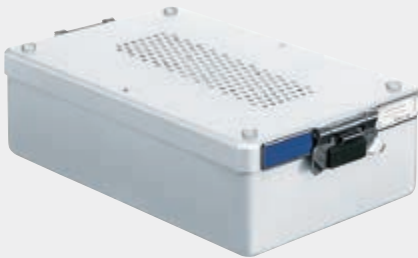
To take following trays:  
1 x JG381R or 1 x JG383R or 1 x JG389R  
JK286 container body and JN091 lid can also be ordered singly  
312 x 190 x 46 mm

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To take following trays:  
2 x JG380R or 1 x JG384R or 1 x JG386R  
JK288 container body and JN091 lid can also be ordered singly  
312 x 190 x 65 mm

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To take following trays:  
2 x JG381R or 1 x JG381R + 1 x JG384R or 1 x JG285R  
JK289 container body and JN091 lid can also be ordered singly  
312 x 190 x 92 mm

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To take all trays or 1 x JF282R basket with lid JF289R  
JK290 container body and JN091 lid can also be ordered singly  
312 x 190 x 130 mm

Verbrauchsmaterialien bitte separat bestellen. Please order accessories separately.

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AESCULAP® reusable filter for dental containers JN092-JN096 and JN292-JN296, service life 1,000 cleaning and sterilization cycles.

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AESCULAP® paper filter with process indicator\* for steam sterilization, for dental containers JN092-JN096 and JN292-JN296.

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Filter retainer universal for AESCULAP® reusable filter JK091 and AESCULAP® paper filter JF166.

Suitable for lids and bodies of dental containers JN092-JN096 with narrow perforation.

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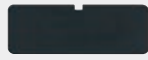
Note: Process aluminium identification labels with pH-neutral cleaning agent.

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Plastic identification labels, white, with 10-character barcode for software applications, labelling as specification

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Tray, contains two instrument racks (2 x JG394) for 6 instruments

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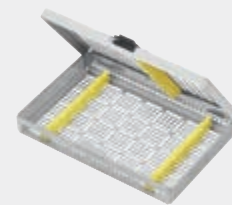
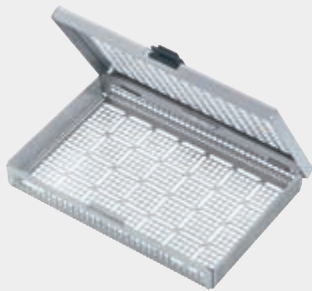
Tray, contains two instrument racks (JG400 + JG401) for 9 instruments

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Tray, contains three instrument racks (3x JG391), with lid handle, fitting into Dental Container JN092

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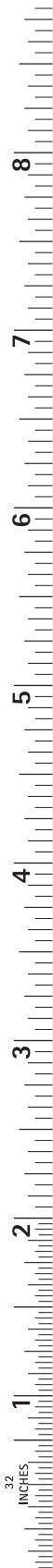
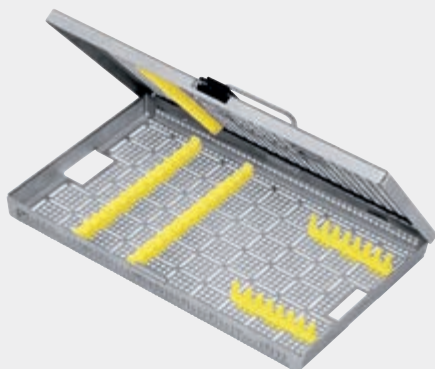
Tray, with perforation for retractor, contains three instrument racks (3x JG391), with lid handle, fitting into Dental Container JN092

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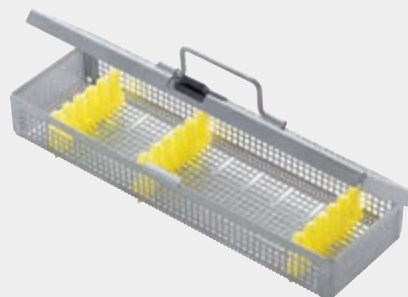
Tray, contains five instrument racks (2x JG394, 2x JG400, 1x JG401), with lid handle, fitting into Dental Container JN092

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Tray, contains three instrument racks (3x JG392), with lid handle, fitting into Dental Container JN094

273 x 84 x 41 mm (hoch/high)

**JG386R**

Siebtray, passend in Dental Container JN094

Tray, without inserts, with lid handle, fitting into Dental Container JN094

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Tray, contains three instrument racks (JG393), with lid handle, fitting into Dental Container JN094

273 x 176 x 41 mm (hoch/high)

**JG285R**

Siebtray, passend in Dental Container JN095

Tray with lid fitting into Dental Container JN095

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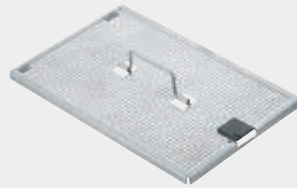
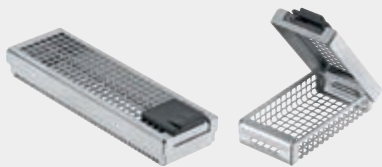
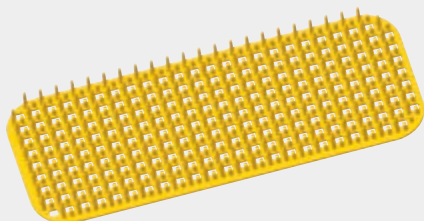
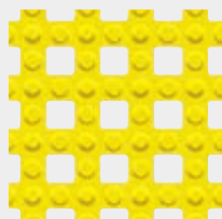
Non-slip storage of surgical instruments, efficient  
drying and careful storage during machine cleaning.

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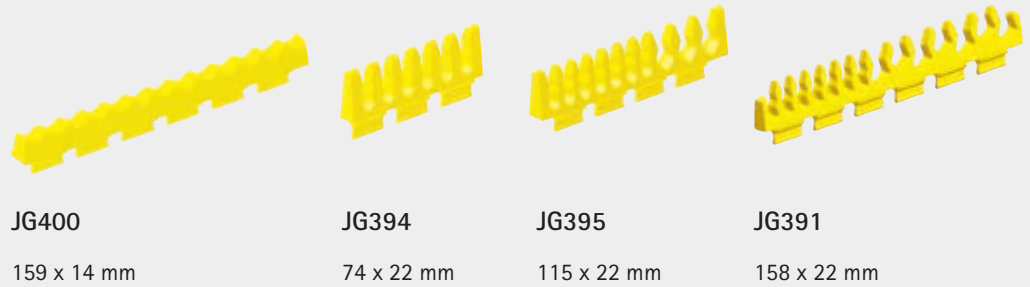
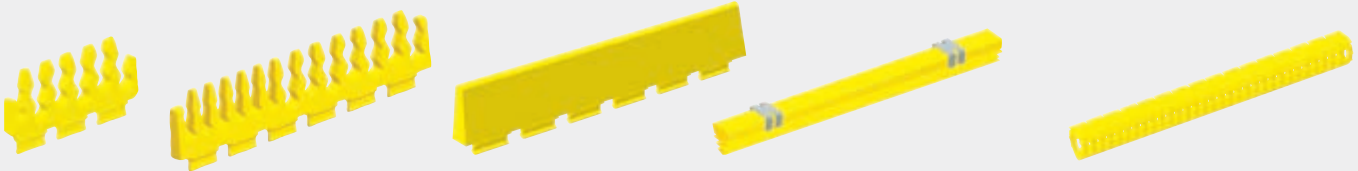
Press pad, for suppressing the instrument, to assemble to the lower side of the lid.

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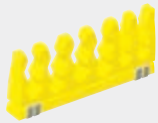
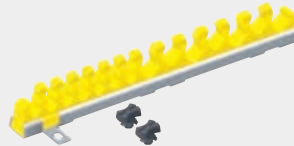
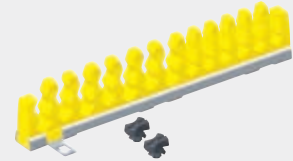
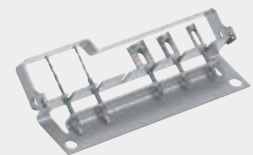
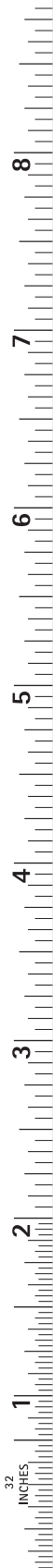
AESCULAP® oil spray, in aerosol can (ozone-friendly), for maintenance of instruments prior to sterilization. Physiologically inert acc. to DAB and § 31 LMBG (German Pharmacopoeia)

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AESCULAP® instrument oil, for box locks, ratchets etc. prior to sterilization. Physiologically inert acc. to DAB and § 31 LMBG (German Pharmacopoeia)

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Reprocessing of instruments in dental practice, brochure for download from <http://www.a-k-i.org>

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For further details see  
brochure No. **C40402**

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For further details see  
brochure No. **4004000**

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Seite

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## INDEXES

	Page		Page
<b>A</b>			
Abdominal and intestinal spatulas	121	Arkansas grinding stone	54, 273
Accessories, containers	374	Atraumatic micro forceps	101
Eloxal cleaner	377	Atraumatic forceps	159, 160
Filter retainers	375	Atraumatic micro forceps	52
Fixation pins	382	Atraumatic papilla elevators	269
Grinding stones oil	42, 54, 273	Awls	169
Indicator labels	377	for zygomatic arch	169
Instrument oil	383	Mandibular awls	169
Instrument racks	381	Maxillary awls	169
Marking pens	377	<b>B</b>	
Oil for grinding stones	42, 54, 273	BABY-CRILE-WOOD, Needle holders	276
Oil spray	383	BABY-ALLIS, Tissue grasping forceps	168
Paper filters	375	BABY-CRILE, Haemostatic forceps	167
Press pads	381	BABY-CRILE-WOOD, Needle holders	201
Process identification labels	376	BABY-METZENBAUM, Dissecting scissors	139, 153
Reusable filters	375	Surgical scissors	151
Seals	377	BABY-MOSQUITO (HARTMANN), Haemostatic forceps	166
Storage, non-slip	380	BACKHAUS, Towel clamps	165
Test rods	42	Ball burnisher/spatulas	71
ADERER, Three-finger pliers	342, 364	Band removing pliers	359
ADSON, Atraumatic forceps	160	Bandage and cloth scissors	156
Suction instruments	209	BANE-HARTMANN, Bone rongeurs	196
Surgical forceps	163	BARRY, Elevators	257
ADSON-BROWN, Surgical forceps	163	BARTH, Bone curettes	181
After-GRACEY, Curettes	34, 35	Basic kit	283
AINSWORTH, Rubber-dam punches	96	BAUMGARTNER, Needle holders	197
Alastik holders	361	BEEBE, Crown scissors	337
ALLIS, Tissue grasping forceps	168	BEERENDONK, Calipers	332
ANATOMICA, separating forceps	223	BEIN, Elevators	250
Tooth forceps	217-230	BERGMANN, Retractors	132
ANDERSEN, Round-nosed pliers	343	BEYER, Bone rongeurs	195, 196
ANDRESEN, Wire bending pliers	345	Bird beak pliers	340, 358
ANGLE, Bird beak pliers	340, 358	BLACK, Cutting instruments	93
Wire bending pliers	340, 358	Excavators	95
Wire cutters	349	Plastic-filling instruments	82
Applicator, for bone split retractors	306	BLAKESLEY-WILDE, Ethmoidal forceps	119
for bone material	298, 313	BLUMENTHAL, Bone rongeurs	195
for bone split retractors	306		
Arch bending pliers	357		

## ALPHABETICAL INDEX

	Page		Page
<b>BÖHM</b> , Telescope pliers	336	Bone split, step-osteotomes	301
Bone condenser	307, 308	Bone mill, cleaner	296
Kit	309	Cutting disks	296
Bone curettes	178-182, 270	Micro bone mill	296
Bone cutting forceps	196	Micro bone mill kit	297
Bone files	182, 300	<b>BOWDLER-HENRY</b> , Retractors	127
Bone fixation, bone graft preparation forceps	291	<b>BOWMAN</b> , Probes	207
Box for the twistdrills	293	Box for the twistdrills	293
Dishes	292	Brain spatulas	121
Forceps	193, 291	<b>BREWER</b> , Clamp forceps	97
Osteotomes	292	Bridge and crown remover	339
Permanent filters	293	<b>BROPHY</b> , Surgical forceps	162
Plate cutters	291	<b>BROWN</b> , Surgical forceps	163
Plates	291	<b>BRUENINGS</b> , Tongue spatulas	117
Rack for screw magazines	293	<b>BÜCHS</b> , Plastic-filling instruments	83
Rack for twist drills	293	Raspatories	177
Screw driver blades	289, 292	Retractor for cheeks and soft tissue	128
Screw driver handles	289, 292	Soft tissue hooks	128
Screw magazines	287, 288, 291	Soft tissue retractors	323
Screw safety sleeves	289, 292	Tooth forceps	217, 218, 220, 222, 224, 225, 228-230
Set	293	<b>BUCHWALD</b> , Tongue spatulas	117
Sterile containers	293	Burnisher	88, 90
Twist drills	287, 288, 292	Ball burnisher/spatulas	71
Bone graft preparation forceps	291	<b>BUSER</b> , Bone raspatories	300
Bone harvesting, applicator for bone material	298	Micro-raspatories	171, 268
Bone file	300	Raspatories	57, 171, 269
Bone raspatories	300		
Bone rongeurs	299	<b>C</b>	
Membrane placing instruments	298	Calipers	332
Bone holding forceps	193	Carver	81
Bone processing forceps	193	Micro carver	73
Bone raspatories	300	Carving instruments	92
Bone rongeurs	194-196, 299	<b>CASTROVIEJO</b> , Needle holders	200
Bone split, applicator for bone split retractors	306	Cement spatulas	334, 335
Bone split retractors	306	<b>CHADWICK</b> , Delicate scissors	145
Kit	304	Chain with clip lock	115, 325
Mallet	301	Channeled retractors	130
Osteotomes	302, 303	Cheek and lip retractors	116
Rack for bone split retractors	306		
Retractors	306		
Spare plastic disks	301		

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## INDEXES

	Page		Page
Cheek retractors	116, 122, 125, 127, 322	Crown scissors	337
Children tooth forceps	232-235	Crown widening pliers	328, 338
American pattern	243	<b>CRYER</b> , Children tooth forceps, american pattern	243
Chin retractors	130	<b>CRYER</b> , Elevators	254
Chisels	183, 184, 186, 187	<b>CRYER</b> , Tooth forceps, american pattern	236, 241
Haemostatic chisels	186	<b>CRYER-WHITE</b> , Elevators	254
Mini chisels	60, 61	Curettes	23, 39, 66, 319
Chisel-osteotomes	186	After-GRACEY-curettes	34, 35
<b>CHOMPRET</b> , Syndesmotomes	261	Bone curettes	178-182, 270
Clamp forceps	97	Disk-shaped curette	66
Clamps, towel clamps	165	GRACEY-curettes	30-32
Cleaner, eloxal cleaner	377	Mini-GRACEY-curettes	36, 37
Cloth and bandage scissors	156	Recall set	41, 321
<b>COLUMBIA</b> , Universal curettes	28, 29	Rigid GRACEY-curettes	33
Combination clamp forceps	360	Universal curettes	27-29
Condenser, bone condenser	307, 308	<b>CUSHING</b> , Atraumatic forceps	160
Bone condenser kit	309	Cutters, distal end cutters	352
Containers	374	Hard wire cutters	352
Filter retainers	375	Ligature cutters	353
Paper filters	375	Pin- and wire cutters	347
Reusable filters	375	Wire cutters	349, 350
Seals	377	Wire- and pin cutters	347
Contra-angle set, plastic-filling instruments	84, 85	Cutting disks	296
<b>CONVERSE</b> , Needle holders	201	Cutting instruments	93
<b>COOLEY</b> , Atraumatic micro forceps	52, 101		
<b>COOPER</b> , Surgical scissors	136	<b>D</b>	
Coperring, temporary measure and crown removing pliers	338		
<b>COTTLE</b> , Chisel-osteotomes	186	<b>DARBY-PERRY</b> , Excavators	95
Mallets	191	<b>DE BAKEY</b> , Dissecting forceps	159
Osteotomes	186	Dissecting scissors	141
Cotton plugger	134	Needle holders	202
<b>COUPLAND</b> , Elevators	253	<b>De La ROSA</b> , Arch bending pliers	357
<b>CRANE CAPLAN</b> , Scalers	23	<b>DEAN</b> , Delicate scissors	145
<b>CRILE</b> , Haemostatic forceps	167	<b>DEAVER</b> , Surgical scissors	138
Needle holders	199	Delicate scissors	142-145, 150
<b>CRILE-MURRAY</b> , Needle holders	197	Desmotomes	260
<b>CRILE-WOOD</b> , Needle holders	201	Desmotomes/raspatories	57, 173, 261, 268
Crown and bridge remover	339	<b>DIADUST</b> <sup>®</sup> , telescope pliers	336
Crown depressor	338	Diagonal cheek retractors	128
Crown remover	339		
Crown removing, coperring and temporary measure pliers	338		

## ALPHABETICAL INDEX

	Page		Page
<b>DINGMANN</b> , Bone holding forceps	193	Ergofill, excavators	81
<b>DINGMANN, modif.</b> , Bone holding forceps	193	Moulding instruments	78
Dishes	213, 292	Pluggers	79
Round bowls	213	Spatulas	80
Disk-shaped curette	66	Ergoperio forceps	47-49, 103-105, 279-281
Disk-shaped elevator and dissectors	64	<b>ERGOPLANT</b> , Atraumatic forceps	160
Dissecting forceps	157-159	Forceps	159
Dissecting scissors	139-141, 146, 147, 153-155	Mouth wedges	115, 325
Dissection raspatories	60	Surgical forceps	163
Dissector, disk-shaped elevator and dissectors	64	Tongue retractors	117, 120
Periodontal dissectors	60	Vestibulum hooks	116
Dissector/raspatories	313	<b>ERGOPLANT</b> , applicator for bone material	298, 313
Distal end cutters	352	Applicator for bone split retractors	306
<b>DOCKHORN</b> , Retractors	129	Bone condenser	307, 308
<b>DOYEN-JANSEN</b> , Mouth gags	113	Bone condenser kit	309
Dressing and sponge forceps	164	Bone files	182, 300
Drill racks	382	Bone raspatories	300
Drills, box for the twistdrills	293	Bone split kit	304
Twist drills	287, 288, 292	Bone split retractors	306
<b>DUROGRIP®</b> , atraumatic forceps	160	Bone mill cleaner	296
Needle holders	53, 200-205, 276	Cheek retractors	322
Tooth tweezers	15	Cutting disks	296
<b>DUROTIP®</b> , delicate scissors	150	Mallets	191, 301
Gum scissors	149	Membrane placing instruments	298
Ligature scissors	152	Micro bone mill	296
Scissors	275	Micro bone mill kit	297
Surgical scissors	151	Non-traumatic dissecting forceps	277
Wire scissors	348	Osteotomes	302, 303
		Pair of spare disks	191
		Periotomes	260, 274
		Sinus elevators	310
		Sinutome kit	316
		Sinutomes	314, 315
		Soft tissue forceps	277
		Soft tissue retractors	323
		Spare plastic disks	301
		Spatulas	274
		Step-osteotomes	301
		Tissue forceps	277
		Tongue retractors	324
		Vestibulum hooks	323
<b>E</b>			
<b>EAST GRINSTEAD (WARD)</b> , Chisels	186		
Osteotomes	186		
Elevators, disk-shaped elevator and dissectors	64		
Sinus elevators	310		
Elevators	174, 311-313, 248-258		
Eloxa cleaner	377		
<b>ENGL. MOD.</b> , Towel clamps	165		
Ergofill, carver	81		

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## INDEXES

	Page		Page
Ethmoidal forceps	118, 119	Forceps, children tooth forceps, american pattern	243
Excavators	81, 93-95	Clamp forceps	97
Explorers	6-9	Combination clamp forceps	360
Micro explorer	67	Dissecting forceps	157-159
Exterior box	206	Dressing and sponge forceps	164
Extraction instruments	245-247	Ergoperio forceps	47-49, 103-105, 279-281
ExtraLux, extraction instruments	245-247	Ethmoidal forceps	118, 119
		Forceps for articulating paper	92, 332
		Haemostatic forceps	166-168, 275
		Ligature and wire forceps	169
		Micro dissecting forceps	52, 101
		Micro tissue forceps	52, 67, 101
		Non-traumatic dissecting forceps	277
		Pin holding forceps	74, 208
		Plate cutters	291
		Reposition forceps	192
		Root splinter forceps	231
		Separating forceps	223
		Soft tissue forceps	277
		Splinter forceps	208, 231
		Sponge and dressing forceps	164
		Surgical forceps	161-163
		Suture forceps	158, 200
		Tissue forceps	277
		Tissue grasping forceps	168
		Tooth forceps	217-230
		Tooth forceps for children	232-235
		Tooth forceps, american pattern	236-242
		Wire and ligature forceps	169
		<b>FOSTER FLAGG</b> , Plastic-filling instruments	87
		<b>FRAHM</b> , Finishing instruments	91
		<b>FRAZIER</b> , Retractors	132
		<b>FREER</b> , Chisels	184
		Raspatories	59, 173, 176, 267
		Retractors	132
		<b>FREER-YASARGIL</b> , Raspatories	174
		<b>FRETWURST</b> , Desmotomes/raspatories	57, 173, 261, 268
		Furcation probes	9
		<b>FURRER</b> ,	
		Coperring, temporary measure and crown removing pliers	338
<b>F</b>			
<b>FARABEUF</b> , Retractors	124		
<b>FEINER</b> , Tooth forceps	223, 230		
<b>FELSCH</b> , Tooth forceps	220		
<b>FERGUSSON</b> , Suction instruments	210		
<b>FERGUSSON-ACKLAND</b> , Mouth gags	113		
<b>FICKLING</b> , Haemostatic forceps	168		
Files, bone files	182, 300		
Filling remover	92		
Filters, paper filters	375		
Permanent filters	293		
Reusable filters	375		
Retainers	375		
Finishing instruments	90, 91		
Fixation pins	382		
Flat pliers	341, 363		
<b>FLOHR</b> , Elevators	252		
<b>FOMON</b> , Dissecting scissors	146		
Retractors	132		
Forceps	75, 159, 361		
Articulating paper	92, 332		
Atraumatic micro forceps	101		
Atraumatic forceps	159, 160		
Atraumatic micro forceps	52		
Bone cutting forceps	196		
Bone fixation forceps	193, 291		
Bone graft preparation forceps	291		
Bone holding forceps	193		
Bone processing forceps	193		
Children tooth forceps	232-235		

## ALPHABETICAL INDEX

	Page		Page
<b>G</b>			
GABKA, Elevators	256	Hard wire cutters	352
Gauze cases	134	HARRIS, Tooth forceps, american pattern	237
GERALD, Surgical forceps	162	HARTMANN, Nasal speculum	114
GILLIES, Needle holders	198, 203	Tongue spatulas	120
Retractors	132	HAUER, Medicine applicators	135, 208
Surgical forceps	162	Periodontometers	21
Gingival margin trimmer	92	HEGAR-MAYO, Needle holders	201
Gingivectomy knives	54, 62, 271	HEGAR-OLSEN, Needle holders	198, 203
GLICKMAN, Raspatories	60, 171	HEGAR-OLSEN modif. LANGE, Needle holders	203
GOLDMAN-FOX, Dissecting scissors	147	HEIDBRINK, Elevators	251
Gum scissors	149	HEIDEMANN, Spatulas	80, 88, 274
Scalers	27	HEISTER, Mouth gags	113
Universal curettes	28	HEMINGWAY, Bone curettes	179, 180
GOSLEE, Flat pliers	341, 363	Holder, alastik holders	361
Gouges	183, 184	Holding device for identification labels	41, 321, 382
GRACEY, Curettes	30-32	HOLLENBACK, Finishing instruments	90
GRAEFE, Retractors	132	Plastic-filling instruments	86
GRAY, Tooth forceps for children	233	Hooks	177
Grinding Stones	42	Soft tissue hooks	128
Arkansas grinding stone	54, 273	Vestibulum hooks	116, 323
Oil	42, 54, 273	HOPSON, Burnisher	88
GROSS, Dressing and sponge forceps	164	Plastic-filling instruments	86
Gum scissors	148, 149, 275	HÖSEL, Needle holders	205
GUY, Tooth forceps	218	HOWARTH, Raspatories	176
<b>H</b>			
HAASE, Reposition forceps	192	HULL, Children tooth forceps, american pattern	243
Haemostatic chisels	186	Tooth forceps, american pattern	236
Haemostatic forceps	166-168, 275	HYLIN, Elevators	252
HAJEK, Mallets	191	<b>I</b>	
HALLE, Elevators	174	Identification labels	41, 321, 376
HALSEY, Needle holders	197, 201	Holding device for identification labels	41, 321, 382
HALSTED, Haemostatic forceps	168	Impression trays	331
HALSTED-MOSQUITO, Forceps	361	Indicator labels	377
Haemostatic forceps	166, 275	Instrument oil	383
Handle for micro mirrors and microsurgery blades	69	Instrument racks	381, 382
		Interior box	206
		Intestinal and abdominal spatulas	121
		IVORY, Rubber-dam punches	96

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## INDEXES

	Page		Page
<b>J</b>		<b>L</b>	
JACOBSON, Bone curettes	181	LA GRANGE, Delicate scissors	145
JAMESON, Dissecting scissors	141	Labels, identification labels	376
JAMESON-WERBER, Dissecting scissors	140	Indicator labels	377
JANSEN, Bone curettes	180, 181	LADMORE, Plastic-filling instruments	86
JARABAK, Light wire pliers	340, 358	LAMBOTTE, Osteotomes	183
JESCO®, Impression trays	331	LANGE, Raspatories	58, 172
JONES, Towel clamps	165	LANGENBECK, Needle holders	204
JOSEPH, Delicate scissors	144	Retractors	124, 125
Retractors	132	LANGENBECK-GREEN, Retractors	133
		LANGER, Curettes	39, 319
		Universal curettes	27
		LASTER, Cheek retractors	127
		LEXER, Dissecting scissors	146
		LICHTENBERG, Needle holders	205
		Lid for trays	380
		Ligature and wire forceps	169
		Ligature cutters	353
		Ligature scissors	152, 156
		Light wire pliers	340, 358
		LINDO-LEVIAN, Elevators	253
		Lip and cheek retractors	116
		LISTER, Bandage and cloth scissors	156
		Sponge and dressing forceps	164
		LITTAUER, Ligature scissors	156
		LOCKLIN, Dissecting scissors	147
		Gum scissors	275
		LONDON HOSPITAL COLEMAN, Elevators	256
		LONDON-COLLEGE, Tooth tweezers	13, 14
		LONDON-COLLEGE, mod., Tooth tweezers	14
		Loop bending pliers	346, 356
		LORNA, Towel clamps	165
		LUCAS, Bone curettes	178, 270
		Chisels	184
		Gouges	184
		LUER, Bone rongeurs	194
		LUER-FRIEDMANN, Bone rongeurs	194, 299
		LUNIATSCHEK, Cotton plugger	134
		LUNIATSCHEK, modif., Cotton plugger	134
		Luxators	259
<b>K</b>			
KAISER, Tooth forceps	221, 223, 230		
KAY (AUSTIN), Retractors	127		
KELLY, Delicate scissors	144		
Haemostatic forceps	167		
KELSEY FRY, Awls	169		
KILNER, Dissecting scissors	140		
Needle holders	202		
Retractors	132		
KIRKLAND, Gingivectomy knives	62, 271		
Mini chisels	61, 185		
KLEIN, Children tooth forceps	234, 235		
Tooth forceps for children	234, 235		
Knives, gingivectomy knives	54, 62, 271		
KOCHER, Haemostatic forceps	167		
KOCHER-LANGENBECK, Retractors	124		
KÖHLER, Triangular bending clamp pliers	346		
KÖNIG, Elevators	174		
Raspatories	175		
KORKHAUS, Wire and ligature forceps	169		
KRAMER-NEVINS, Mini chisels	61, 185		

	Page		Page
<b>M</b>			
Mallets	190, 191, 301	MICRO-ADSON, Surgical forceps	163
Pair of spare disks	190	MICRO-HALSTED, Haemostatic forceps	166
Spare plastic disks	301	Micro-Raspatories	171, 268
Mandibular awls	169	Microblade holders	265
Marking pens	377	Microsurgery blades	45, 107
MARTINI, Bone curettes	181	Microsurgical blades	265
MATHIEU, Needle holders	199, 205	MIDDELDORPF, Cheek retractors	122, 125
MATHIEU-STANDARD, Needle holders	205	MILLER, Bone curettes	182
Matrix retainers	98	Bone files	182, 300
Maxillary awls	169	Mini chisels	60, 61, 185
Maxillary mobilizers	130	Mini trays	380
MAYO-HEGAR, Needle holders	198	Mini-GRACEY, Curettes	36, 37
MC INDOE, Surgical forceps	162	MINNESOTA, modif., Cheek retractors	116
McCALL, Scalers	25	Mirror, handle for micro mirrors and microsurgery blades	69
Universal curettes	29	Micro mirrors	68
MEAD, Bone rongeurs	196	Mouth mirrors	3, 266
Mallets	190	Mouth mirror handles	3, 4, 266
Pair of spare disks	190	MOLT, Mouth gags	113
Raspatories	176	Raspatories	59, 172, 267
Medicine applicators	135, 208	Moulding instruments	78
Membrane placing instruments	298	Mouth gags	113, 114
MERIAM, Tooth tweezers	12, 13	Mouth mirrors	3, 266
MERSHON, Crown depressor	338	Handles	3, 4, 266
METZENBAUM, Dissecting scissors	139, 153-155	Mouth wedges	115, 325
Surgical scissors	151	Chain with clip lock	115, 325
Micro bone mill	296	MÜHLING, Raspatories	177
Micro bone mill kit	297		
Micro carver	73	<b>N</b>	
Micro dissecting forceps	52, 101	NABERS, Furcation probes	9
Micro explorer	67	NANCE, Step loop bending pliers	357
Micro ligature scissors	51, 100, 282	Nasal speculum	114
Micro mirrors	68	Needle attachments	211, 212
Micro needle holders	53, 101, 200, 282	Needle cases	206
Micro scalpels	106	Exterior box	206
Micro scissors	100	Interior box	206
Spring type	51	Needle holders	53, 197-205, 276
Micro soft tissue set	285	Micro needle holders	53, 101, 200
Micro tissue forceps	52, 67, 101	Nerve instruments	208, 231
MICRO-ADSON, Dissecting forceps	157		

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## INDEXES

	Page		Page
Nerve instruments, forceps	75	Pair of spare disks	190, 191
<b>NEVIUS</b> , Tooth forceps, american pattern	238	<b>PALMER</b> , Clamp forceps	97
<b>NIEDERDELLMANN</b> , Osteotomes	186	Paper filters	375
Noir®, Dissecting scissors	153-155	Papilla elevators	58
Non-traumatic dissecting forceps	277	Atraumatic papilla elevators	269
<b>NORTHBENT</b> , Ligature scissors	156	<b>PARMLY</b> , Tooth forceps, american pattern	239
<b>NYSTRÖM</b> , Matrix retainers	98	<b>PARTSCH</b> , Chisels	183
		Gouges	183
		Raspatories	177
		<b>PASSOW</b> , Haemostatic chisels	186
		Tampers	186
		<b>PEAN</b> , Haemostatic forceps	166, 167
<b>O</b>		<b>PEESO</b> , Wire bending pliers	344
<b>OBWEGESER</b> , Awl for zygomatic arch	169	<b>PEET</b> , Splinter forceps	231
Channeled retractors	130	Periodontal dissectors	60
Chin retractors	130	Periodontal probes	320
Ligature and wire forceps	169	Periodontometers	10, 11, 18-21, 40
Mandibular awls	169	Periotomes	260, 274
Maxillary awls	169	Permanent filters	293
Pterygomaxillary osteotomes	186	<b>PERRY</b> , Tooth tweezers	12
Raspatories	175	<b>PHW</b> , Atraumatic papilla elevators	269
Retractors	131	Papilla elevators	58
Soft tissue retractors	126	<b>PHYSICK</b> , Tooth forceps, american pattern	242
Spina-nasalis retractors	129	Pin holding forceps	74, 208
Wire and ligature forceps	169	Pin- and wire cutters	347
<b>OBWEGESER-FREER</b> , Elevators	174	<b>PLANERT</b> , Crown widening pliers	328, 338
<b>OCHSENBEIN</b> , Mini chisels	61, 185	Plastic-filling instruments	82-87
Oil, instrument oil	383	Burnisher	88
for grinding stones	42, 54, 273	Spatulas	88
Spray	383	Plate cutters	291
<b>ORBAN</b> , Gingivectomy knives	54, 62, 271	Plates	291
Orthodontic probes	362	<b>PLESTER</b> , Suction instruments	210
Osteotomes	183, 186, 188, 189, 292, 302, 303	Pliers, arch bending pliers	357
Chisel-osteotomes	186	Band removing pliers	359
Pterygomaxillary osteotomes	186	Bird beak pliers	340, 358
Step-osteotomes	301	Coperring, temporary measure and crown removing pliers	338
		Crown widening pliers	328, 338
		Flat pliers	341, 363
		Light wire pliers	340, 358
		Loop bending pliers	346
		Loop forming pliers	356
<b>P</b>			
Packer, retraction thread packers	135, 333		

## ALPHABETICAL INDEX

	Page		Page
Pliers, round pliers	343	Raspatories, dissection raspatories	60
Round-nosed pliers	343	Micro-raspatories	171, 268
Separating pliers	359	Raspatories/desmotomes	57, 173, 261, 268
Step loop bending pliers	357	Raspatories/dissectors	313
Telescope pliers	336	<b>READ</b> , Tooth forceps	221
Temporary measure, crown removing and coperring pliers	338	Recall set	41, 321
Three-finger pliers	342, 364	<b>REILL</b> , Reposition forceps	192
Torque pliers	355	Wire cutters	350
Triangular bending clamp pliers	346	<b>REINWALD</b> , Hooks	177
Universal orthodontic pliers	354	Reposition forceps	192
Wire bending pliers	340, 344, 345, 358	Retraction thread packers	135, 333
Pluggers	70, 72, 79	Retractors	65, 123-125, 127, 129, 131-133
Cotton pluggers	134	Bone split retractors	306
Press pads	381	Channeled retractors	130
<b>PRICHARD</b> , Raspatories	56, 170, 266, 267	Cheek and lip retractors	116
Probes	207	Cheek and soft tissue retractors	128
Furcation probes	9	Cheek retractors	116, 122, 125, 127, 322
Orthodontic probes	362	Chin retractors	130
Process identification labels	376	Diagonal cheek retractors	128
Pterygomaxillary osteotomes	186	Lip and cheek retractors	116
Punches, rubber-dam punches	96	Ramus retractors	129
Pusher/scaler	362	Retractors for cheeks and soft tissue	128
		Soft tissue retractors	126, 323
		Spina-nasalis retractors	129
		Tongue retractors	117, 120, 324
		Reusable filters	375
		Marking pens	377
<b>Q</b>		<b>REYNOLDS</b> , Dissecting scissors	141
<b>QUINBY</b> , Dissecting scissors	147	<b>RHODES</b> , Mini chisels	60, 61, 185
		<b>RICHTER</b> , Retractors	65
		<b>Rigid GRACEY</b> , Curettes	33
		<b>ROBERTS</b> , Raspatories	176
		Rongeurs, bone rongeurs	194-196, 299
		Root splinter forceps	231
		<b>ROSER-KÖNIG</b> , Mouth gags	114
		<b>ROSIN</b> , Luxators	259
		Round bowls	213
		Round pliers	343
		Round-nosed pliers	343
		<b>ROUTURIER</b> , Tooth forceps	227
		<b>ROUX</b> , Cheek retractors	122
Rack, box for the twistdrills	293		
for bone split retractors	306		
for drills	382		
for screw magazines	293		
for twist drills	293		
Ramus retractors	129		
Raspatories	56-60, 170-177, 266, 267, 269		
Bone raspatories	300		

## ALPHABETICAL INDEX

	Page		Page
Sponge and dressing forceps	164	Telescope pliers	336
Spray, oil spray	383	Temporary measure, crown removing and coperring pliers	338
<b>STAMM</b> , Bone cutting forceps	196	<b>TESSIER</b> , Maxillary mobilizers	130
<b>STANDARD</b> , Atraumatic forceps	160	Osteotomes	189
Dissecting forceps	158	Test rods	42
Surgical forceps	161	Three-finger pliers	342, 364
<b>STELLBRINK</b> , Bone rongeurs	195	Tissue forceps	277
Step loop bending pliers	357	Tissue grasping forceps	168
Step-osteotomes	301	<b>TOBOLD</b> , Tongue spatulas	120
Sterile containers	293	<b>TOENNIS</b> , Needle holders	199, 204
Sterile scalpels	112	<b>TOENNIS-ADSON</b> , Dissecting scissors	146
Sterilit <sup>®</sup> i, instrument oil	383	<b>TOFFLEMIRE</b> , Matrix retainers	98
Oil for grinding stones	42, 54, 273	<b>TOMES</b> , Tooth forceps, american pattern	239
Oil spray	383	Tongue retractors	117, 120, 324
<b>STERNBERG</b> , Cheek and lip retractors	116	Tongue spatulas	117, 120
<b>STIEGLITZ</b> , Splinter forceps	208, 231	Tooth forceps	217-230
Storages, non-slip	380	for children	232-235
<b>STRICKER</b> , Applicator for bone material	298, 313	american pattern	236-242
Applicator for bone split retractors	306	Tooth tweezers	12-15
Bone fixation forceps	193, 291	Torque pliers	355
Membrane placing instruments	298	Towel clamps	165
Sinus elevators	310	Trays	41, 321, 378-380
Spatulas	274	Fixation pins	382
Step-osteotomes	301	Holding device for identification labels	382
Tunnelling instruments	55, 272, 273	Instrument racks	381, 382
Suction instruments	209, 210	Lid for trays	380
Surgery blades,		Mini trays	380
handle for micro mirrors and microsurgery blades	69	Impression trays	331
Surgical forceps	161-163	<b>TREYMANN</b> , Crown remover	339
Surgical scissors	136-138, 151	Triangular bending clamp pliers	346
Suture forceps	158, 200	<b>TROTTER</b> , Tooth forceps	217, 225
<b>SYME</b> , Raspatories	174	Tunnelling instruments	55, 272, 273
Syndesmotomes	261	<b>TWEED</b> , Torque pliers	355
Syringes	211, 212	Tweezers, tooth tweezers	12-15
Needle attachments	211, 212	Twist drills	287, 288, 292

## T

Tampers 186

## U

**UFFENORDE**, Bone curettes 181

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## INDEXES

	Page		Page
Universal curettes	27-29	WITZEL modif., Root splinter forceps	231
Universal orthodontic pliers	354	WOOD, Tooth forceps	226
		WOODSON, Plastic-filling instruments	86
		WÖSTMANN, impression trays	331
<b>V</b>			
Vestibulum hooks	116, 323	<b>Y</b>	
VOLKMANN, Bone curettes	181	YOUNG, Loop bending pliers	346
<b>W</b>			
WALLIS, Burnisher	88	<b>Z</b>	
WARD, Cheek retractors	127	ZAUFAL-JANSEN, Bone rongeurs	196
Finishing instruments	91	ZIELINSKY, Calipers	332
Raspatories	177		
WARWICK-JAMES, Elevators	256		
WASSMUND, Retractors	129		
WAUGH, Surgical forceps	162		
WEIL BLAKESLEY, Ethmoidal forceps	118		
WEINGART, Universal orthodontic pliers	354		
WESTON, Explorers	9		
WHITE, Burnisher	90		
Elevators	250		
Excavators	95		
Plastic-filling instruments	87		
WHO, Periodontal probes	320		
Periodontometers	10, 11, 18, 19, 40		
WILAND, Carver	81		
WILLIAMS-FOX, Periodontometers	21		
WILLIGER, Bone curettes	181		
Elevators	174		
Raspatories	175		
WINTER, Elevators	255, 257		
Wire and ligature forceps	169		
Wire bending pliers	340, 344, 345, 358		
Wire cutters	349, 350		
Wire scissors	348		
Wire- and pin cutters	347		

ALPHABETICAL INDEX

Page

Page

NOTIZEN  
NOTICE

A large rectangular area filled with horizontal dotted lines, intended for taking notes. The lines are evenly spaced and extend across the width of the page.

# AESCULAP<sup>®</sup> – a B. Braun brand

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany  
Phone +49 7461 95-0 | Fax +49 7461 95-2600 | [www.aesculap-dental.com](http://www.aesculap-dental.com)

The main product trademark "Aesculap" and the product trademarks "DIADUST", "DUROGRIP", "DUROTIP", "JESCO", "Noir", "Sterican" and "Sterilit" are registered trademarks of Aesculap AG.

Subject to technical changes. All rights reserved. This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.