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**INSTRUCTION FOR CLEANING AND STERILIZATION OF AUTOCLAVABLE MULTI-USE-SHAVERBLADES (MAX. 5X)**

**MDD multi-use shaverblades for the Arthroscopy are multi-use-devices (max. 5x), meant to generally be shipped non-sterile and being fully autoclavable.**

Please always pay attention to the following safety instructions and rules:

- The cutting edges have to be in proper order
- Speed referral for cutters 800 – 1500 U/min; für burrs 2500 -3500 U/min
- Blades, meant to be used for soft tissue may not be used for bones
- Blades, meant to be used for bones may not be used for soft tissue
- Generally before every application the inner and the outer shaft have to be checked if there might be any damages
- Every damaged Blades may not be used in no case and has to immediately be replaced by a non-damaged one

**CLEANING:**

- For the intended purpose of cleaning the inner shaft has to be separated from the outer one
- The individual parts have to be cleaned by rising with acid or basic solutions
- Eventual dirt has to be cleaned by using eligible pH neutral solution and cleaning brushes or by using a dishwasher or ultrasonic cleaners.

**STERILIZATION:**

- Steam sterilization (Shaverblade in sterile wrap packs) by 134 ° Celsius
- The operator of the sterilization unit and facility is responsible for all kind of sterilization methods and terms
- It should generally be used a validated and approved sterilization method run by highly skilled and trained staff
- Every Shaverblade has to be sterilized in front of its use.

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### How to insert the shaverblade into the shaver- handpiece

- 1 turn the adjusting ring (see righthward arrow ) to the left until it blocks
- 2 insert the shaverblade into the shaver- handpiece;  
by doing so please move the metal pin (see leftward arrow) in its slot of the shaver- handpiece
- 3 turn the adjusting ring (see righthward arrow ) to the righth until it blocks

### How to remove the shaverblade

- 1 turn the adjusting ring (see righthward arrow ) to the left until it blocks
- 2 pull out the shaverblade

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Internet [www.biq-medical.com](http://www.biq-medical.com)

Manufacturer:

MDD Medical Device Development GmbH

Steigäcker 20 / Industrial Area

D-78582 Balgheim